Flairesse

Prophylaxepaste

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English

Product description

Tooth cleaning and polishing paste for professional tooth cleaning and polishing of the tooth surface as part of prophylaxis treatment. Flairesse prophylaxis paste comes in a Unit Dose or a tube and each is available in two different flavors. The pastes' three different abrasion grades enable you to respond to each patient's situation, depending on the level of cleaning required.

Indications

- Removal of superficial soft and hard plaque.
- Removal of extrinsic stains (e.g. stains from coffee, tea or tobacco).

Contra-indications

Do not use the material

- in cases of allergies to any of the ingredients or in cases of contact allergies,
- if the patient's control over their swallowing reflex is not guaranteed.

Basic safety instructions

- For dental use only!
- Keep out of reach of children!
- Do not swallow!
- Avoid contact with the skin! In the event of inadvertent skin contact wash the affected area thoroughly with soap and water.
- Avoid contact with the eyes! In the event of inadvertent contact with the eyes, immediately rinse eyes thoroughly with plenty of water and consult a physician if necessary.

Side effects

Swallowing excessive amounts of the material may cause nausea.

Application system

- Unit Dose
- Tube

Instructions for use

- ▶ Any commercially available prophylaxis angle or polishing cup may be used.
- ▶ DMG recommends using the prophylaxis pastes one after the other (decreasing the abrasion grade). Use brushes for occlusal surfaces.
- ▶ The Unit Dose and the tube are fitted with a tamper-proof seal. Make sure that the seal is intact before first use.

Recommended use

Caution! Incorrect use may result in overheating.

- ▶ Ensure that there is sufficient material in the polishing cup used.

Caution! Risk of swallowing if saliva suction insufficient.

- ▶ Saliva must be suctioned away thoroughly during treatment.

Using the paste from the Unit Dose

Please note: Each Unit Dose holds enough material for one prophylaxis treatment. Each Unit Dose may only be used for one individual patient.

1. Peel off the tamper-proof seal from the Unit Dose using the tag and discard.

2. Insert the Unit Dose into the appropriate prophylaxis holder.
3. To fill, set the polishing cup to low speed, hold it in the prophylaxis paste and pick up the material.
4. Use the polishing cup to work in the prophylaxis paste for 20s per tooth at 3000 to 5000rpm.
5. Suction away excess prophylaxis paste and get the patient to spit it out.

Applying the paste from the tube.

Please note: To prevent the material from drying out, close the tube as soon as you have expressed the material.

1. When using a tube for the first time, unscrew the cap, peel off the tamper-proof seal using the tag and discard.
2. Express a sufficient quantity of the material onto a mixing block.
3. To fill, set the polishing cup to low speed, hold it in the prophylaxis paste and pick up the material.
4. Use the polishing cup to work in the prophylaxis paste for 20s per tooth at 3000 to 5000rpm.
5. Suction away excess prophylaxis paste and get the patient to spit it out.

Ingredients

Pumice, glycerine, water, o-phosphoric acid (< 2 %), flavors and scents, additives, sodium fluoride (2.72 %, corresponds to 1.23 % fluoride ions).

Free from aspartame, gluten, saccharine.

Contains Xylitol.

Storage and shelf life

- Store at room temperature (15 to 25 °C/59 to 77 °F)!
- Do not store in the refrigerator!
- Protect against exposure to direct sunlight!
- Do not use after the expiration date!

Available product sizes

Unit Dose

One pack contains 200 individual portions @ 1.8 g. Available in the following flavors and grit sizes:

Mint	
Fine grit	REF 220414
Medium grit	REF 220415
Coarse grit	REF 220416

Melon	
Fine grit	REF 220420
Medium grit	REF 220421
Coarse grit	REF 220422

Tube

One pack contains 1 tube @ 75 ml. Available in the following flavors and grit sizes:

Mint	
Fine grit	REF 220417
Medium grit	REF 220418
Coarse grit	REF 220419

Melon	
Fine grit	REF 220423
Medium grit	REF 220424
Coarse grit	REF 220425

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