# Venus® Bulk Fill

- Gebruiksaanwijzing**
- Instructions for use**
- Mot de emploi**
- Instrucciones de uso**
- Istruzioni per l'uso**
- Instruções de utilização**
- Gebruiksaanwijzing**

86240798/001 – 11.10.2023

Venus® Bulk Fill

## Venus® Bulk Fill

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## Venus® Bulk Fill

**Product description**
Venus Bulk Fill is a posterior flowable, light-curing, radiopaque nano-hybrid composite. It is used as a base in Class I and II restorations and can be placed in 4 mm increments. Venus Bulk Fill's flow on demand characteristics enable the material to adapt to the cavity walls. The universal shade allows curing of 4 mm layer thickness (base fill) with low shrinkage stress. A universal restorative material is used in conjunction with Venus Bulk Fill as a facial/oral/occlusal enamel replacement. Venus Bulk Fill is compatible with methacrylate-based restorative materials designed for Class I and II occlusal restorations (e.g. Venus Diamond® see Application).

**Composition**
Venus Bulk Fill is based on multifunctional methacrylate monomers (UDMA, EBADMA) and contains approximately 65% w/w and 38% vol inorganic fillers, such as Ba-Al-F-silicate glass, YbF<sub>3</sub> and SiO<sub>2</sub>. The filler particle size is between 0.02 µm and 5 µm.

**Color**
Universal shade

- Indications**
- Base in Class I and II direct restorations (up to 4 mm)
- Curing under direct restorative materials in Class I and II

**Contraindications**
Do not use in cases of a known allergy to any of the constituents of Venus Bulk Fill.

Application to the dentin less than 1 mm to the pulp

**Side effects**
Allergies to the product or its constituents may occur in isolated cases. If an allergy is suspected, request a list of ingredients from the manufacturer.

**Warnings / precautions**
Avoid contact with skin or eyes. May cause sensitization in case of skin contact. May irritate eyes. In case of contact with skin, wash skin with soap and water. In case of contact with eyes, rinse with plenty of water and seek medical attention.
PLTs are intended for single use. To avoid cross-contamination between patients, PLTs must be discarded after a single use!

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info.dent@heraeus.com
www.heraeus-dental.com

Made in Germany

Distributed in USA/Canada exclusively by:
Heraeus Kulzer, LLC
300 Heraeus Way
South Bend, IN 46614-2517
1-800-431-1785

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For deep cavities with only a minimal layer of dentin over the pulp or those which have already had a small area of pulp exposed, the pulp must be indirectly or directly capped using calcium hydroxide. The calcium hydroxide should be covered with a glass ionomer cement before Venus Bulk Fill is applied to the cavity.
Prepared enamel and dentin surfaces on which Venus Bulk Fill will be adhesively luted must not come in contact with preparations containing eugenol, as these could inhibit optimal polymerization of the composite.
Deviations from the instructions for use are the sole responsibility of the dentist. The product must be used solely in accordance with the instructions for use.

**Application**
NOTE: The shade selection of the final composite should be selected using the shade guide before inserting the dental dam as teeth noticeably change color when dried.

1. Kariesentfernung und zahnrattsubstanzonahene Präparation entfernt den allgemeinen Regeln der adhäsiven Füllungstherapie. Anschließend Kavität spülen, reinigen und trocknen, ohne die Zahnoberfläche zu stark aus zu trocknen.
2. Use a dental dam (e.g. Ivory® Rubber Dam) to isolate the treatment area from contamination. Refer to manufacturer's directions for placement.
3. Shape the proximal contacts with the aid of a matrix system. Using a matrix band with a wedge serves two purposes: compressing matrix band against the tooth structure to create a tighter seal, preventing fluid overfill and provides slight separation between the teeth, compensating for the thickness of the matrix band, therefore helping create tight contact between the finished restoration and the adjacent tooth. Therefore, a thin matrix band is recommended. The matrix band should be burnished, or shaped, to achieve a contour facilitating its proper adaptation to the tooth structure. The matrix band should also be shaped while applying the composite, because Venus Bulk Fill does not exert pressure during application.
4. An appropriate adhesive system (e.g., IBOND® Self Etch, IBOND® Total Etch) should be applied to the enamel and dentin according to the manufacturer's instructions for use before filling the matrix. It is the responsibility of the dental practitioner if another dentist/named adhesive product is utilized. Adhesive-treated cavities must not be contaminated with moisture, blood or saliva.
5. Begin adhering Venus Bulk Fill directly into the cavity preparation at the deepest point, keeping tip submerged. When applying, ensure that it buries slowly and evenly to prevent the formation of air bubbles. Immerse the PLT tip into the material and fill the cavity from bottom to top. If necessary, use a probe or a micro-brush to remove any air bubbles and excess material before the application. The distance to the occlusal margin of the cavity must be at least 2 mm.
6. Alternative application would be to use Venus Bulk Fill in a thin layer as a liner on exposed dentin.

Venus Bulk Fill is designed to be cured in increments up to 4mm in depth. Polymerize Venus Bulk Fill for 20 seconds, on each of the restoration surface with a curing light (e.g., Translux® Power Blue) in accordance with the table below (light output > 550 mW/cm²).

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