Instructions for use
3M™ Comply™ Disposable Bowie and Dick Type Autoclave Test Pack
INTRODUCTION

The 3M™ Comply™ Disposable Bowie and Dick Type Autoclave Test Pack is used in the daily Bowie and Dick Test for Porous Load Autoclave **[STEAM]** processes and consists of an indicator test sheet and two blocks of paper sheet material. The entire assembly is enclosed within a paper wrapper, secured with tape. The wrapper must remain unopened until the end of the test. An indicator label is present on the pack and will change from red to purple when the pack has been processed in the sterilizer. The 3M™ Comply™ Disposable Bowie and Dick Type Autoclave Test Pack is designed to function in a Bowie and Dick test cycle employing **[STEAM]** at 134 to 137°C for 3.5 minutes. When exposed to **[STEAM]** at 134 to 137°C for 3.5 minutes the indicator sheet changes from red to purple.

Use of an inappropriate test cycle may lead to erroneous results.

The 3M™ Comply™ Disposable Bowie and Dick Type Autoclave Test Pack must not be used for more than a single test.

PERFORMING THE TEST

- Ensure that the warm up cycle has been completed.
- Place the test pack in the chamber, printed face uppermost. Position the pack centrally on the bottom shelf, approximately 100 mm to 200 mm above the chamber base (See figure 1).
- Conduct the Bowie and Dick Autoclave test.
- At the end of the autoclave cycle the pack will be hot, therefore wear protective gloves when handling.
- Open the wrapper and remove the indicator test sheet from the centre of the pack.

INTERPRETATION

Examine the sheet in a good light source.

If the test is a PASS the indicator ink will be purple across its entire surface (See figure 2).

If the test is a FAIL, the centre of the indicator ink pattern will remain red (See figure 2).

RETENTION OF RESULTS

After use the indicator should be stored in a cool, dry, dark place and in the catalogue no 1300R Record File. The test result forms part of the sterilizers daily qualification program. Results should be filed for reference in accordance with local procedures. The test sheet may darken during storage, but this is a natural aging process and should not be a cause for concern.

NOTE:
Symbols used on the 3M™ Comply™ Disposable Bowie and Dick Type Autoclave Test Pack and its indicator test sheet are as follows:
[LOT]  = The Lot Number and the Expiry Date

Take one of the indicator sheets and record in the spaces provided:

a) Symbol * date b) Symbol hospital c) Symbol department
d) Symbol machine number e) Symbol cycle number f) Symbol Date of manufacturing

(* For these symbols refer to the symbol page within this leaflet.)

Date of issue: 2015-04

PRECAUTIONS:

- Wear heat protective gloves when removing the test pack from the autoclave and during disassembly.
- The test sheet must be protected from light during storage. Failure to observe this precaution may result in fading of the ink colours before and after use.

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