

DISINFECTANT SOLUTION FOR MEDICAL DEVICES IN DENTAL ENVIRONMENTS

DENTASEPT SPRAY 60 PRO

DISINFECTANT SOLUTION

Fast disinfection of previously cleaned, non-immersible, non-invasive, alcohol-resistant, medical equipment and medical devices in dental environment.

Ready to use solution

- Ethanol-based composition
- Broad activity spectrum in short contact time
- Does not leave any trace when drying
- Pleasantly fragranced
- Without dye



DENTASEPT SPRAY 60 PRO

Disinfectant solution

PRODUCT CHARACTERISTICS


- Ready to use product
- Clear colourless solution
- Density: approx 0.9
- pH at +20°C: approx 5.5

INSTRUCTIONS FOR USE


Ready-to-use.




Use on previously cleaned medical devices. Before application, check the compatibility of the areas to be treated with alcohol. Electrical devices being disinfected must be switched off.

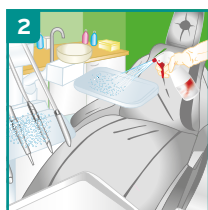

1 L Spray Bottle: Spray on the areas to be treated until they are totally moistened.
5 L can: Wet the areas to be treated until they are totally moistened. Apply in sufficient quantity (+/- 40 ml/m²). Complete wetting of all devices to be disinfected is important.

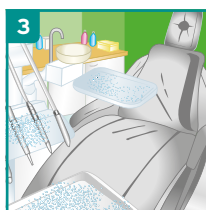

Respect the indicated contact time for the required antimicrobial activity. Rinsing is unnecessary except for areas intended to come into contact with skin or mucosa.

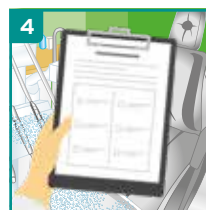

For the frequency of use, refer to the hygiene plan in place. Please refer to the manufacturer's instructions for use for disinfecting medical devices.

MICROBIOLOGICAL PROPERTIES

CLAIM/CONDITIONS	CONTACT TIME
Bactericidal* - According to EN 13727 (30 sec), EN 17387 (1 min)	1 min
Yeasticidal* - According to EN 13624/EN 17387	30 sec
Fungicidal* - According to EN 13624/EN 17387	5 min
Tuberculocidal* - According to EN 14348	30 sec
Mycobactericidal* - According to EN 14348	5 min
Active against enveloped viruses* - According to EN 14476/EN 16777	30 sec
Limited spectrum virucidal* - According to EN 14476/EN 16777	5 min
Adenovirus	30 sec
Norovirus (MNV)	5 min

* Clean conditions

ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
Dentasept Spray 60 Pro	3 x 1 L spray bottle	XXXXXX
Dentasept Spray 60 Pro	1 x 5 L can	XXXXXX



D.M.D.
1, Rue de l'Espoir
59260 Lezennes
France

Distributed by:
LABORATOIRES ANIOS
1, Rue de l'Espoir
59260 Lezennes
France

APPLICATION AREA

Fast disinfection of previously cleaned, non-immersible, non-invasive, alcohol-resistant medical equipment and medical devices in dental environment.

INSTRUCTIONS FOR USE

Ready-to-use.

1. Use on previously cleaned medical devices. Before application, check the compatibility of the areas to be treated with alcohol. Electrical devices being disinfected must be switched off.
2. 1 L Spray Bottle: Spray on the areas to be treated until they are totally moistened.
5 L can: Wet the areas to be treated until they are totally moistened. Apply in sufficient quantity (+/- 40 ml/m²). Complete wetting of all devices to be disinfected is important.
3. Respect the indicated contact time for the required antimicrobial activity. Rinsing is unnecessary except for areas intended to come into contact with skin or mucosa.
4. For the frequency of use, refer to the hygiene plan in place. Please refer to the manufacturer's instructions for use for disinfecting medical devices.

INGREDIENTS

Ethanol, N,N-Didecyl-N-methyl-poly(oxyethyl) ammonium propionate, perfume.

CERTIFICATES

Medical Device. Full antimicrobial activity for Dentasept Spray 60 Pro is included in the scientific dossier, available on request.

STORAGE

Storage: between +5°C and +25°C.

MEDICAL DEVICE USE SAFELY

Please read label and product information.



FOR PROFESSIONAL USE ONLY.



www.anios.com