Sani-Tip® Disposable Air/Water Syringe Tip

Embouts jetables pour seringues air/eau
Puntas desechables para agua/aire
Einwegeinsätze für Luft-/Wassersprays
Puntali per siringa aria-acqua monouso

Sani-Shield® Disposable Covers*

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Manufactured For:
Dentsply LLC
1301 Smile Way
York, PA 17404 USA
www.dentsplysirona.com

Dentsply DeTrey GmbH
De Trey Strasse 1
Konstanz D-78457
Germany

See website for symbols glossary

www.dentsplysirona.com/IFU

Made in China

120313 Rev 04/23

Instructions For Use

Mode d'emploi / Instrucciones de uso / Gebrauchsanleitung / Istruzioni per l'uso

* Quantity optional / Quantité en option / Cantidad opcional / Wahlweise je nach Menge / Quantità opzionale

Sani-Tip® Disposable Air/Water Syringe Tip

INDICATIONS FOR USE

Used to transmit air or water to the operatory field during operative procedures.

CONTRAINDICTIONS

Not for use in sterile procedure environments.

WARNINGS

- Sani-Tip® syringe tips are designed to function exclusively with air/water syringes converted using a Sani-Tip® Conversion Kit. If your syringe has not been converted for Sani-Tip® disposable tip use, the appropriate Sani-Tip® Conversion Kit MUST be installed before proceeding. Contact your dealer for a complete list of available conversion kits.
- Use only adapter components and clear Sani-Tip plastic syringe tips manufactured by Dentsply Sirona for the Sani-Tip® System to ensure optimal performance and safety including reducing potential for cross-contamination. Follow the enclosed instructions.
- For SINGLE USE ONLY. Disposable tips are not autoclavable. Dispose after use.
- This risk of reuse of product for another patient could result in cross-contamination.
- Where asepsis is required or deemed appropriate in the best professional judgement of the DHCP, this product should not be used.

PRECAUTIONS

- Air/water syringe users should follow the syringe and dental unit manufacturers' instructions to maintain water quality, to minimize biofilm, and ensure tip is properly seated to prevent dislodging during procedure.
- When properly installed Sani-Tip air/water syringe tips do not contribute to cross-contamination due to retraction of oral flora. To prevent system "suckback," add an anti-retraction (anti-siphon) device in an appropriate location. Consult local sanitary codes.
- The risks of re-use include equipment malfunction and cross-contamination.

ADVERSE REACTIONS

None Know.

RESIDUAL RISK

- Tip is not properly installed into adapter.
- Cross-contamination can lead to patient infection or illness, use appropriate PPE when handling and dispose tip after each patient use.
- Aerosol Contamination.
- Cross contamination from waterline.

TROUBLESHOOTING

PROBLEM: Water leaking, wet air, and/or loose tips. Indicates either:

- a) Center hole of Sani-Tip® syringe tip is not fully seated over water spindle inside new adaptor, or
- b) Correct Sani-Tip® Conversion Kit is not properly installed, and/or is missing parts; thumb nut has not been fully tightened, or incorrect tips are being used.
- c) Internal water spindle is damaged.

SOLUTION:

- a) LOOSEN THUMB NUT when interchanging Sani-Tip® syringe tip. Insert new tip and PUSH into adaptor as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle. Tighten thumb nut.
- b) Make sure correct Sani-Tip® Conversion Kit is properly installed. Confirm presence of plastic cone and all O-rings. Fully tighten thumb nut after inserting new tip. Verify tips are clear plastic and manufactured by DENTSPLY Professional for the Sani-Tip® System.
- c) If internal water spindle inside Sani-Tip® adaptor is damaged, replace adaptor.

PROBLEM: Tip does not fit through thumb nut.

SOLUTION: Loosen thumb nut a full turn to expand opening of plastic cone.

PROBLEM: Little or no water.

SOLUTION: Check for clogged or damaged water spindle inside adaptor, or blocked water outlet in syringe head.

PROBLEM: Too much/little air or water pressure

SOLUTION: Minor adjustments to air and water pressure can be made at the dental unit. Follow manufacturer's specifications.

TEST PRIOR TO EACH USE

INSTALLATION INSTRUCTIONS

1. LOOSEN THUMB NUT on air/water syringe.

2. Insert new Sani-Tip® syringe tip and PUSH into adaptor as far as possible. While still pushing inward, twist tip 1/2 turn clockwise to create a watertight seal over internal tapered water spindle.

3. Hand-tighten thumb nut. To redirect tip, push in on tip while rotating clockwise.

4. Test prior to each use. Express water only, then air only. If air contains moisture, or tip dislodges, refer to the Troubleshooting section of your Conversion Kit Instructions for Use.
5. To replace tip, LOOSEN THUMB NUT. Remove and discard used tip. Repeat steps 2 and 3.

Sani-Shield® Disposable Covers (Air/Water Syringe Polyethylene Sleeve)

INDICATIONS FOR USE

Sani-Shield® Barrier Sleeves and Covers are intended to serve as a disposable barrier for dental instruments and equipment. This device is non-sterile and intended for single patient use only.

RESIDUAL RISK

None.

INSTRUCTIONS

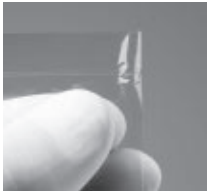
1. Insert new Sani-Tip® syringe tip. Test for proper air and water separation.

4. Slide tip along top edge of sleeve, toward tip opening.

7. A slight tug at open end of sleeve will create a seal around tip at thumb nut. For best results: push sleeve over tip, do not pull.

2. Locate tip opening at sealed end of sleeve.

5. Guide tip through opening. Slide opening along tip to thumb nut.

This product does not replace cleaning and disinfection of dental instruments. Please clean and disinfect dental instruments after each patient as described by the instrument manufacturer.

3. Slip Sani-Tip® syringe tip into open end of sleeve and aim toward tip opening.

6. Slide sleeve down over syringe handle.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the Competent Authority of the Member State.

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