

# GENTLE



# LOW-NOISE

**roeko**  
**VacuSoft**

The cannula for greater comfort during aspiration

[vacusoft.coltene.com](http://vacusoft.coltene.com)

 **COLTENE**

# Gentle and low-noise aspiration

Aspiration cannula distinguished by exceptional comfort, flexibility and silence!

- Soft tips provide gentle contact with the oral mucosa
- Low-noise aspiration without reflux
- Flexible adjustment to the treatment situation




Soft exchangeable tips ensure gentle contact with the oral mucosa. The extra soft **sensitive comfort** tip is particularly gentle.



# GENTLE



The special conical geometry of the tips optimises flow, thus reducing noise levels. At the same time, it prevents the gravity-induced reflux of aspirated media into the patient's mouth.

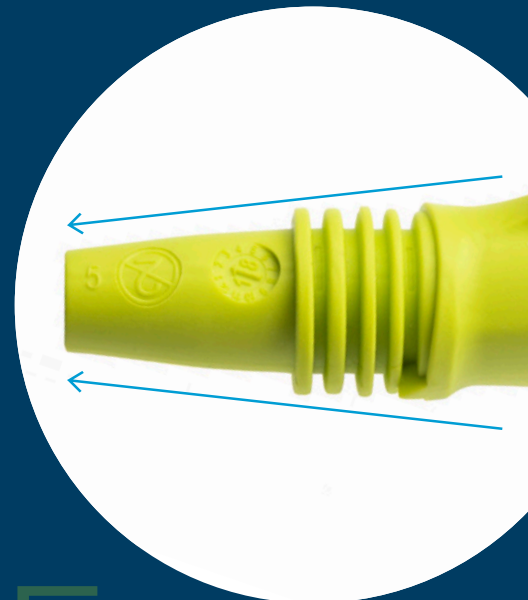

# LOW-NOISE

## Responding to the needs of special patient groups with VacuSoft



### Patients with dementia

Use low-irritation treatment methods



### Cancer patients

Protect sensitive oral mucosa



### Paediatric dentistry

Reduce foreign body sensation and pressure



## VacuSoft Base

Durable and reusable cannula tube with matt surface.

The tube is designed according to ergonomic aspects:

Specially flattened areas provide orientation when holding and increase grip, even when wearing gloves



The slightly angled geometry allows the cannula to lie comfortably in the hand and assists guiding.



Autoclavable

## VacuSoft Tip

Cannula tip for attachment to the VacuSoft base.

VacuSoft tips are made of soft TPE plastic and are therefore particularly gentle in contact with the patient's oral mucosa, tongue and cheeks. The wide and flat cannula tip is ideally suited for keeping the patient's cheeks and tongue away and ensuring a clear view of the treatment field. Different degrees of hardness of the colour-coded tips provide flexibility in aspiration and allow for individual treatment situations.



Not autoclavable



sensitive comfort

Very soft tip for gentle aspiration in particularly (pressure-) sensitive patients



comfort

Soft tip for gentle aspiration in uncomplicated treatment cases



solid comfort

Stable tip for aspiration situations where more resistance is needed for retraction

# Order Information



## VacuSoft Starter Kit

- 60023549** 2 × VacuSoft Base
- 5 × VacuSoft Tip sensitive comfort
- 5 × VacuSoft Tip comfort
- 5 × VacuSoft Tip solid comfort

## VacuSoft Base Refill

- 60023550** 10 × VacuSoft Base



## VacuSoft Tips Refill

50 pcs.

200 pcs.

- 60023551** **60023556** sensitive comfort
- 60023552** **60023557** comfort
- 60023554** **60023558** solid comfort



sensitive comfort



comfort



solid comfort

© COLTENE – [www.coltene.com](http://www.coltene.com)

### Coltène/Whaledent GmbH + Co. KG

Raiffeisenstraße 30  
89129 Langenau/Germany  
T +49 7345 805 0  
F +49 7345 805 201  
[info.de@coltene.com](mailto:info.de@coltene.com)

### Coltène/Whaledent AG

Feldwiesenstrasse 20  
9450 Altstätten/Switzerland  
T +41 71 757 5300  
F +41 71 757 5301  
[info.ch@coltene.com](mailto:info.ch@coltene.com)

### Coltène/Whaledent Inc.

235 Ascot Parkway  
Cuyahoga Falls, Ohio 44223/USA  
T +1 330 916 8800  
F +1 330 916 7077  
[info.us@coltene.com](mailto:info.us@coltene.com)