
# Dispensing gun

**KULZER**  
MITSUBI CHEMICALS GROUP

|                                         |    |                                             |     |
|-----------------------------------------|----|---------------------------------------------|-----|
| <b>(DE)</b> Gebrauchsanweisung .....    | 3  | <b>(PL)</b> Instrukcja obsługi .....        | 63  |
| <b>(GB)</b> Instructions for use .....  | 7  | <b>(HR)</b> Upute za uporabu.....           | 67  |
| <b>(FR)</b> Mode d'emploi .....         | 11 | <b>(RO)</b> Instrucțiuni de utilizare ..... | 71  |
| <b>(ES)</b> Instrucciones de uso.....   | 15 | <b>(AE)</b> تعليمات الاستخدام .....         | 75  |
| <b>(IT)</b> Istruzioni per l'uso .....  | 19 | <b>(CN)</b> 使用说明 .....                      | 79  |
| <b>(PT)</b> Instruções de uso .....     | 23 | <b>(KR)</b> 사용 설명서.....                     | 83  |
| <b>(NL)</b> Gebruiksaanwijzing .....    | 27 | <b>(SK)</b> Návod na použitie .....         | 87  |
| <b>(SE)</b> Bruksanvisning .....        | 31 | <b>(BG)</b> Инструкции за употреба .....    | 91  |
| <b>(DK)</b> Brugervejledning .....      | 35 | <b>(TH)</b> คำแนะนำในการใช้ .....           | 95  |
| <b>(NO)</b> Bruksanvisning .....        | 39 | <b>(UA)</b> Інструкція по застосуванню..... | 99  |
| <b>(FI)</b> Käyttöohjeet .....          | 43 | <b>(KZ)</b> Пайдалану нұсқаулығы.....       | 103 |
| <b>(CZ)</b> Návod k použití .....       | 47 | <b>(EE)</b> Kasutusjuhend.....              | 107 |
| <b>(HU)</b> Használati utasítás .....   | 51 | <b>(SI)</b> Navodila za uporabo .....       | 111 |
| <b>(LV)</b> Lietošanas instrukcija..... | 55 | <b>(IR)</b> دستورالعمل استفاده .....        | 115 |
| <b>(LT)</b> Naudojimo instrukcija ..... | 59 | <b>(BA)</b> Uputstvo za upotrebu.....       | 119 |

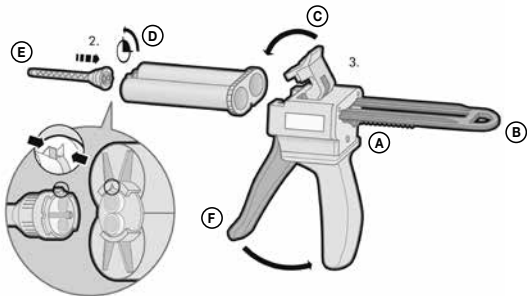# Dispensing gun

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# Dispensing gun

## Instructions for use

**Medical Device – for use by dental health care professionals only.**

### **1.1 Intended purpose:**

The Dispensing gun is a device for quick and hygienic direct or indirect application dental materials from Kulzer delivered in cartridges of ratios 1:1/2:1, 4:1 and 10:1.

This dental accessory is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant. Kulzer medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically, or they are accessories in the rehabilitation process.

### **1.2 Indications:**

The Dispensing gun is suitable for all common dental materials of Kulzer in cartridges.

### **1.3 Contraindications:**

In case of known or suspected allergy to components of the product, its use is contraindicated.

### **1.4 Potential side effects:**

Hypersensitivities to the product or its components cannot be excluded in individual cases. In case of suspicion please contact the manufacturer for ingredients information.

## **2.0 Product Use**

See pictures at the top of the Instruction for Use:

### **Insert cartridge**

1. Push up the lever [A] at the back of the dispensing gun and pull back the slide [B] completely.
2. Lift up the plastic lever [C], insert the cartridge and lock it by pushing down the plastic lever [C].

**Note:** Make sure that the notches on the cartridge line up with those on the Dispensing gun.

### **Attach mixing tip**

1. Remove the cartridge cap [D] or used mixing tip by turning it 90° counterclockwise, and discard it.
  2. Attach a new mixing tip [E].
- Note:** Make sure that the notches on the mixing tip line up with those on the cartridge.
3. Turn the mixing tip clockwise 90° to lock it into place.

### **Apply material**

Remove the cartridge cap [D] or used mixing tip by turning it 90° counter clockwise, and discard it. When using a cartridge for the first time, express a pea-sized amount of the material and discard it. Mix the material in the mixing tip by pulling the trigger [F] on the Dispensing gun and then apply directly.

**Note:** Leave the used mixing tip on the cartridge after use. It serves as a cap. Mixing tips and intraoral tips are single-use products. To

prevent cross-contamination between patients, mixing tips and intraoral tips must be discarded after each use.

### **Remove cartridge**

1. Push up the lever [A] at the back of the Dispensing gun and pull back the slide [B] completely.
2. Lift up the plastic lever [C] and remove the cartridge.

### **3.0 General information:**

Please pay attention to the following safety instructions and all processing instructions in other sections. This product must be used according to the instruction for use only. Any use not complying with these instructions is at the discretion and sole responsibility of the dental professional itself.

KULZER cartridges fit exactly into Dispensing gun. When using other brands make sure that they are securely locked in the Dispensing gun.

### **3.1 Warnings and safety instructions:**

Apply the cartridge content by evenly activating the lever. Excessively fast extrusion of the contents from the cartridge or the exertion of extremely high force during application should be avoided. Do not use visibly damaged devices. Before each use check functionality (ease of movement); see section: Maintenance, Inspection and Testing of processing instructions. Following this procedure there are no restrictions for reuse.

### **3.2 Storage conditions:**

When storing protect against dust and moisture.

### **3.3 Disposal information:**

**Recommendation:** Dispose of in accordance with official regulations. Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system.

### **4.0 Cleaning, Disinfection and Sterilization:**

**WARNING:** Please pay attention to the country-specific regulation regarding the hygiene standards and cleaning / disinfection of medical devices in dental practices. Wear suitable gloves. The Dispensing gun is delivered unsterile. It must be sterilized before first use and cleaned, disinfected and sterilized after each use.

### **Processing restrictions:**

Oxidizing disinfectant solutions are not suitable. Strong acids and strong bases can damage the plastic.

### **Instructions:**

**Point of use:** No specific requirements.

**Containment and Transport:** Transport the contaminated instrument (Dispensing gun) in a closed container.

**Preparation for decontamination:** Personal protective equipment (gloves, water repellent protective apron, face protection mask or protective glasses).

### **Cleaning: Manually:**

- **Equipment:** mild alkaline cleaner detergent (e.g. neodisher MediClean forte, Dr. Weigert #405033), Tap water / flowing water ( $20 \pm 2^{\circ}\text{C}$ ) (minimum drinking water quality), cleaning brush (e.g. Interlock #09098 brush head length: 35 mm x 10 mm, 25 mm x 5 mm, total length: 175 mm), tank / basin for detergent.

- 1) The cleaner is prepared in accordance to the manufacturer's instructions for the detergent (e.g. neodisher MediClean forte 1%) (temperature maximum 40°C).
- 2) Carry out all further steps below the liquid level to avoid splashing of contaminated liquid.
- 3) Immerse the instrument (Dispensing gun) completely in the cleaning solution.
- 4) Brush the hard to reach areas of the submerged instrument (Dispensing gun) with a soft brush for 1 minute. Pay attention to the critical, hard to reach areas where it is not possible to assess the cleaning efficacy.
- 5) Move all moveable components 5 times in the cleaning solution.
- 6) Total exposure time in the cleaning solution minimum 5 minutes (5 minutes was validated).
- 7) Remove the instrument (Dispensing gun) from the cleaning solution and rinse under running tap water (minimum drinking water quality) for 1 minute to remove the cleaning solution completely. Move the moveable components 5 times.

Inspect for cleanliness, if debris is visible, repeat the above steps.

#### **Disinfection: Manually:**

- **Equipment:** Virucidal (VAH or at least DWV-listed) instrument disinfectant based on peroxide compounds (e.g. Dr. Weigert neodisher Septo Fin #420735) demineralized water (demineralized water, according to KRINK / BfArM recommendation free of facultative pathogenic micro-organism) ( $20 \pm 2^\circ\text{C}$ ), disinfectant tank, lint free gauze, medical compressed air (according to the European Pharmacopoeia).

- 1) The disinfectant is prepared in accordance to the manufacturer's instructions for the disinfectant (e.g. neodisher Septo Fin 3% was validated).
- 2) Immerse the instrument (Dispensing gun) completely in the disinfectant solution. Make sure that all surfaces are completely moistened with disinfectant solution. Move the moveable components 5 times. Exposure time of the disinfectant solution in accordance to the manufacturer's instructions (15 minutes were validated).
- 3) Remove the instrument (Dispensing gun) from the disinfectant solution.
- 4) Rinse the instrument (Dispensing gun) completely under demineralized water (according to KRINK / BfArM recommendation free of facultative pathogenic micro-organism) for minimum 1 minute to remove the disinfectant solution completely. Move the moveable components 5 times.
- 5) Wipe with a lint free gauze and / or dry with medical compressed air (according to the European Pharmacopoeia).

#### **Cleaning and disinfection: Automated WD**

- **Equipment:** Washer disinfectant in accordance with EN ISO 15883-1+-2 with thermal program (temperature  $90 - 95^\circ\text{C}$ ), mild alkaline cleaner detergent (e.g. neodisher MediClean forte, Dr. Weigert #405035 was validated), medical compressed air (according to the European Pharmacopoeia).

- 1) Open the cover of the retainer and place the instrument (Dispensing gun) in a suitable tray or place on the load carrier so that all inner and outer surfaces will be cleaned and disinfected.
- 2) Close the WD and start the program. The program parameters are shown in the table below.
- 3) At the end of the program, remove the instrument.

| Program step   | Water           | Dosage                                   | Time                                           | Temperature                              |
|----------------|-----------------|------------------------------------------|------------------------------------------------|------------------------------------------|
| Pre-Rinse      | Cold            |                                          | 5 min                                          |                                          |
| Dosage cleaner |                 | According to manufacturer's instructions |                                                | According to manufacturer's instructions |
| Cleaning       | Deionised water |                                          | 10 min                                         | 55°C                                     |
| Rinse          | Deionised water |                                          | 2 min                                          |                                          |
| Disinfection   | Deionised water |                                          | A0 value >3000 <sup>1</sup> (e.g. 5 min, 90°C) |                                          |
| Drying         |                 |                                          | 15 min                                         | Up to 120°C                              |

<sup>1</sup> Public authorities may adopt other implemented provisions (parameter for disinfection performance) within their field of responsibility.

- 4) Check for dryness. If it is necessary, wipe with a lint free gauze and / or dry with medical compressed air (according to the European Pharmacopoeia).
- 5) After removal from the WD, inspect the instrument (Dispensing gun) for cleanliness. If debris is still visible, clean the instrument manually. Afterwards the automated cleaning process must be carried out again.

#### Maintenance, Inspection and Testing:

- 1) All instruments must be visually checked for cleanliness, dryness and functionality, if necessary, using an illuminated magnifier (3 – 6 Diopters)

- **Note:** Do not use instruments (Dispensing gun) with cracks, fractures, holes or similar.

#### Sterilization:

- **Equipment:** Steam sterilizer according to DIN EN 285 or DIN EN 13060 with type B process

- Pre-vacuum process, 132°C and sterilization time minimum 3 min (longer holding times are possible) (132°C, 3 minutes were validated)
  - 1) Place the device in the sterilizer chamber.
  - 2) Start the sterilization program.
  - 3) At the end of the sterilization program remove the device and let it cool down.

#### Complaint notice:

Please quote batch number and article number or UDI in all correspondence about the product. Article number **REF** and batch number **LOT** or UDI: See note on packaging.

#### Undesired Side Effects and Events:

According to EU Medical Device Regulation users/patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

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Datum: 2023-10

#### Explanation of symbols on labelling

|                                                                                             |                |                                                                                              |                              |
|---------------------------------------------------------------------------------------------|----------------|----------------------------------------------------------------------------------------------|------------------------------|
|  <b>MD</b>  | Medical device |             | Date of manufacture          |
|  <b>REF</b> | Reorder no.    |  <b>UDI</b> | Unique Device Identification |
|             | Manufacturer   |             | Consult instructions for use |
|  <b>LOT</b> | Batch code     |             | Keep dry                     |

 **Manufacturer:**  
Kulzer GmbH  
Leipziger Straße 2  
63450 Hanau (Germany)

 **+49 800 4372522**

Made in China

Distributed in USA / Canada exclusively by:  
Kulzer, LLC  
4315 South Lafayette Blvd.  
South Bend, IN 46614-2517  
1-800-431-1785

Caution: Federal law restricts this device to sale by or on the order of a dental professional.

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