Instructions for use

- EN** see www.fkg.ch for additional information
- FR** informations supplémentaires sur www.fkg.ch
- DE** weitere Informationen auf www.fkg.ch

XP-endo® Shaper

Endodontic instruments
Instruments endodontiques
Endodontische Instrumente

FKG REF. 99.X00.10.05A.XX - n°108 - 2016/06

XP-endo® Shaper

Description - Beschreibung

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Recommended speed: 800 rpm

Torque: 1 Ncm

- Prepare the glide path (min. size ISO 15/.02) before using the XP-endo Shaper.
- In multirooted teeth, begin with the largest canal.
- Each instrument must be used applying long, gentle longitudinal movements.
- Irrigate copiously throughout the protocol.

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General information

Indications for use:

These endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

XP-endo Shaper is used for endodontic treatments.

Contraindications:

This product contains nickel, and must not be used on individuals with a known allergic sensitivity to this metal.

Adverse reactions: None known

Warnings and precautions:

- Single use instruments must not be retreated for subsequent use. That could compromise their integrity.
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.

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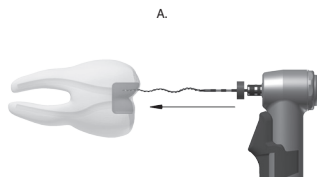
Protocol for use


Speed : 800 rpm (800-1000 rpm)
Torque : 1 Ncm

1. Use XP-endo Shaper (XP-S) only after preparing a glide path up to minimum ISO 15/.02.
2. Remove an XP-S from the sterile blister pack, and place it in a contra-angle handpiece. There is no need to straighten the instrument before use.
3. Define the working length by positioning the rubber stop 0.5 mm short of the working length (WL).
4. Insert the tip of the instrument inside the canal (Fig. A), and then switch the instrument to rotate mode. The canal must always contain some irrigant.
5. Use the XP-S until the WL, applying light up-and-down movements (3 to 5 up-and-down strokes), in order to obtain the final size (Fig. B).
6. Remove the XP-S from the canal while it is rotating. Deactivate rotate mode.
7. Clean the instrument and irrigate the canal.
8. Once you reach the WL, apply 5 more up-and-down movements over the entire length of the canal (Fig. C).
9. Irrigate the canal in order to eliminate suspended debris.
10. Check the size using a Gutta Percha 30/.04.
11. If the final size has not been reached (30/.04), start over at point 7.
12. If a preparation larger than 30/.04 is required, use a Race instrument of the right size to obtain the desired final preparation.

Once the tooth has been fully shaped and cleaned:

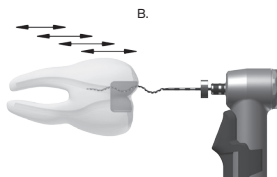13. Dry the canals and fill them using an inert cone (e.g. gutta percha) and a sealer (e.g. TotalFill®).

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