





Temperature Range



- Wide temperature range of 90° to 250°
- Five memory programs to customize temperature and cut-off time as per obturation material used or any other intended use of the device

Internally Heated Tip



- Only the working end (4 6 mm) of the tip is heated avoids need for insulating sleeves
- The non-working end is insulated and stays cool avoiding soft tissue burn



Class-leading Controlled Heating

- Fast heating Quick heating prevents PDL damage
- Swift cooling Swift cooling ensure GP does not follow along the heating tip

Experience Ergonomics



- 360° start / stop ring button allows holding the device in any angle comfortable for you
- Left right handed operation
- Comfortable grip, light in weight

Powerful Battery



- Highly efficient 2600mAh Lithium battery
- Direct Plug & Play In case of low battery, use the device directly with adaptor

Specifications

Heating Time	0.2 seconds for 200 °C 0.5 seconds for 250 °C
Cooling Time	2 seconds for cooling
Charging Time	2 hours to complete charge
Operating Time	4 hours of continuous operation at 250 °C







Medium (Yellow) 50/0.05 - 1 Qty



Small (Black) 40/0.025 - 1 Qty

Fast-Pack

21



Fast-Fill

Obturation Fill Handpiece

PEighteeth

Fast-Fill



Fast-Fill Tips

- Autoclavable silver tips for multiple usages
- Silver tips maintain the thermoplastic temperature of GP and offers smooth flow inside canal
- 360 ° rotating tip for easy filling at all angles
- Extra long needle offers convenience and visibility
- Easy to maintain and replace

Motorized GP Flow



- Provides consistent flow of GP at desirable rate
- Precludes voids and makes obturation homogeneous
- Eliminates hand fatigue that resulted from earlier gun type devices

Comprehensive Display



- Three customizable memory programs T1 to T3
- Temperature range 100 °C 200 °C
- GP flow rate Low, Medium, High
- Reserve GP level indicator

Intuitive Design

- Sleek and ergonomic hand-piece
- Simple push button to dispense GP
- LED indicator reflects when the device is ready for use

Powerful Battery



- Highly efficient 2600mAh Lithium battery
- Direct Plug & Play In case of low battery, use the device directly with adaptor

Specifications

Heating Time	20 seconds to 200 °C
Charging Time	4 hours to complete charge
Hexagonal Wrench	Use the wrench to bend the tip, rotate the tip and clean the device




23 Gauge Large



25 Gauge Small



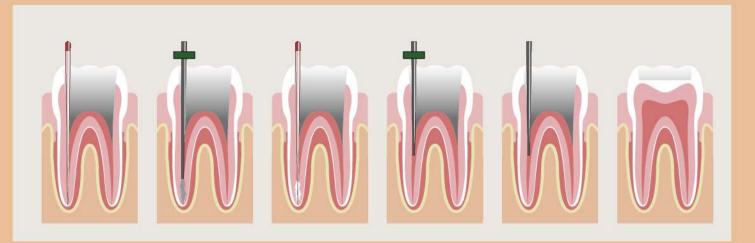

Protocol for Use

Master Cone Verification & Resizing

Select a master cone which corresponds to the master apical preparation. Check for tug-back and confirm the same with radiograph. Resize the GP by cutting it to 0.5 to 1mm short of the working length. This will prevent peri-apical extrusion of the GP

Selection of Fast-Pack & Plugger Tip

Select the Fast-Pack tip which reaches 4-5 mm short of the working length and give a snug fit at the tip. Mark this length using the silicon stopper. Similarly, select the hand plugger which reaches 4-5mm short of the working length

Gutta Percha & Sealer

Coat the master cone apically with the sealer and insert it into the canal. Brush the canal wall using strokes of GP and sealer

Down Pack with Fast-Pack

Remove the GP from the orifice/coronal part with the help of heated Fast-Pack tip. Plug the GP cone with pre-fit hand plugger at the orifice. Now take the heated Fast-Pack tip 4-5mm short of the working length. The bite of GP will come out of the canal along with the tip. Heated Fast-Pack tip will transfer heat to the GP and convert it into thermoplastic state. Condense the GP with the help of hand plugger to ensure the complete sealing of accessory canals and apex

Fill with Fast-Fill

Filling the canal using the Fast-Fill device in increments of 4mm till the canal is filled completely while condensing the warm GP with the appropriate plugger after each increment

Fast-Pack & Fast-Fill Fast-Pack & Fast-Fill 36