Dyract[®] eXtra

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Dyract[®] eXtra

The Caries Preventing Restorative

CAUTION: This is a medical device. For dental use by dental professionals only.

USA: Rx only.

Content	Page
1 Product description	2
2 Safety notes	3
3 Step-by-step instructions	5
4 Hygiene	7
5 Lot number, expiration date and correspondence	8

1 Product description

Dyract[®] eXtra restorative is a light-curing restorative material for all cavity classes in anterior and posterior teeth.

Dyract[®] eXtra restorative combines the fluoride release of glass ionomer materials with the strength and esthetics of a light-curing composite.

eXtra Care

Dyract[®] eXtra restorative restorations continuously release fluoride ions, act on the tooth restoration interface as an acid buffer and effectively support the prevention of approximal caries.

The choice of **Dyract[®] eXtra** restorative for patients prone to risk of caries is therefore advisable as an additional care against the onset of new caries.

1.1 Intended purpose

Direct restoration of defects in teeth.

1.2 Indications

Direct restorations of all cavity classes in anterior and posterior teeth. Cavity width must be less than 2/3 of the intercuspal distance.

1.3 Contraindications

- Use in patients with a history of severe allergic reaction to methacrylate resins or any of the other components.
- Use in Class I and II cavities whose width exceeds 2/3 of the intercuspal distance.
- For core build-up.

1.4 Delivery forms

- Predosed Compules[®] Tips

1.5 Composition

- Strontium-alumino-sodium-fluoro-phosphor-silicate glass
- Urethane dimethacrylate (UDMA)
- Dimethacrylate resin
- Strontium fluoride
- Carboxylic acid modified dimethacrylate (TCB resin)
- Trimethacrylate
- Triethyleneglycol dimethacrylate (TEGDMA)
- Highly dispersed silicon dioxide
- Silanated barium-aluminum-borosilicate glass
- Ethyl-4-dimethylaminobenzoate (photoaccelerator)
- Butylated hydroxy toluene (BHT)
- UV stabilizer
- Camphorquinone (photoinitiator)
- Titanium dioxide
- Iron oxide pigments

1.6 Compatible adhesives

Dyract® eXtra is used following application of a suitable dentin/enamel adhesive and is chemically compatible with conventional (meth)acrylate-based dentin/enamel adhesives including Dentsply Sirona adhesives designed for use with visible light-cured composite restoratives (see complete Instructions for Use of selected adhesive).

2 Safety notes

Be aware of the following general safety notes and the special safety notes in other chapters of these Instructions for Use.

Safety alert symbol.

- This is the safety alert symbol. It is used to alert you to potential personal injury hazards.
- Obey all safety messages that follow this symbol to avoid possible injury.

Information symbol.

This information symbol is used to highlight important information for the correct use.

2.1 Warnings

The material contains methacrylates and polymerizable monomers that may be irritating to skin, eyes and oral mucosa and may cause sensitization by skin contact and allergic contact dermatitis in susceptible persons.

This product contains Ethyl-4-dimethylaminobenzoate, which is classified as a CMR substance according to Regulation EC 1272/2008 on Classification, Labelling and Packaging of substances and mixtures. Based on a toxicological risk assessment, the presence of Ethyl-4-dimethylaminobenzoate does not pose a toxicological risk, if the product is used as intended.

- **Avoid eye contact** to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek medical attention.
- **Avoid skin contact** to prevent irritation and possible allergic response. In case of contact, reddish rashes may be seen on the skin. If contact with skin occurs, remove material with cotton and alcohol and wash thoroughly with soap and water. In case of skin sensitization or rash, discontinue use and seek medical attention.

- **Avoid contact with oral soft tissues/mucosa** to prevent inflammation. If accidental contact occurs, remove material from the tissues. Flush mucosa with plenty of water and expectorate/evacuate the water. If inflammation of mucosa persists, seek medical attention.

2.2 Precautions

This product is intended to be used only as specifically outlined in these Instructions for Use. Any use of this product inconsistent with these Instructions for Use is at the discretion and sole responsibility of the dental practitioner.

- Safety and effectiveness in pregnant or breastfeeding women have not been established.
- Use protective measures for the dental team and patients such as glasses and rubber dam in accordance with local best practice.
- Compules® Tips may be used for direct intraoral application of restorative material into a cavity or for indirect application by first placing the restorative material on a pad. After intraoral use, discard Compules® Tips and do not reuse in other patients in order to prevent cross-contamination.
- The Compules® Tips cannot be reprocessed. To prevent Compules® Tips from exposure to sputter or spray of body fluids or contaminated hands when used for indirect application it is mandatory that the devices are handled offside the dental unit with clean/disinfected gloves. Discard devices if contaminated.
- Use of Compules® Tips with the Compules® Tips Gun is recommended.
- For further information please refer to the Compules® Tips Gun Instructions for Use.
- The material should extrude easily. **DO NOT USE EXCESSIVE FORCE.** Excessive pressure may cause the Compules® Tip to rupture or to eject from the Compules® Tips Gun.
- Contact with saliva, blood or sulcus fluid during application may cause failure of the restoration. Use adequate isolation such as rubber dam.
- Interactions:
 - Do not use eugenol- and hydrogen peroxide-containing materials in conjunction with this product since they may interfere with hardening of the product.
 - Dyract® eXtra is a light-cured material. Therefore, it should not be exposed to high intensity lights sources (e.g., operator light or LED lights of loupes). Proceed immediately once material has been placed.

2.3 Adverse reactions

- Eye contact: Irritation and possible corneal damage.
- Skin contact: Irritation or possible allergic response. Reddish rashes may be seen on the skin.
- Contact with mucous membranes: Inflammation (see [2.1 Warnings](#)).

2.4 Storage conditions

Inadequate storage conditions may shorten the shelf life and may lead to malfunction of the product.

- Store in sealed packs at temperatures between 10 °C and 24 °C (50 °F and 75 °F).
- Keep out of direct sunlight and protect from moisture.
- Do not freeze.
- Do not use after expiration date.

Humidity can adversely affect the properties of the material. For unsealed Compules® Tips, the following is recommended:

- Store in a dry environment (< 80% relative humidity).
- Use within 4 weeks.

3 Step-by-step instructions

3.1 Shade selection

Shade selection is made whilst the teeth are hydrated and prior to the restorative procedure. With only 6 shades, teeth within the complete shade range of the VITA®¹ shade range can be restored satisfactorily. Dyract® eXtra restorative is also available in two opaque shades (O-A2, O-B3). For the convenience of former Dyract® restorative and Dyract® AP restorative users, the shades B3 and C3 are available.

1. Remove any extraneous plaque or surface stain.
2. Use the VITAPAN®¹ classical shade guide. Use the central part of the respective VITA® tooth for shade selection.
3. Use the table below to select the most suitable Dyract® eXtra shade.

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3.2 Cavity preparation

Cavity width.

Cavity width (a) must be less than 2/3 of the intercuspal distance (b). The graphic on the right also appears on all outer packages.

1. Clean the tooth surface with a fluoride-free prophylaxis paste (e.g. Nupro® prophylaxis paste).
2. Prepare cavity (unless not necessary, e.g. cervical lesion).
3. Use adequate isolation such as rubber dam.
4. Rinse surface with water spray and carefully remove rinsing water. Do not desiccate the tooth structure.

3.2.1 Matrix placement and wedging (Class II restorations)

Place a matrix (e.g. AutoMatrix® matrix system or Palodent® matrix system) and wedge. Burnishing of the matrix band will improve contact and contour. Pre-wedging or Palodent® V3 ring placement is recommended.

3.3 Pulp protection, tooth conditioning/dentin pre-treatment, adhesive application

Refer to adhesive manufacturer's Instructions for Use for pulp protection, tooth conditioning and/or adhesive application. Once the surfaces have been properly treated, they must be kept uncontaminated. Proceed immediately to placement of the material.

¹ Not registered trademarks of Dentsply Sirona, Inc.

3.4 Application

Apply material in increments immediately after the application of the adhesive. The material is designed to be cured in increments up to a 2 mm depth/thickness.

Using Compules® Tips

1. Insert Compules® Tip into the notched opening of the Compules® Tips Gun. Be certain that the collar on the Compules® Tip is inserted first.
2. Remove the colored cap from the Compules® Tip. The Compules® Tip may be rotated 360° to gain the proper angle of entrance into the cavity or to the mixing pad.
DO NOT USE EXCESSIVE FORCE.
Indirect use: Dispense the necessary amount of material onto a mixing pad and protect against light. Apply material into the cavity with a plastic instrument.
- or -
Direct intraoral use: Dispense the material directly into the cavity preparation using a slow, steady pressure.
3. To remove the used Compules® Tip, be sure that the Compules® Tips Gun plunger is pulled back completely by allowing the handle to open to its widest position. Apply a downward motion to the front end of the Compules® Tip and remove.

Excessive force – To reduce the risk of injury.

1. Apply slow and steady pressure on the Compules® Tips Gun.
2. Do not use excessive force. Compules® Tip rupture or ejection from Compules® Tips Gun may result.

3.5 Light-curing

1. Light-cure each area of the restoration surface with a suitable curing light². Refer to curing light manufacturer's Instructions for Use for compatibility and curing recommendations.
2. Cure each layer according to the table below.

Insufficient curing – To reduce the risk of inadequate polymerization.

1. Check compatibility of curing light.
2. Check curing cycle.
3. Check minimum irradiance.
4. Cure each area of each increment for the recommended curing time.
5. Check distance to surface to be cured.

Curing time for 2 mm increments

Output	≥ 500 mW/cm²
Shades A2, A3, A3.5, A4, B1, B3, C2, C3	10 seconds
Shades O-A2, O-B3	20 seconds

² Curing light designed to cure materials containing camphorquinone (CQ) initiator. Peak of spectrum in the range of 440-480 nm and irradiance of 500-2000 mW/cm².

3.6 Finishing and polishing

1. Contour the restoration using finishing burs or diamonds.
2. Use Enhance® finishing instruments and interproximal finishing strips. In patients with adequate oral hygiene, the high final luster of the restoration comes with use.

For contouring, finishing, and/or polishing, follow the manufacturer's Instructions for Use.

4 Hygiene

Cross-contamination – To reduce the risk of infection.

1. Do not reuse single use products. Dispose of Compules® Tips in accordance with local regulations.
2. Reprocess reusable products according to the instructions.

4.1 Compules® Tips Gun

For reprocessing instructions please refer to the Instructions for Use of the Compules® Tips Gun, which is available on our webpage at <https://www.dentsplysirona.com/ifu>. If requested, we will send you a free printed copy of the Instructions for Use in the language you require within 7 days. You can use the order form provided on <https://www.dentsplysirona.com/ifu> for this purpose.

4.2 Compules® Tips – after intraoral use

Cross-contamination – To reduce the risk of infection.

1. Do not reuse Compules® Tips after intraoral use and/or if contaminated.
2. Dispose the intraorally used and/or contaminated Compules® Tips in accordance with local regulations.

4.3 Disposal

Dispose of in accordance with local regulations.

5 Lot number (), expiration date () and correspondence

1. Do not use after expiration date.
ISO standard is used: "YYYY-MM" or "YYYY-MM-DD"
2. The following numbers should be quoted in all correspondence:
 - Reference number (REF)
 - Lot number (LOT)
 - Expiration date (EXP)
3. Any serious incident in relation to the product should be reported to the manufacturer and the competent authority according to local regulations.
4. Device Identification (Basic UDI-DI): ++D010FIM02QZ

A summary of the safety and clinical performance (SSCP) for this product can be found (upon activation) at <https://ec.europa.eu/tools/eudamed> by searching using the Basic UDI-DI number listed above and at <https://www.dentsplysirona.com/ifu> using the reference number (REF).

For an explanation of the symbols used on the label, please refer to the Symbols Glossary at <https://www.dentsplysirona.com/ifu>.

© Dentsply Sirona 2025-05-05

[These Instructions for Use are based on Master Version 12]

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Dentsply DeTrey GmbH