

CONTACTEZ



DIRECTA

ContacEZ Restorative Strip System





The three-step **ContactEZ Restorative Strip System** offers you an easy and effective way to finish a crown seating, breaking the process down into three simple steps: **Adjustment, Cleaning, and Polishing.**

ContacEZ Restorative Strip System

1 Adjust proximal contact

The Black Diamond Dental Strip – The Proximal Contact Adjuster – Fine Diamond – Single-sided

- Adjust proximal contacts between the indirect restoration and the adjacent teeth.
- A strip to be used prior to definitive cementation of crowns, inlays/onlays, and veneers.
- Adjust and contour proximal contacts on composite fillings.

The Orange Serrated Diamond Strip

- A strip to be used to adjust and contour proximal contacts for composite fillings.
- The serrated edge readily separates and penetrates tight contacts to prepare for easy access to proximal contact adjustments.





2 Clean out trapped cement

The White Serrated Strip – The Gentle Saw - Interproximal Cement Cleaner – No Diamond – Double-serrated.

- An ultra-thin serrated strip that is designed to safely cut and clean out excess cement.
- A strip to be used for interproximal cleaning of multiple veneers or composite fillings.

The Blue Serrated Strip – The Heavy-Duty Dental Saw – No Diamond – Double-serrated

- This heavy-duty dental saw strip is designed for more rigorous cutting and cleaning in the interproximal space.
- A strip that safely cuts into massive excess resin cement trapped in the interproximal space.

3 Polish to a natural finish

The Grey Final Polishing Strip – The Proximal Surface Polisher – Extra-fine Diamond – Single-sided

- A strip to be used to polish proximal contact surfaces of restorations and to restore a final finish in one step.

ContactEZ IPR Strip System

The **ContactEZ IPR Strip System** is an innovative set of dental instruments specifically crafted to make interproximal reduction safe, fast and easy. These flexible dental strips are ergonomically designed, patient-friendly, and conveniently labelled with specifications on every strip.

ContactEZ's interproximal reduction strips make IPR safe and accurate without creating sharp corners or cutting lips and gums. Designed for use with clear aligners and orthodontic treatments, these strips prevent excessive enamel removal and patient comfort and safety.

ContactEZ Rainbow Set

The ContactEZ Rainbow Set includes both the IPR Basic Strips as well as IPR PLUS. This 24-strip set gives dentists a full range of options for every possible case they encounter. Designed for use with clear aligners and other orthodontic treatments, these diamond strips are designed to be flexible, preserving the natural contours of the teeth while also preventing any harm or discomfort to patients.

- **Solutions for most cases** – With the ContactEZ IPR Strip System, dentists have precise, easy-to-use devices to fit most patients' IPR need.
- **Well contoured contacts** – The good-quality, stainless steel strips are flexible to preserve and conform to the natural contours of teeth during IPR. They are also taut enough to effortlessly access even tight contacts.
- **Accurate** – Paired with the Incremental Thickness Gauges, ContactEZ provides systematic, precise enamel removal without cutting lips, gums or cheeks.





IPR Strip System



Yellow Starter
0.07 mm
Single-Sided. Extra-Fine Diamond. Serrated




Red Opener
0.12 mm
Double-Sided. Medium Diamond



Dark Blue Widener
0.15 mm
Double-Sided. Coarse Diamond



Green Widener
0.20 mm
Double-Sided. Coarse Diamond

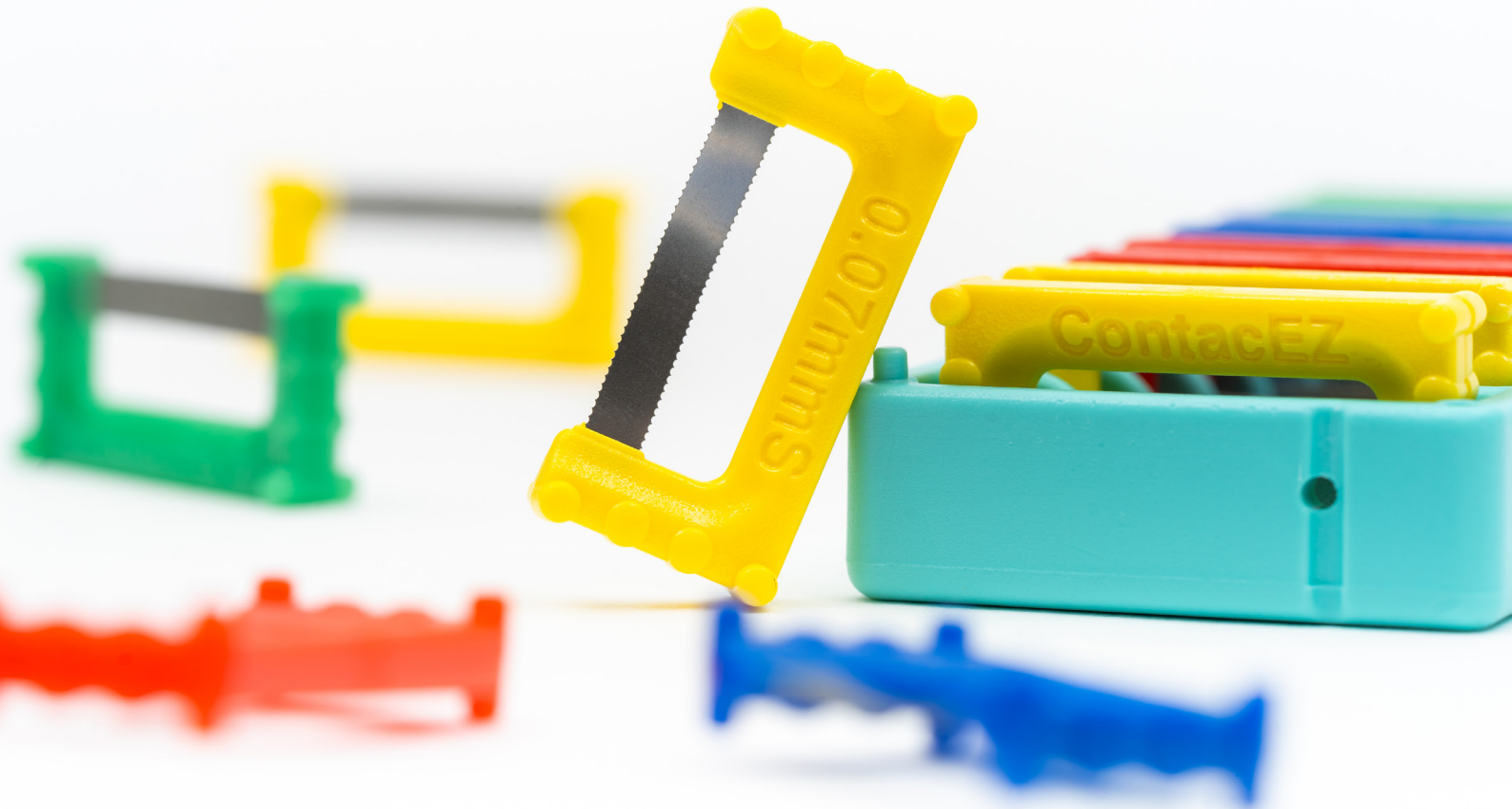

IPR PLUS



Clear Opener
0.10 mm
Single-Sided. Medium-Fine Diamond.
Serrated



Cyan Opener
0.15 mm
Single-Sided. Medium-Coarse Diamond



Purple Widener
0.25 mm
Double-Sided. Coarse Diamond



Brown Widener
0.30 mm
Double-Sided. Coarse Diamond



Mint Widener
0.35 mm
Double-Sided. Coarse Diamond



Aqua Widener
0.40 mm
Double-Sided. Coarse Diamond



Turquoise Widener
0.45 mm
Double-Sided. Coarse Diamond



Teal Widener
0.50 mm
Double-Sided. Coarse Diamond

Incremental thickness gauge



The **ContactEZ Incremental Thickness Gauge** efficiently and accurately confirms the amount of interproximal enamel removal or contact reduction for orthodontic treatment.

ContacEZ Subgingival Strip



The **Subgingival Strip System** is designed to remove carefully and conveniently hard-to-reach subgingival rough margins and overhangs and polish the surface to restore a natural finish. This system is specially designed to access the subgingival area to make overhang removal simple while preventing damage to patients' gingival tissue and lips. In addition, these ultra-narrow strips are perfect for cleaning out cement debris in the subgingival area of the interproximal space after crown cementation.



Cut

The Blue Narrow Strip – Subgingival Cutter Double-serrated – No Diamond

- The Blue Narrow Strip can be used when large, protruding overhangs are detected.
- The thin, serrated strip easily accesses the subgingival area and cuts away excess composite material or crown cement.



Trim

The Gold Narrow Strip – Subgingival Trimmer - Single-sided – Medium Coarse Diamond – Central Gateway

- A strip to be used when overhangs are smaller or do not stick out from the tooth.
- The thicker diamond strip allows you to quickly trim away overhangs, while the blank central gateway enables smooth access to the interproximal space.



Polish

The Copper Narrow Strip – Subgingival Polisher Single-sided – Fine Diamond

- A strip to be used to clean and polish the restoration after removing overhang.
- Restores a natural finish to the surface of the tooth in one easy step.

Hygienist Strip



Remove stains, discolorations, and clean proximal contact areas without damaging enamel surfaces.

The **ContacEZ Hygienist Strip** is a dental strip designed to remove extrinsic discoloration and plaque from interproximal surfaces without causing damage to tooth enamel. The non-abrasive, flexible, stainless-steel strips clean proximal contact areas of tough stains from coffee, tea, tobacco, and red wine.

Strip Tray

These convenient, autoclavable trays are perfect for keeping strips organized and accessible. Each tray holds 8 strips.



LAB Strip System



The **ContacEZ LAB Strip System** is designed to reduce the frustrations inherent in coordinating proximal contact restoration of crowns between dental clinics and dental labs, creating a consistent procedure that reduces remakes and rebakes. For laboratory use only.



LAB Stone Strip - Clear

0.10 mm

Single-Sided. Medium-Fine Diamond.

Scrape adjacent model teeth 0.01 mm with each stroke.



LAB Stone Strip - Cyan

0.12 mm

Single-Sided. Medium Diamond.

A thicker alternative to LAB Stone Strip (Clear).



LAB Bisque Strip - Black

0.06 mm

Single-Sided. Fine Diamond.

Adjust proximal contact of bisque baked crown on model.



LAB Glaze Strip - Gray

0.04 mm

Single-Sided. Ultra-Fine Diamond.

Adjust proximal contact of crown on model after glazing.

ContacEZ IPR System



Yellow - Starter
0.07 mm
Single-Sided. Extra-Fine Diamond. Serrated



Dark Blue - Widener
0.15 mm
Double-Sided. Coarse Diamond



Red - Opener
0.12 mm
Double-Sided. Medium Diamond



Green - Widener
0.20 mm
Double-Sided. Coarse Diamond

ContacEZ IPR PLUS



Clear - Opener
0.10 mm
Single-Sided. Med-Fine Diamond. Serrated



Cyan - Opener
0.15 mm
Single-Sided. Medium-Coarse Diamond



Purple - Widener
0.25 mm
Double-Sided. Coarse Diamond



Brown - Widener
0.30 mm
Double-Sided. Coarse Diamond



Mint - Widener
0.35 mm
Double-Sided. Coarse Diamond



Aqua - Widener
0.40 mm
Double-Sided. Coarse Diamond



Turquoise - Widener
0.45 mm
Double-Sided. Coarse Diamond



Teal - Widener
0.50 mm
Double-Sided. Coarse Diamond

Hygienist Strip



Pink
0.05 mm
Perforated. Stainless Steel

ContacEZ Restorative Strip System



Black - Diamond Strip
Single-Sided. Fine Diamond



Orange - Serrated Diamond Strip
Single-Sided. Extra-Fine Diamond
Double Serrated



White - Gentle Saw
Double-Sided. Coarse Diamond



Blue - Heavy Duty Saw
Double Serrated. No Diamond



Grey - Final Polishing Strip
Single-Sided. Ultra-Fine Diamond

ContacEZ Subgingival Strip



Blue - Narrow Strip
0.07 mm
No Diamond. Double Serrated



Gold - Narrow Strip
0.10 mm
Single-Sided. Medium Diamond



Copper - Narrow Strip
0.06 mm
Single-Sided. Extra-Fine Diamond

LAB Strip System



Clear - LAB Stone Strip
0.10 mm
Single-Sided. Medium-Fine Diamond



Cyan - LAB Stone Strip
0.12 mm
Single-Sided. Medium Diamond



Black - LAB Bisque Strip
0.06 mm
Single-Sided. Fine Diamond



Gray - LAB Glaze Strip
0.04 mm
Single-Sided. Ultra-Fine Diamond

Art. Nr

Prod No	Product Description	Colour	Quantity
Restorative Strip System			
31008	Restorative Strip System Assorted	2 Black, 1 Orange, 1 Gray, 2 White, 2 Blue	8 pcs
31020	Restorative Strip System Assorted	4 Black, 4 Orange, 4 Gray, 4 White, 4 Blue	20 pcs
31040	Restorative Strip System Assorted	8 Black, 8 Orange, 8 Gray, 8 White, 8 Blue	40 pcs
31016	Restorative Strip System	4 Black, 2 Orange, 2 Gray, 4 White, 4 Blue	16 pcs
31108	Restorative Strip System - Black	Black	8 pcs
31116	Restorative Strip System - Black	Black	16 pcs
31132	Restorative Strip System - Black	Black	32 pcs
31208	Restorative Strip System - Orange	Orange	8 pcs
31216	Restorative Strip System - Orange	Orange	16 pcs
31232	Restorative Strip System - Orange	Orange	32 pcs
31408	Restorative Strip System - White	White	8 pcs
31416	Restorative Strip System - White	White	16 pcs
31432	Restorative Strip System - White	White	32 pcs
31508	Restorative Strip System - Blue	Blue	8 pcs
31516	Restorative Strip System - Blue	Blue	16 pcs
31532	Restorative Strip System - Blue	Blue	32 pcs
31308	Restorative Strip System - Gray	Gray	8 pcs
31316	Restorative Strip System - Gray	Gray	16 pcs
31332	Restorative Strip System - Gray	Gray	32 pcs
31023	Signature Set	1 Clear LAB, 1 Cyan LAB, 3 Black, 3 Orange, 2 Gray, 3 White, 3 Blue, 2 Gold, 2 Copper, 2 Blue Narrow, 2 Pink	24 pcs
31034	Crown Seating Kit	8 Clear LAB, 8 Black, 8 Gray, 8 White	32 pcs
31035	Veneer Finishing Kit	8 Black, 8 Gray, 8 White, 8 Blue	32 pcs
31026	Composite Finishing Kit	8 Orange, 8 Gray, 8 White	24 pcs
Hygienist Strip System			
35108	Hygienist Strip System	Pink	8 pcs
35116	Hygienist Strip System	Pink	16 pcs
35132	Hygienist Strip System	Pink	32 pcs
IPR Strip System			
32024	IPR Rainbow Set (0.07 - 0.50)	3 Yellow, 3 Red, 3 Dark Blue, 3 Green, 2 Clear, 2 Cyan, 2 Purple, 2 Brown, 1 Mint, 1 Aqua, 1 Turquoise, 1 Teal	24 pcs
32007	IPR Combined (0.07mm - 0.30mm)	1 Yellow, 1 Clear, 1 Cyan, 1 Red, 1 Dark Blue, 1 Green, 1 Purple, 1 Brown	8 pcs
32040	IPR Combined (0.07mm - 0.30mm)	8 Yellow, 8 Red, 8 Dark Blue, 8 Green, 2 Clear, 2 Cyan, 2 Purple, 2 Brown	40 pcs
IPR Assorted Kits			
32008	IPR Assorted (0.07mm - 0.20mm) Small	2 Yellow, 2 Red, 2 Dark Blue, 2 Green	8 pcs
32016	IPR Assorted (0.07mm - 0.20mm) Medium	4 Yellow, 4 Red, 4 Dark Blue, 4 Green	16 pcs
32032	IPR Assorted (0.07mm - 0.20mm) Large	8 Yellow, 8 Red, 8 Dark Blue, 8 Green	32 pcs
32004	IPR Strip System Intr	1 Yellow, 1 Red, 1 Dark Blue, 1 Green	4 pcs
36004	IPR 40/50 Promotional	1 Mint, 1 Aqua, 1 Turquoise, 1 Teal	4 pcs
Single			
32108	IPR Yellow Starter - Serrated (0.07mm)	Yellow	8 pcs
32116	IPR Yellow Starter - Serrated (0.07mm)	Yellow	16 pcs
32132	IPR Yellow Starter - Serrated (0.07mm)	Yellow	32 pcs
32408	IPR Red Opener (0.12mm)	Red	8 pcs
32416	IPR Red Opener (0.12mm)	Red	16 pcs
32432	IPR Red Opener (0.12mm)	Red	32 pcs
32508	IPR Blue Widener (0.15mm)	Blue	8 pcs
32516	IPR Blue Widener (0.15mm)	Blue	16 pcs
32532	IPR Blue Widener (0.15mm)	Blue	32 pcs
32608	IPR Green Extra Widener (0.20mm)	Green	8 pcs
32616	IPR Green Extra Widener (0.20mm)	Green	16 pcs
32632	IPR Green Extra Widener (0.20mm)	Green	32 pcs
IPR PLUS Assorted Kits			
32081	IPR PLUS Assorted Kit (0.10mm - 0.50mm) Small	1 Clear, 1 Cyan, 1 Purple, 1 Brown, 1 Mint, 1 Aqua, 1 Turquoise, 1 Teal	8 pcs
32082	IPR PLUS Assorted Kit (0.10mm - 0.50mm) Medium	2 Clear, 2 Cyan, 2 Purple, 2 Brown, 2 Mint, 2 Aqua, 2 Turquoise, 2 Teal	16 pcs
32084	IPR PLUS Assorted Kit (0.10mm - 0.50mm) Large	4 Clear, 4 Cyan, 4 Purple, 4 Brown, 4 Mint, 4 Aqua, 4 Turquoise, 4 Teal	32 pcs

Prod No	Product Description	Colour	Quantity
Single IPR PLUS			
32208	IPR PLUS Clear Opener Medium (0.10mm)	Clear	8 pcs
32216	IPR PLUS Clear Opener (0.10mm)	Clear	16 pcs
32232	IPR PLUS Clear Opener (0.10mm)	Clear	32 pcs
32308	IPR PLUS Cyan Opener (0.15mm)	Cyan	8 pcs
32316	IPR PLUS Cyan Opener (0.15mm)	Cyan	16 pcs
32332	IPR PLUS Cyan Opener (0.15mm)	Cyan	32 pcs
32708	IPR PLUS Purple Widener (0.25mm)	Purple	8 pcs
32716	IPR PLUS Purple Widener (0.25mm)	Purple	16 pcs
32732	IPR PLUS PurpleWidener (0.25mm)	Purple	32 pcs
32808	IPR PLUS Brown Widener (0.30mm)	Brown	8 pcs
32816	IPR PLUS Brown Widener (0.30mm)	Brown	16 pcs
32832	IPR PLUS Brown Widener (0.30mm)	Brown	32 pcs
36608	IPR PLUS Mint Widener (0.35mm)	Mint	8 pcs
36616	IPR PLUS Mint Widener (0.35mm)	Mint	16 pcs
36632	IPR PLUS Mint Widener (0.35mm)	Mint	32 pcs
36708	IPR PLUS Aqua Widener (0.40mm)	Aqua	8 pcs
36716	IPR PLUS Aqua Widener (0.40mm)	Aqua	16 pcs
36732	IPR PLUS Aqua Widener (0.40mm)	Aqua	32 pcs
36808	IPR PLUS Turquoise Widener (0.45mm)	Turquoise	8 pcs
36816	IPR PLUS Turquoise Widener (0.45mm)	Turquoise	16 pcs
36832	IPR PLUS Turquoise Widener(0.45mm)	Turquoise	32 pcs
36908	IPR PLUS Teal Widener (0.50mm)	Teal	8 pcs
36916	IPR PLUS Teal Widener (0.50mm)	Teal	16 pcs
36932	IPR PLUS Teal Widener (0.50mm)	Teal	32 pcs
32009	IPR PLUS Optional Assorted Kit (0.10mm - 0.25mm)	2 Clear, 2 Cyan, 2 Purple, 2 Brown	8 pcs
32017	IPR PLUS Optional Assorted Kit (0.10mm - 0.25mm)	4 Clear, 4 Cyan, 4 Purple, 4 Brown	16 pcs
32033	IPR PLUS Optional Assorted Kit (0.10mm - 0.25mm)	8 Clear, 8 Cyan, 8 Purple, 8 Brown	32 pcs
36008	IPR PLUS Optional Assorted Kit (0.35mm - 0.50mm)	2 Mint, 2 Aqua, 2 Turquoise, 2 Teal	8 pcs
36016	IPR PLUS Optional Assorted Kit (0.35mm - 0.50mm)	4 Mint, 4 Aqua, 4 Turquoise, 4 Teal	16 pcs
36032	IPR PLUS Optional Assorted Kit (0.35mm - 0.50mm)	8 Mint, 8 Aqua, 8 Turquoise, 8 Teal	32 pcs

Subgingival Strip System

33108	Subgingival Strip Assorted System	2 Blue, 3 Gold, 3 Copper	8pcs
33124	Subgingival Strip Assorted System	8 Blue, 8 Gold, 8 Copper	24 pcs
33408	Subgingival Strip System GOLD	Gold	8 pcs
33416	Subgingival Strip System GOLD	Gold	16 pcs
33432	Subgingival Strip System GOLD	Gold	32 pcs
33708	Subgingival Strip System BLUE	Blue	8 pcs
33716	Subgingival Strip System BLUE	Blue	16 pcs
33732	Subgingival Strip System BLUE	Blue	32 pcs
33608	Subgingival Strip System COPPER	Copper	8 pcs
33616	Subgingival Strip System COPPER	Copper	16 pcs
33632	Subgingival Strip System COPPER	Copper	32 pcs

LAB Strip System

13032	LAB Strip Assorted System	8 Clear, 8 Cyan, 8 Black, 8 Gray	32 pcs
13108	LAB Strip	CLEAR	8 pcs
13116	LAB Strip	CLEAR	16 pcs
13132	LAB Strip	CLEAR	32 pcs
13208	LAB Strip	BLACK	8 pcs
13216	LAB Strip	BLACK	16 pcs
13232	LAB Strip	BLACK	32 pcs
13308	LAB Strip	GRAY	8 pcs
13316	LAB Strip	GRAY	16 pcs
13332	LAB Strip	GRAY	32 pcs
13508	LAB Strip	Cyan	8 pcs

Accessories

92106	Incremental Thickness Gauge		
92501	Strip Tray		1 pc
92504	Strip Tray		4 pcs



ContacEZ

Design By Dentists

Directa's products are developed, tested and evaluated by our selected team of highly-qualified and renowned Swedish and international dental professionals. The aim is to make life easier for all dental professionals in their daily clinical work.

DIRECTA

DIRECTA AB

Finvids väg 8-10
194 47 Upplands Väsby, SWEDEN
+46 8 506 505 75

DIRECTA INC

64 Barnabas Road, Unit 3
Newtown, CT 06470, United States
203.491.2273

DIRECTA GERMANY

Hammergutstr. 11
92245 Kümmerbruck, GERMANY

info@directadental.com
www.directadental.com
www.directadentalgroup.com
www.facebook.com/directadental
www.instagram.com/directadental
www.youtube.com/user/DirectaAB

BRO_ContacEZ_en_2023-04-24_REV07



DirectaDentalGroup

DIRECTA **topdental** **ORSING** **parkell** **KOHLER** RÖNVIG
TrollDental **Sendoline** **physics** **CONTAC EZ** **ceramir**

The contents of this brochure including layout and images are protected by copyright ©2023 Directa AB. All rights reserved.