Use carefully, read the instructions for use.

**EN**

**G-enial® Universal Injectable**  
UNIVERSAL LIGHT-CURED RADIOPAQUE INJECTABLE COMPOSITE  
For use only by a dental professional in the recommended indications.

#### RECOMMENDED INDICATIONS

- Direct restorative for Class I, II, III, IV and V cavities
- Fissure sealant
- Sealing hypersensitive areas
- Repair of (indirect aesthetic restorations, temporary crown & bridge, defect margins when margins are in enamel)
- Blocking out enucleus
- Liner or base
- Realisation of crowns & bridges, inlays and veneers using the indirect technique in combination with GRADIA or GRADIA PLUS components (please refer to their dedicated IFU).
- Splicing of teeth in combination with fibres such as GC everStick fibres. Follow manufacturer's instructions.

#### CONTRAINDICATIONS


- Direct pulp capping.
- Avoid use of this product in patients with known methacrylate monomer and methacrylate polymer allergies.

#### OPERATION

##### 1. Preparations

- Hold the syringe upright and remove the wing cap by turning counterclockwise.
- Promptly and securely attach the dispensing tip to the syringe by turning clockwise.
- Place the light protective cover unit use.

**Note:**  
Take care not to attach the dispensing tip too tightly. This may damage its screw. In order to ensure a tight connection make sure that the threads are free of remnant material.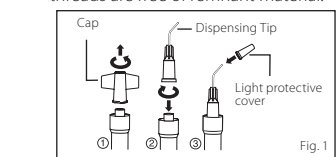

##### 2. Shade Selection

Select shade from 16 shades of XBW, BW, A1, A2, A3, A3.5, A4, B1, B2, CV, CVD, AO1, AO2, AO3, JE, AE. A, B, C, D shades are based on Vita® Shades.

\*Vita® is a registered trademark of Vita Zahnfabrik, Bad Säckingen, Germany.

##### 3. Cavity Preparation

Prepare cavity using standard techniques. Dry by gently blowing with oil free air. Rubber dam is recommended to isolate the prepared tooth from contamination with saliva, blood or sulcus fluid.

**Note:**  
For pulp capping, use calcium hydroxide.

##### 4. Bonding Treatment


For bonding G-enial Universal Injectable to enamel and/or dentin, use a light-cured bonding system such as GC G-Premio BOND, GC G-enial Bond or GC G-BOND. Follow manufacturer's instructions.

##### 5. Placement of G-enial Universal Injectable

- Remove the cover from the dispensing tip.

**Note:**  
Prior to syringe material into the cavity, make sure to check the following:

- Dispensing tip is securely attached to the syringe.
- Any possible air in the dispensing tip is removed by gently pushing forward the syringe plunger with the tip pointing upwards until material reaches the opening of the tip (Fig. 2).

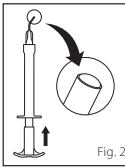- Place the dispensing tip as close as possible to the prepared cavity, and slowly push the plunger to extrude material. Alternatively, dispense material onto a mixing pad and transfer to the cavity using a suitable instrument.

- Plugged, remove the dispensing tip and extrude a small amount of material directly from the syringe.
- Minimise exposure to ambient light. Ambient light can shorten the manipulation time.
- After use, immediately remove and dispose of the dispensing tip and tightly close the syringe with the wing cap.

#### Clinical Hints

In order to inject effectively, use the surface tension of the material to ensure uniformity across the entire surface of the restoration during build up. Once the required amount has been injected, release the pressure on the plunger and withdraw the syringe in a direction perpendicular to the surface. This will allow the material to separate from the dispensing tip and provides a smooth surface over the restoration.

#### 6. Light-curing

Light-cure G-enial Universal Injectable using a light curing unit. Keep light guide as close as possible to the surface. Refer to the following chart for Irradiation Time and Effective Depth of Cure.

#### Irradiation Time and Effective Depth of Cure (ISO 4049)

|   | 10 sec. (High power LED) (> 1200mW/cm <sup>2</sup> )<br>20 sec. (Halogen/LED) (> 700mW/cm <sup>2</sup> ) |
|---|--|
| A1, A2, A3, B1, B2, JE, AE                | 2.5mm  |
| XBW, BW, A3.5, A4, AO1, AO2, CV, CVD, AO3 | 2.0mm  |

**Note:**

- Material should be placed and light-cured in layers. For maximum layer thickness, please consult above table.
- Lower light intensity may cause insufficient curing or discoloration of the material.
- For the indirect technique light-cure each layer for 30 seconds with GC LABOLIGHT LV-III, II or for 10 seconds with GC STEPLIGHT SL-I. Final cure for 3 minutes using GC LABOLIGHT LV-III.

#### 7. Finishing and Polishing

Finish and polish using standard techniques.

**SHADES**  
XBW, BW, A1, A2, A3, A3.5, A4, B1, B2, CV, CVD, AO1, AO2, AO3, JE, AE

#### STORAGE

Recommended for optimal performance, store in a cool and dark place (4 - 25°C) away from high temperatures or direct sunlight. (Shelf life: 3 years from date of manufacture)

#### PACKAGES

- Syringe 1.7g (1.0mL) x1, Dispensing Tip Long Needle x10, Light-protective cover x1
- Dispensing Tip Refill, Dispensing Tip Long Needle x30, Light-protective cover x3

#### AUTION

- In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or cocoa butter can be used to isolate the operation field from the skin or oral tissue.

**GC**

MANUFACTURED BY  
GC DENTAL PRODUCTS CORP.  
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED BY  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240,  
Interleuvenlaan 33, B-3001 Leuven, Belgium  
TEL: +32 16 74 10 00

PRINTED IN BELGIUM

In case of contact with eyes, flush immediately with water and seek medical attention.

- Take care to avoid ingestion of the material.
- Do not mix with other similar products.
- Dispensing tips are for single use only. To prevent cross-contamination between patients, do not reuse the dispensing tip. The dispensing tip cannot be sterilised in an autoclave or chemically.

- Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
- Wear protective eye glasses during light curing.
- When polishing the polymerised material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
- Avoid getting material on clothing.
- In case of contact with unintended areas of tooth or prosthetic appliances, remove with instrument, sponge or cotton pellet before light-curing.
- Do not use G-enial Universal Injectable in combination with eugenol containing materials as eugenol may hinder G-enial Universal Injectable from setting.
- Do not wipe the dispensing tip with excessive alcohol as this may cause contamination of the paste.
- Dispose of all wastes according to local regulations.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarise yourself with the safety data sheets available at <http://www.geeurope.com>. They can also be obtained from your supplier.

#### CLEANING AND DISINFECTING:

MULTI-USE DECONTAMINATION SYSTEMS: to avoid cross-contamination between patients, this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

**DO NOT IMMERGE.** Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional/national guidelines.

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