

PermaFlo® Purple




Certified
GF
Gluten-Free




PermaFlo Purple is used with an adhesive system to create an easily identified coronal seal. The purple color simplifies location of the pulp chamber floor when accessing the pulp chamber for future therapy.


Courtesy of Dr. Carlos Ramos.

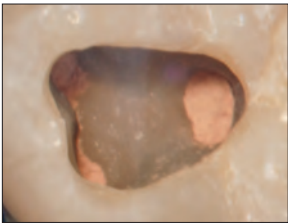

1. Root canal has just been completed and cleaned of excess EndoREZ canal sealer in the pulp chamber. (If significant unset EndoREZ canal sealer is exposed at canal orifice, coat with thin layer of Ultra-Blend® plus liner and light cure.) Blot or air dry. Note: If eugenol or similar-based sealers have been used, wait until set and freshen all chamber and/or preparation surfaces with diamond bur prior to bonding.

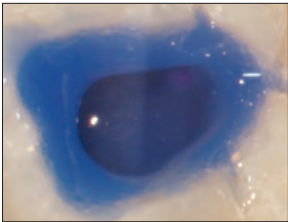

2. Etch and place Peak® Universal Bond adhesive; light cure. Optional: Use Peak® SE Primer and Peak® Universal Bond adhesive.

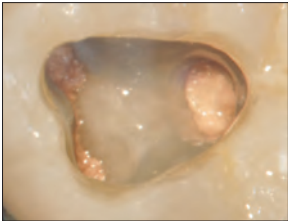

3. Apply a 1–1.5 mm-thick layer of PermaFlo Purple. Light cure 20 seconds to create an immediate “coronal seal.” When a post and/or core is prepared, the purple identifies the extent of root canal preparation and/or pulp chamber. The contrast shows the clinician the pulp chamber floor in relation to the canal orifices, minimizing risk of perforation.

Note: Apply dentin bonding agent first. Remember that eugenol-containing sealers can prevent polymerization of bonding resins. We recommend EndoREZ hydrophilic resin sealer.

962

PermaFlo Purple Syringe Kit

2 x 1.2 ml syringes
4 x Micro 20 ga tips



Prefilled Whitening Trays, see page 155.

Opalescence
go®



MY SMILE IS
Radiant
#MYSMILEISPOWERFUL