ELASTI-DAM® SIMPLE DAM KIT - Reorder No. H02900, H02902

Caution: Federal (United States) law restricts this device to sale by or on the orders of a licensed healthcare practitioner.

Caution: Intended for use by dental professionals during dental procedures.

- Dental Dam - Latex 6x6
- Wedjets Stabilizer Cord

SINGLE-USE ONLY TO PREVENT CROSS CONTAMINATION

CONTAINS PRESENCE OF NATURAL RUBBER LATEX

Instrument Cleaning & Sterilization Protocol

- **Dental Dam Clamps / Dental Dam Forceps / Dental Dam Punch / Dental Dam Frame (Plastic):**

⚠ IMPORTANT: Inspect, clean and sterilize instruments before each use. Instruments should be replaced when they become distorted, worn or cracked. Dispose of according to local regulations. Failure to follow these instructions will compromise patient safety.

Rinse & Clean: Rinse and clean immediately following the procedure. Failure to clean will decrease the life of the instrument and can result in staining or corroding.

Ultrasonic Cleaning (recommended): Follow ultrasonic cleaner manufacturer's instructions. Rinse and remove excess materials prior to ultrasonic cleaning. Use a pH neutral, general purpose cleaning solution (Non Ionic Surfactants), such as, but not limited to BioSonic UC30 or UC31. Rinse cleaning agents from instruments. Inspect to see that all materials have been removed from instrument. Repeat if necessary. Allow instrument to dry.

Manual Cleaning: Use a stiff nylon cleaning brush to carefully scrub instrument. Use a pH neutral cleaning solution (Non Ionic Surfactants), such as, but not limited to BioSonic UC30 or UC31. Rinse cleaning agents from instrument. Repeat if necessary. Allow instrument to dry.

Autoclave: Follow autoclave manufacturer's instructions and ensure that maximum load is not exceeded. Lubricate hinged instruments. Disposable sterilization pouches are recommended. For Clamps, Forceps and Punch, steam sterilize for 15 minutes at 134°C / 273°F. For plastic Dam Frame, steam sterilize for 20 minutes at 121°C / 250°F. Do not heat above 177°C / 350°F. Allow instrument to dry.

Instructions for Use:

Dental Dam Clamp: Tie a safety ligation (dental floss) around the clamp leaving sufficient floss to retrieve the clamp should the clamp break or be-


come dislodged. Failure to follow these instructions will compromise patient safety. Pre-fit the clamp to insure that there is a four-point contact with the anchor tooth's proximal surfaces. Using Winged Technique, the dam is placed on the frame, and the clamp is placed on top of the dam, while the dam is stretched over the wings of the clamp. The clamp forceps secure the clamp in place and the dental dam is slipped off the wings of the clamp.

Dental Dam Forceps: Engage the clamp by placing the tips of the forceps into the holes in the clamp. Spread forceps open so slight tension is applied to clamp. Overextension could cause distortion of the clamp and premature wear, including breakage.

Dental Dam Punch: Punch dagger/pin must be sharp and clean to prevent nicking or tearing the dental dam. For maximum cutting efficiency, remove any residual rubber from punch disc with an explorer. Using cutting disc, select appropriate hole size (1-5) (see Diagram 1). Use a dental dam template or rubber stamp to mark the hole positions on the dental dam. To properly punch the holes in the dam, align the dagger/pin with the hole marks on the dam, puncture the dam material while pulling the dam up over the point of the punch.

Dental Dam Frame (Plastic): Place the dam on top of the frame and stretch the material tautly while securing the dam on retention tines (points) (Do not puncture dam). Smooth dental dam on the frame for maximum retraction and stability.

Warranty: Coltene/Whaledent (C/W) warrants these products (clamps/forceps/punch/frame) to be free from defects in workmanship and materials for a period of twenty-four (24) months from the date of original purchase. During such warranty period, C/W undertakes to replace or repair this product, at its option, if such product does not function properly under normal use and service, which malfunction is due solely to a defect in workmanship or material. For warranty questions, please contact info@coltene-whaledent.com.

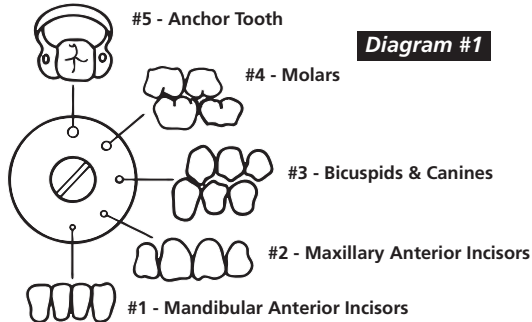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© 2015 04.10.2015
Coltène/Whaledent Inc.
Coltène/Whaledent GmbH + Co. KG
Coltène/Whaledent AG