# VOCO

## Meron Plus QM

CE 0482

**EN** Instructions for use  
**MD** EU Medical device  
In accordance with DIN EN ISO 9917-2

### Product description:

**Meron Plus QM** is a radiopaque, self-curing, fluoride-releasing, resin-modified glass ionomer luting cement available in a paste-paste formula in the practical QuickMix syringe for quick and simple direct application. **Meron Plus QM** offers optimal removal of excess material following light-curing.

### Indications:

- Luting of
- metal-based inlays, onlays, crowns and bridges
  - high-strength (e.g., zirconium dioxide-based or lithium disilicates, etc.) all-ceramic inlays, onlays, crowns and bridges
  - metal, ceramic and glassfibre-reinforced posts
  - orthodontic bands and aids

### Contraindications:

**Meron Plus QM** contains polyacrylic acid, peroxide, methacrylates and BHT. **Meron Plus QM** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

### Patient target group:

**Meron Plus QM** is suitable for use in all patients without any age or gender restrictions.

### Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

### User:

**Meron Plus QM** should only be used by a professionally trained dental practitioner.

### Use:

#### 1. Preparation:

Bring material to room temperature before use. Ensure that the surfaces to be cemented are clean. Thoroughly clean and dry enamel and dentine, and the inside surfaces of the restoration. Before luting, clean the stumps (e.g., pumice, water), rinse thoroughly and dry, ideally with a foam pellet or gentle blasts of air (take care not to overdry!). Prepare and treat the restoration as per the manufacturer's instructions.

Remove the protective cap from the QuickMix syringe. Due to technical reasons, there may be slight differences in the filling levels of the syringe chambers prior to first use. For this reason, when using a new syringe, dispense paste until the material exits both openings evenly. When reusing syringes, check the openings for any blockages and unblock as necessary. Attach a type 10 mixing tip in such a way that the notches on the mixing tip and the syringe are aligned. Turn the mixing tip 90° clockwise to lock the tip into place. To ensure even mixing of the two pastes, dispense and discard a little material before commencing the application.

### Curing behaviour:

Working time:	2 min (23 °C, from start of mixing)
Removal of excess material:	a) in gel state (approx. 2 min after placement in mouth) or b) following light-curing (approx. 5 - 10 s per segment)
Setting time:	4,5 min (after placement in mouth)

#### 2. Placing the restoration:

Apply material to the preparation and/or the prepared restoration. Then place the restoration, applying gentle pressure so as to expel excess material. At 23 °C, the working time is 2 min from the start of mixing. Remove the used mixing tip and seal the syringe with the original twist cap until the next time it is required. The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

#### 3. Removal of excess material:

Excess cement is best removed in the gel phase, either following light-curing (5 - 10 s per segment) or following self-curing (approx. 2 min after placement), using a suitable instrument (e.g., scaler). Excess material in hard-to-reach areas should be removed first. Clean contact points and interproximal areas with dental floss. No material residue should remain in the sulcus, as this can result in irritation of the gingiva.

#### 4. Curing:

Curing is complete 4.5 min after placement. The restoration must be held in position during curing. Cured excess material can be removed with a finishing diamond or flexible polishing wheels. Finish the approximal areas with finishing/polishing strips.

### Warnings, precautionary measures:

- In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- Ensure sufficient residual dentine thickness to prevent pulp irritation. If necessary, protect the pulp or build up the core.
- Phenolic substances, especially preparations containing eugenol or thymol, inhibit the curing of **Meron Plus QM**. Therefore, do not use zinc oxide eugenol cements or other eugenol-containing substances in combination with **Meron Plus QM**.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

### Constituents (in descending order according to content):

Fluoroalicate glass, dental glass, HPMA, polyacrylic acid, water, HEMA, fumed silica, GlyDMA, UDMA, initiators, stabilisers, pigments

### Storage instructions and application method:

Storage at 4°C - 23°C. Refit the original twist cap to prevent the **Meron Plus QM** pastes drying out during storage. Do not use after the expiry date.

### Disposal:

Dispose of the product in accordance with local regulations.

### Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Meron Plus QM** must be reported to VOCO GmbH and the responsible authority.

### Note:

The Summary of Safety and Clinical Performance of **Meron Plus QM** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at [www.voco.dental](http://www.voco.dental).

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VC 60 AA1727 E1 0723 99 © by VOCO