VOCO

Admira® Protect

EN Instructions for use
MD EU Medical device

CE 0482

Product description:

Light-curing, fluoride-releasing ORMOCER®-based preparation for the treatment of dentine hypersensitivity.

Admira Protect completely reduces hypersensitivity, in most cases after only one application, and additionally protects against external influences and abrasion.

Indications:

- Treatment of hypersensitive dentine
- Treatment of cervical areas (including crown margins, after tooth cleaning or tartar removal, exposed cervical areas following surgery etc.)

Contraindications:

Admira Protect contains methacrylate, BHT and organic acids. **Admira Protect** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

Patient target group:

Admira Protect is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Admira Protect should only be used by a professionally trained dental practitioner.

Use:

1. Preparation

- Clean the teeth to be treated.
- Isolate areas to be treated with cotton rolls. Remove excess moisture with an airjet. Do not overdry dentine. The dentine surface should be slightly moist.
- Avoid any contamination with blood or saliva after cleaning.

2. Dispensing

- Dispense **Admira Protect** onto a **Single Tim** or a disposable brush.
- Alternatively, dispense **Admira Protect** onto a mixing pallet.

3. Application

- Apply **Admira Protect** with **Single Tim** or a disposable brush evenly on all dentine surfaces to be treated and allow to act for 20 s. Then disperse **Admira Protect** with a faint airjet and light-cure with a conventional polymerisation device for 10 s. The light output should be a minimum of 500 mW/cm². Apply a second layer of **Admira Protect**, disperse it with a faint airjet and light-cure for 10 s.

4. Removal of the inhibition layer

- Remove the oxygen-inhibited layer (soft surface) with a cotton pellet or a cotton roll. Check the gingival margins and, if necessary, remove any residual material.

Warnings, precautionary measures:

- Phenolic substances (e.g. eugenol-containing materials) disturb polymerisation and must not be brought into contact with **Admira Protect**.
- Avoid contact with the mucosa.
- In case of eye contact, rinse well with plenty of water and consult an ophthalmologist.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Constituents (in descending order according to content):

Acetone, BisGMA, methacrylate phosphoric acid ester, HEMA, UDMA, TEGDMA, methacrylate functionalised polyacid, ORMOCER® resin, initiators, stabilisers

Storage instructions and application method:

Close the **Admira Protect** bottle tightly after use. Store at temperatures of 4 °C to 23 °C (storage at 5 °C - 8 °C prolongs the shelflife of the preparation). Do not expose **Admira Protect** to direct sunlight or the chair light. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Admira Protect** must be reported to VOCO GmbH and the responsible authority.

Note:

The Summary of Safety and Clinical Performance of **Admira Protect** can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>). Detailed information can also be found at www.voco.dental.

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VC 60 BB1650 E1 0626 99 © by VOCO
ORMOCER® is a registered trademark of Fraunhofer Gesellschaft zur Förderung der angewandten Forschung e.V., München, Germany.