MicroCab+™ and MacroCab+™

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

The MicroCab+™ and MacroCab+™ dust cabinets are ideal for sandblast dust and handpiece grinding containment. It features a built in suction/filter system, as well as bright, LED lights for even illumination. The cabinets are compact and portable making it perfect for any lab or operatory.

USE

The cabinet is shipped with the power cord inside the unit. Prior to use, connect the power cord to the inlet on the back (see Figure 3) and into an electrical receptacle. The on/off switch under the integrated handle will operate the LED lights and suction/filter system.

When using the cabinet, only the fingertips with the MicroEcher™ (Danville Materials) or handpiece need to be placed in the cabinet. Directing the abrasive spray towards the back of the cabinet will help contain the abrasive. It may tend to escape if the spray is directed towards a hand hole.

PLACEMENT

Keep the cabinet at least 6 inches (15 cm) away from any wall (back or sides) to ensure proper air flow and performance. The cabinet should be placed on a stable work surface or laboratory table.

Do not position the AC/DC adapter in such a way as to block easy access to the power cord from the inlet on the back of the cabinet. Do not disconnect the power cord.

Do not set the cabinet in a location where liquids or chemicals are used. Ensure that no liquids are spilled into the unit.

WINDOW INSTALLATION

Four disposable windows are supplied with the original kit. Each window has a blue protective film covering it, which must be removed prior to use. The windows are installed by sliding the wide section up under the metal frame top (see Figure 1). By slightly bowing the window, the bottom edge is inserted into the front lip on the bottom of the window opening. Windows only need to be removed for replacement.

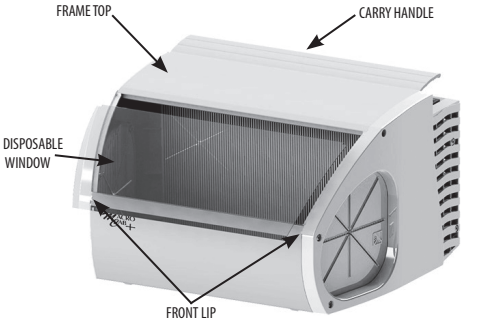
Figure 1

ELECTRICAL SHOCK HAZARD

For protection against electrical shock hazards, the equipment must be directly connected to an approved power supply with the power supply cord provided by the manufacturer. Do not use an ungrounded receptacle. Any break in the electrical ground path could be hazardous. If the power cord becomes cracked, frayed, broken, or otherwise damaged, please contact the manufacturer. Do not replace with a lower rated cord set.

CLEANING

The cabinet is easily opened for cleaning. Cleaning is to be done regularly depending on frequency of use (see Figures 2 and 3).

- Unplug the power cord from the inlet on the back of the cabinet.
- Open the two latches on the back of the cabinet.
- Remove the window.
- Grasping the top handle, pull the filter and cabinet top straight up and out of the cabinet body.
- Dump all debris out of the cabinet. The filter can be removed and tapped against the sides of a trash pail. This allows powder trapped in the filter fins to fall out. If the filter is removed from the filter assembly, reinstall it with the fins vertical.
- Reinstall the filter assembly by sliding it down into the cabinet. Ensure that the orange rubber of the filter slides behind the sealing fins that project into the cabinet from the side panels. There is also a channel under the front of the cabinet lid that captures the small rod running across the cabinet frame. This assures proper alignment.
- Reconnect the back cabinet latches and replace the window and power cord.

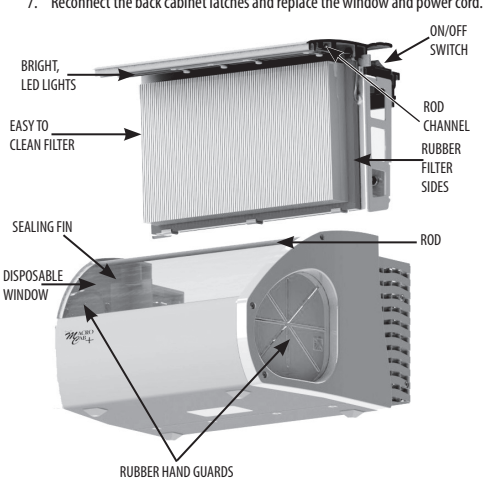
Figure 2

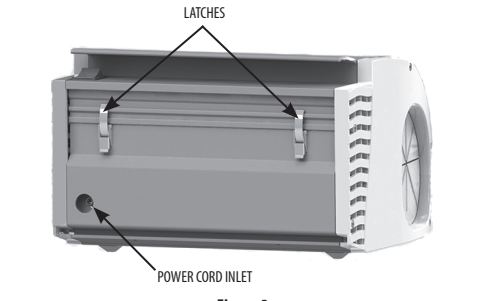
Figure 3

DISINFECTING

Once the filter assembly is removed the interior of the cabinet can be sprayed with disinfectant and wiped down after abrasive/contaminants have been discarded.

Environmental Conditions

The cabinet is designed for indoor operation in the following conditions: Altitude up to 2,000 m Temperature 5 °C to 40 °C Maximum relative humidity 80 % up to 31 °C decreasing linearly to 50 % at 40 °C Mains supply voltage fluctuations of AC/DC adapter up to ±10% of nominal voltage Transient over voltages of AC/DC adapter up to the levels of OVERVOLTAGE CATEGORY II, POLLUTION DEGREE 2

If the equipment is used in a manner not specified by the manufacturer, the protection provided by the equipment may be impaired.

TROUBLESHOOTING

If you encounter any problems with operation of the cabinet, contact the manufacturer for help.

PRODUCT SPECIFICATIONS


MicroCab+™
Size: 10.4" (265mm) x 8.7" (222mm) x 7.7" (195mm) (depth x width x height)
Weight: 5.7 lbs (2.6 kg)
Power Supply AC Input: 100-240V, 1.5A, 50-60Hz
Power Supply DC Output: 12.0V, 3.3A

MacroCab+™
Size: 10.4" (265mm) x 12.6" (320mm) x 7.7" (195mm) (depth x width x height)
Weight: 8.4 lbs (3.8 kg)
Power Supply AC Input: 100-240V, 1.5A, 50-60Hz
Power Supply DC Output: 12.0V, 3.3A

DEFINITIONS OF SYMBOLS

The following symbols may appear on the product packaging or labeling

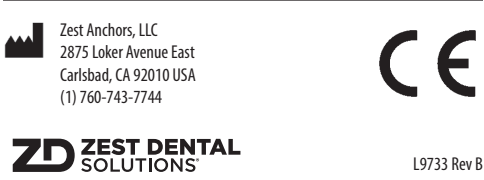SYMBOL	TITLE	REFERENCE
	Manufacturer	ISO 7000-3082
	Date of Manufacture	ISO 7000-2497
	Catalogue Number	ISO 7000-2493
	Batch Code	ISO 7000-2492
	Consult Instructions for Use	ISO 7000-1051
	Caution	ISO 7000-0434 B
	Waste electrical and electronic equipment	IEC 60950-1
	This way up	ISO 7000-0623
	Fragile, handle with care	ISO 7000-0621
	European Mark of Conformity	LVD 2014/35/EU
	Certification Mark	n/a
	Direct current	IEC 60417-5031

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Zest Anchors, LLC