


# Easy-to-Read 3M™ Attest™ Steam Chemical Integrators, Type 5

## Unprocessed

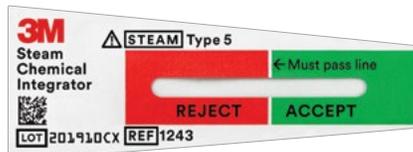


Attest Steam Chemical Integrators, Type 5 comply with ISO 11140-1:2014.

© Solventum 2025. Solventum, the S logo, Attest and other trademarks are trademarks of Solventum or its affiliates. 3M, the 3M logo and Attest are trademarks of 3M. Other trademarks are the property of their respective owners.


70-2009-8635-5

## Processed

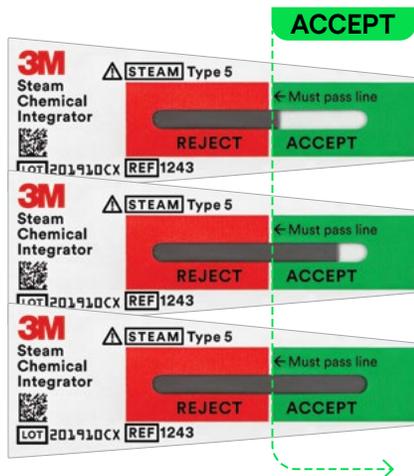

The color bar has reached the **ACCEPT** window in all three samples shown above, indicating that the necessary conditions for sterilization have been met. **If the color bar crosses into the ACCEPT region, it is considered a pass.**

## Processed

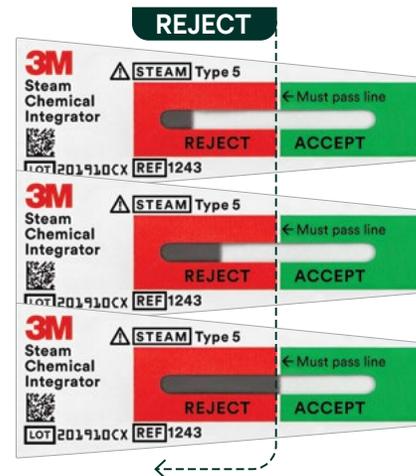

The color bar is in the **REJECT** window, indicating that the necessary conditions for sterilization have not been met. The pack should be reprocessed and the cause of sterilization failure should be investigated. **If the color bar is in the REJECT region or on the line, it is considered a fail.**