**VOCO****Structur 2 SC/QM****EN** Instructions for use  
MD EU Medical device**Product description:**

**Structur 2 SC/QM** is a fluorescent self-curing paste-paste system. **Structur 2 SC/QM** consists of base paste and catalyst paste. **Structur 2 SC** is supplied in 7 shades A1, A2, A3, A3.5, B1, C2 and BL. **Structur 2 SC** is provided in cartridges and QM syringes. Cartridges are to be inserted into the **VOCO** dispenser (type 2) or a comparable dispenser. **Structur 2 QM** is supplied in shades A1, A2, A3 and B1.

**Indications:**

Fabrication of temporary:  
– Crowns  
– Bridges  
– Inlays and onlays

**Contraindications:**

**Structur 2 SC/QM** contains methacrylates, amines, terpenes, benzoylperoxide and BHT. **Structur 2 SC/QM** should therefore not be used in patients with a known hypersensitivity (allergy) to these ingredients.

**Patient target group:**

**Structur 2 SC/QM** is suitable for use in all patients without any age or gender restrictions.

**Performance features:**

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

**User:**

**Structur 2 SC/QM** should only be used by a professionally trained dental practitioner.

**Use:****Usable mixing tips:**

**Cartridges:** mixing tips type 6  
**QM syringes:** mixing tips type 10

Due to technical reasons, there may be slight differences in the fill levels of both chambers prior to initial use. You should therefore remove the cap and continue to press out the material until an equal amount has been squeezed out of both outlets. Then attach a mixing tip and lock (with a 90° turn clockwise). The material is automatically mixed by the mixing tip in the correct ratio prior to application.

All QM syringes come with a detachable piston. The piston can easily be pulled backwards out of the syringe and then reinserted.

The seals in the syringe do not change their position whilst this is being done. This prevents cross-contamination of the materials in the syringe. The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories. The attached mixing tip is intended for single use only. After use, store the cartridge / QM syringe tightly sealed. Check that material flows freely through the openings before reusing.

**Processing directions:**

First, level out any undercuts in the preparation. Cavity linings and core build-ups made from composite materials should be insulated beforehand. Phenolic substances, especially products containing eugenol and thymol, lead to curing failures from composites. The use of zinc oxide eugenol cements or other eugenol-containing materials in combination with **Structur 2 SC/QM** should therefore be avoided.


Prepare an impression if prefabricated dental copings or similar restorative materials are not being used. Prepare the impression by levelling undercuts and cutting drainage channels. If the wall thickness of the temporary restoration is expected to be too thin, enlarge the impression, paying particular attention to removing the septums in the interdental embrasures. Then clean and air dry the impression.

Apply **Structur 2 SC/QM** directly into the impression. Do not overfill.

**Structur 2 SC/QM** attains an elastic phase after 1–1.5 min from beginning of mixing (approx. 0.5–1 min after insertion into the mouth).

The temporary restoration should be removed from the mouth within 1.5 min after mixing at the latest, as removal without any problems is only possible during the elastic phase. In addition, monitor the setting process by observing the excess material in the mouth. The setting time of material (in mixing tips) that has not been used cannot be compared with the setting time in the mouth.

The working times at a room temperature of 23°C and 50% humidity are as follows. Different room temperatures may result in variations from the above times:

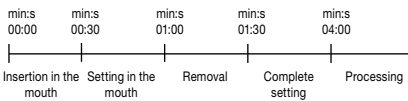After approx. 2–3 min take the temporary restoration out of the impression and wipe dry (if necessary, use a solvent, e.g. ethyl alcohol). This removes the thin inhibition layer usually caused by oxygen and allows easier finishing. If necessary, finish the temporary restoration with fine, cross-cut carbide burs. If required, polish the temporary restoration manually (e.g. with a rubber polisher, fine sandpaper or goat hair brush).

Do not start to finish the temporary restoration until the material has completely set (approximately 4 minutes after beginning to mix). Then carefully grind any excess material and rough areas near the gingiva. Wear a mask and protective goggles and use an extraction system. Cement the temporary restoration using an eugenol-free temporary cement.

**Repair of defective temporary restorations:**

If the temporary restoration should break despite the extreme stability of **Structur 2 SC/QM**, the following procedures are recommended:

- Air bubbles: fill with **Structur 2 SC/QM**.
- In the case of a fracture shortly after the manufacture of the temporary restoration: re-connect the fracture with freshly mixed **Structur 2 SC/QM**.
- Repair of temporary restorations that have already been worn: Roughen the fracture with a bur and place undercuts. Apply bond to the prepared fractures and cure according to the respective instructions for use. Apply a flowable composite to the prepared areas. Reassemble the fractured pieces and cure according to the instructions for use.

**Warnings, precautionary measures:**

- Composite-based cavity linings and core build-ups should be sufficiently insulated (e.g. with glycerine) to avoid removal with the temporary restoration.
- Before the temporary restorations are made, the prepared cores can be insulated to prevent hypersensitivity (by using fluoride varnishes, core varnishes and the like).
- In order to avoid fractures or injuries from debris, the temporary should not be subjected to excessive loads (molar area).
- Inadequate hygiene or the intensive use of certain mouth rinses may lead to shade variations.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.
- Combination only with eugenol-free products (cements)

**Storage:**

Store at 4°C–23°C. Do not use after its expiry date. Protect **Structur 2 SC/QM** from bright light, i.e. store in closed packages, drawers, etc.

**Disposal:**

Dispose of the product in accordance with local regulations.

**Reporting obligation:**

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Structur 2 SC/QM** must be reported to VOCO GmbH and the responsible authority.

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VC 60 BB1478 E1 0222 99 © by VOCO