



Back to Bright

Opalescence[®] Endo

- 35% hydrogen peroxide
- Specifically formulated to whiten endodontically treated, discolored, non-vital teeth
- Designed for “walking bleach” technique
- Convenient, premixed syringe delivery

Opalescence[®]
tooth whitening systems



ULTRADENT
PRODUCTS, INC.

800.552.5512 ULTRADENT.COM



Non-Vital “Walking Bleach”



Before



After



Before



After

Opalescence® Endo


1270 Opalescence Endo Mini Kit
2 x 1.2 ml Opalescence Endo syringes
20 x Black Mini tips



1323 Opalescence Endo Mini Refill
2 x 1.2 ml Opalescence Endo syringes



Must be refrigerated.

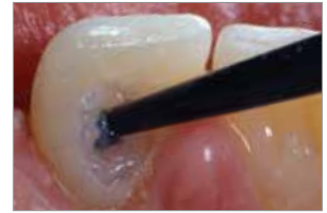

1. Before starting the Opalescence Endo whitening procedure, a glass ionomer should be placed on the floor of the pulp chamber to seal the obturation from penetration of the hydrogen peroxide.



2. Deliver mixed UltraTemp® Regular filling material directly to site.



3. Easily wipe away excess with a wet cotton swab or gauze before it sets.



4. Finished.



5. Remove easily after 3–5 days.

