Description

Monobond Etch & Prime is used to condition silica-based ceramic surfaces for the adhesive bond with luting composites from the Variolink or Multilink product lines, for example.

Monobond Etch & Prime etches and silanizes silicate ceramic surfaces in one working step.

Composition

Alcoholic aqueous solution of ammonium polyfluoride, silane methacrylate, phosphoric acid methacrylate and colourant.

Indication

- Conditioning of silica-based ceramic surfaces for the adhesive bond with luting composites

Contraindication

- Any use not listed in the indication
- If a patient is known to be allergic to any of the ingredients of Monobond Etch & Prime.
- Intraoral application

Side effects

Systemic side effects are not known to date. In rare cases, components of Monobond Etch & Prime may lead to sensitization. The product should no longer be used in such cases.

Interactions

The contamination of surfaces conditioned with Monobond Etch & Prime with saliva and blood must be prevented.

Usage**Conditioning of pretreated and non-pretreated restorations made of silicate ceramic**

(e.g. IPS Empress®, IPS e.max® Press, IPS e.max CAD)

- Following the try-in, thoroughly rinse the restoration with water spray and dry with water- and oil-free air.
- Apply Monobond Etch & Prime on the adhesive surface using a microbrush and agitate it into the surface for 20 seconds. Allow to react for another 40 seconds.
- Then thoroughly rinse off Monobond Etch & Prime with water and dry the restoration with a strong jet of water- and oil-free air for approximately 10 seconds.

Seat the conditioned restoration using a luting composite (e.g. Variolink® Esthetic or Multilink® Automix) in accordance with the respective Instructions for Use.

Additional notes

- To avoid any contamination, Monobond Etch & Prime must not be removed directly from the bottle using a microbrush.
- Carefully close the Monobond Etch & Prime bottle immediately after use.

Warnings

- The product contains alcohols. Keep away from sources of ignition.
- Monobond Etch & Prime is corrosive. Avoid contact with the skin and mucous membrane.
If the material should accidentally come in contact with the skin, immediately wash with copious amounts of water.
- In rare cases, contact with the skin may lead to sensitization towards the ingredients.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage

- Do not use Monobond Etch & Prime after the indicated date of expiration.
- Storage temperature 2–28°C (36–82°F)
- Shelf life: see expiry date on label

Keep out of the reach of children!**For use in dentistry only.**

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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www.ivoclarvivadent.com

Ivoclar Vivadent Pty. Ltd.

1 – 5 Overseas Drive, P.O. Box 367, Noble Park, Vic. 3174, Australia

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www.ivoclarvivadent.us

Ivoclar Vivadent Shanghai

Trading Co., Ltd., 2/F Building 1, 881 Wuding Road, Jing An District

200040 Shanghai, China

Tel. +86 21 6032 1657, Fax +86 21 6176 0968

www.ivoclarvivadent.com

Ivoclar Vivadent Marketing Ltd.

Calle 134 No. 7-B-83, Of. 520, Bogotá, Colombia

Tel. +57 1 627 3399, Fax +57 1 633 1663

www.ivoclarvivadent.co

Ivoclar Vivadent SAS

B.P. 118, F-74410 Saint-Jorioz, France

Tel. +33 4 50 88 64 00, Fax +33 4 50 68 91 52

www.ivoclarvivadent.fr

Ivoclar Vivadent GmbH

Dr. Adolf-Schneider-Str. 2, D-73479 Ellwangen, Jagst, Germany

Tel. +49 7961 889 0, Fax +49 7961 6326

www.ivoclarvivadent.de

Wieland Dental + Technik

GmbH & Co. KG, Schwenninger Strasse 13, D-75179 Pforzheim, Germany

Tel. +49 7231 3705 0, Fax +49 7231 3579 59

www.wieland-dental.com

Ivoclar Vivadent Marketing (India) Pvt. Ltd.

503/504 Raheja Plaza, 15 B Shah Industrial Estate

Veera Desai Road, A, ndheri (West), Mumbai, 400 053, India

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www.ivoclarvivadent.com

Ivoclar Vivadent Ltd.

12 Omega St, Rosedale, PO Box 303011 North Harbour

Auckland 0751, New Zealand

Tel. +64 9 914 9999, Fax +64 9 914 9990

www.ivoclarvivadent.co.nz

Ivoclar Vivadent Polska Sp. z o.o.

Al. Jana Pawla II 78, 00-175 Warszawa, Poland

Tel. +48 22 635 5496, Fax +48 22 635 5469

www.ivoclarvivadent.pl

Ivoclar Vivadent Marketing Ltd.

Prospekt Andropova 18 korp. 6/, office 10-06, 115432 Moscow, Russia

Tel. +7 499 418 0300, Fax +7 499 418 0310

www.ivoclarvivadent.ru

Ivoclar Vivadent Marketing Ltd.

Qlaya Main St., Siricon Building No.14, 2nd Floor, Office No. 204

P.O. Box 300146, Riyadh 11372, Saudi Arabia

Tel. +966 11 293 8345, Fax +966 11 293 8344

www.ivoclarvivadent.com

Ivoclar Vivadent Pte. Ltd.

171 Chin Swee Road, #02-01 San Centre, Singapore 169877

Tel. +65 6535 6775, Fax +65 6535 4991

www.ivoclarvivadent.com

Ivoclar Vivadent S.L.U.

C/ Ribera del Loira nº 46, 5ª planta, 28042 Madrid, Spain

Tel. + 34 913 757 820, Fax + 34 913 757 838

www.ivoclarvivadent.es

Ivoclar Vivadent AB

Dalvägen 14, S-169 56 Solna, Sweden

Tel. +46 8 514 939 30, Fax +46 8 514 939 40

www.ivoclarvivadent.se

Ivoclar Vivadent Liaison Office

: Tesvikiye Mahallesi, Sakayik Sokak, Nisantas' Plaza No:38/2

Kat:5 Daire:24, 34021 Sisli – Istanbul, Turkey

Tel. +90 212 343 0802, Fax +90 212 343 0842

www.ivoclarvivadent.com

Ivoclar Vivadent Limited

Ground Floor Compass Building, Feldspar Close, Warrens Business Park,

Enderby, Leicester LE19 4SE, United Kingdom

Tel. +44 116 284 7880, Fax +44 116 284 7881

www.ivoclarvivadent.co.uk

Ivoclar Vivadent, Inc.

175 Pineview Drive, Amherst, N.Y. 14228, USA

Tel. +1 800 533 6825, Fax +1 716 691 2285

www.ivoclarvivadent.us