Lastic[®] function Medium

Kettenbach GmbH & Co. KG
Im Heerfeld 7
35713 Eschenburg/Germany
www.kettenbach.com

Made in Germany
37065/4715

Gebrauchsinformation

Directions for use

Mode d'emploi

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Lastic[®] function Medium

Medium viscosity, elastomeric precision impression material based on polysiloxane.

ISO 4823, Type 2: medium-bodied consistency

Made in Germany
37065/4715

Range of indications

- Medium viscosity impression material for functional impressions.
- Medium viscosity impression material for relin impressions.

Working and setting times

Mixing time	Loading and placement time	Working time	Intraoral setting time	Setting time
1 min. by hand	1 min.	2 mins.	5 mins.	7 mins.

To ensure excellent impressions, the correct amounts must be dispensed and the total working time and intraoral setting time adhered to exactly. Should the temperature of the material deviate noticeably from 23 °C (74 °F), the total working and setting times will be affected.

Instructions for use/Technical data (ISO 4823)

① Dispensing

using Universal hardener paste

(REF 16113 – 60 ml Universal hardener paste)

Extrude equal lengths of base paste and catalyst paste onto the mixing pad.

10 cm of base paste (7.9 ml) + 10 cm of catalyst paste (0.4 ml). Mixing technique: Manual, with a mixing spatula.

When using the tube for the first time, pierce the membrane seal completely with the spike on the cap.

② Mixing

Mix the base paste and catalyst paste with the mixing spatula, for 30 seconds, until the mixture is homogeneously coloured, and has a bubble-free consistency.

Avoid entrapping air bubbles by repeatedly flattening the material out during mixing.

Insufficient/Excess doses of the activator in levels of up to 25 % prolong or abbreviate the setting time as well as the total setting time. Do not exceed the hardener paste dose by more than 25 % under any circumstances!

Storage conditions: In a dry place, at 18 – 25 °C (64 – 77 °F).
Disinfection: Impressions can be disinfected using, for example, Silosept[®].
Electroplating: Such impressions can be copper or silver plated.
Pouring the impressions: Within 2 – 24 hours, e.g. with high quality dental stone (class III - V).
Shelf life of the impression material: see crimped end of tube.
Linear dimensional change: -0.60 %
Elastic-Recovery test: 99.0 %
Strain in compression: 8.0 %

Warning!

- Do not leave remains of the impression material in the sulcus.
- Avoid contact with eyes. In case of accidental contact, rinse with copious amounts of water and consult a physician immediately.
- Avoid splattering this material onto clothing as it cannot be removed by dry cleaning.
- The possibility of allergic reactions to the product in sensitive persons cannot be fully excluded.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

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