



EZCAM KEY SETTING FOR IMAGE CAPTURE

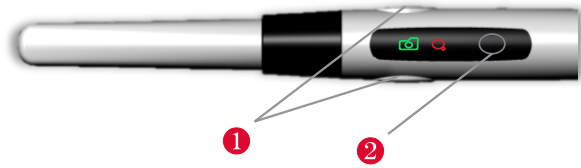

1. Connect the camera.
 2. Open Notepad or Word.
 3. Press buttons ❶ (either of two) and ❷ at the same time and wait for a while.
 4. A buzzer sounds after about 3 seconds.
 5. After the buzzer sounds, press the button ❶ repeatedly while keeping your finger pressed on the button ❷. Then notepad displays /a~/u in order.
- See the capture key setting table below and change the setting according to the software you use.
(*This information may change depending each software version upgrade.)

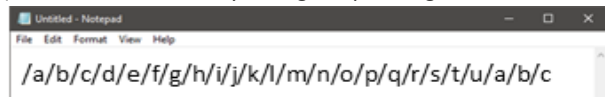

6. Lift the finger away from the button ❷ to complete the setting.

Company	Software	Code	Keyboard short-cut	Remark
Apteryx	Apteryx software	/c	Space	
Carestream	CS Imaging	/e	F5	
Dental eye	Dental eye	/d	Enter	
Dental vision	Dental vision	/p	Space -> Enter	
Dentrix	Dentrix	/g	Alt+C-> Alt+S	
DEXIS	DEXIS	/f	Space +T	
Faro	Dental focus	/d	Enter	
Gendex	Vixwin	/k	Alt+G - Space	
Genoray	Triana	/c	Space	
Image works	EVA Soft	/h	Alt+C -> Alt+L	
Instrumentarium	Cliniview	/c	Space	
Patterson Companies	Eaglesoft	/m	Alt+F -> Alt+S	Freeze -> Save
		/r	Alt+F -> Alt+U	Freeze -> Unfreeze
Planmeca	Romexis	/m	Alt+F -> Alt+S	
Schick	CDR DICOM	/n	Alt+F -> Alt+T	4.5 or lower
		/o	C -> U	5.X
Sirona	Sidexis	/l	Ctrl+Shift+V	
Soredex	Digora	/i	Space+S -> Space	
Suni	Suni	/c	Space	
Vatech	EasyDent	/c	Space	
	EzDent-i	/c	Space	
XDR	XDR	/q	Ctrl+I	XDR (old)
XDR (new)		/c	Space	
	CADI	/j	G	
	Tiger view7	/t	S	
	Tiger view8	/u	F -> S	
		/a	Joystick (Game port)	Switch1
		/b	Snapshoot	When using SDK
		/s	Ctrl+S	