VOCO**Provicol®/Provicol® C/****Provicol® QM**

CE 0482

In accordance with DIN EN ISO 3107

Instructions for use**Product description:**

Provicol/Provicol C/Provicol QM, a eugenol-free cement with calcium hydroxide, is easy to process and does not influence the setting of resin-based filling and luting materials.

Provicol/Provicol C/Provicol QM contains calcium hydroxide to support tooth vitality.

Indications:

Provicol/Provicol C/Provicol QM is used as a temporary cement for the attachment and for sealing of small one-surface cavities.

Method of use:**Provicol QM (in the QuickMix syringe)**

Due to technical reasons slight differences in the filling level between the syringe chambers can occur prior to the first use. Therefore remove syringe cap and squeeze the material out until the material is delivered evenly from both chambers. Place and lock the mixing tip type 10 with a 90° turn clockwise. The material is automatically mixed in the mixing tip.

All QM syringes come with a detachable piston. The piston can simply be pulled from the QM syringe and re-placed without causing any material leakage. The plugs in the syringe chambers remain in place preventing cross-contamination of the materials.

Provicol C (material in cartridges)

Insert **Provicol C** cartridge into a compatible dispenser with a suitable piston. Due to technical reasons, slight differences in the filling level between both cartridge chambers can occur. In this case, remove cartridge cap and squeeze the material out until the material is delivered evenly from both chambers. Then insert a mixing tip type 6 and lock with a 90° turn clockwise. By pressing the lever slowly and evenly, the material is extruded and automatically mixed in the correct ratio in the tip. After use, do not remove the mixing tip from the cartridge. Replace the used mixing tip with a new one just before the next application. When changing the mixing tips, check the opening of the cartridge to ensure that the material can flow freely.

Provicol (material in tubes)

Provicol should be used at room temperature (approx. 18 °C - 23 °C). Lower temperatures (use of a cooled glass plate) result in extended working times.

The **Provicol** catalyst and base paste are mixed in a 1 : 1 w/w ratio. For that purpose, put two equal lengths of base and catalyst paste on a mixing pad of strong, non-absorbing material or a glass plate.

Then mix the two pastes for approximately 20 s until the material reaches a homogeneous consistency. Use a spatula made of plastic or stainless steel for mixing.

To obtain a softer material and to keep **Provicol** workable for a longer period of time, add a small amount of vaseline.

A harder and faster setting material is obtained by adding zinc oxide (temporary sealing).

Application:

Apply a thin layer of **Provicol/Provicol C/Provicol QM** inside the temporary crown or bridge and affix it to the prepared tooth.

Provicol remains workable for about 1,5 - 2 min. **Provicol C/Provicol QM** about 1 min. All materials set in the mouth within 4 - 6 min.

Place the temporary into the mouth. Remove any excess material after 3 - 5 min.

Information, precautionary measures:

In case of opened pulp or inferior dentine thickness, an adequate pulp protection is recommended.

Provicol/Provicol C/Provicol QM contains mod. resin. **Provicol/Provicol C/Provicol QM** should not be used in case of known hypersensitivities (allergies) to this ingredient.

The sensitivity of cores can be reduced by use of desensitizing preparations prior to temporary luting. Remove these preparations completely before definite luting (do not use preparations containing eugenol!).

Extensive superficial and multisurfaced cavities as well as temporary longterm sealing work should be treated with a harder cement.

The warranted product characteristics can only be achieved when using the original Sulzer Mixpac mixing tips as approved by **VOCO**. All the mixing tips for use with our respective products are available from **VOCO**.

Supplementary indications:

When using Provicol in tubes, it is recommended to moisten the cores with some water.

Moisture (humid cores) additionally facilitates the removeability of the temporaries.

Residues of **Provicol/Provicol C/Provicol QM** can be removed from crowns easily after they have been sprayed for a short time with a cooling spray.

Store **Provicol/Provicol C/Provicol QM** at temperatures between 4 °C - 23 °C.

Provicol/Provicol C/Provicol QM corresponds to DIN EN ISO 3107.

Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA1074 E1 1117 99 © by VOCO
VOCO GmbH