Intensiv ProxoStrip Pat. 699 819

DE

Produktbeschreibung

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GB

Product description

- Metal strip, diamond-coated on one side only with two working parts of different grit size: 40µm (red) and 15µm (yellow)
- Perforated anatomical ends for a better gripping
- Height: 2.5mm
- Total length: 80mm
- Thickness: 0.05mm
- Sterilizable and reusable

Indications

- Finishing and polishing of fillings and crown borders in interproximal areas.
- Removal of excess material and overhangs in fillings and crown borders.
- Bevelling in adhesive restorations in cervical interproximal areas.

Practical Tips

- The total length of the strip (80mm) allows an optimal finger support on the adjacent teeth during work.
- To get past the contact area, use the uncoated part of the strip and exert a slight horizontal movement.
- After introduction, work with the less abrasive side (yellow, 15µm), in order to facilitate the access of the more abrasive side (40µm, red). Then begin the finishing procedure using a back-and-forward movement.
- Next, switch again to the less abrasive side (15µm, yellow) to complete the procedure with the polishing.
- Intensiv ProxoStrip must be removed with a vertical movement after repositioning it to the uncoated segment.
- In the event the contact area is very tight, it is useful to start by inserting a wooden wedge to open up a slight separation between the teeth before placing Intensiv ProxoStrip in the area to be treated.

Maintenance and sterilisation

- Intensiv ProxoStrip is supplied non-sterile. Prior to first use on the patient and immediately after each use, the instruments must be disinfected, cleaned, and sterilized.
- Keep ProxoStrip separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying. Chemiclave sterilizers may also be used.
- Concerning the sterilization process we refer to the ISO standard 17664; so we suggest to follow these advices:
 - Cycles at 134°C:
 - Tmin = 134°C – Tmax = 138°C
 - Pressure = 3.15 bar abs
 - Time = 4 min (raisable)
 - Cycles at 121°C:
 - Tmin = 121°C – Tmax = 125°C
 - Pressure = 2.10 bar abs
 - Time = 16 min (raisable)

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Intensiv ProxoPolish
Ref. PXPO1508/6 Packung zu 6 Strips

GB

Product description

- Metal strip, diamond-coated on one side only with two working parts of different grit size: 40µm (red) and 15µm (yellow)
- Anatomical ends in sterilizable plastic for a better gripping
- Height: 2.5mm
- Total length: 83mm
- Thickness: 0.05mm
- Sterilizable and reusable

Indications

- Finishing and polishing of fillings and crown borders in interproximal areas
- Removal of excess material and overhangs in fillings and crown borders
- Bevelling in adhesive restorations in cervical interproxi-mal areas

Practical Tips

- The color of the grips indicates the grit size of the strip, red 40 µm, yellow 15 µm; the opposite white part indica-tes the non-diamond coated side of the strip.
- The total length of the strip (83mm) allows an optimal finger support on the adjacent teeth during work.
- To get past the contact area, use the uncoated part of the strip and exert a slight horizontal movement.
- After introduction, work with the less abrasive side (yellow, 15µm), in order to facilitate the access of the more abrasive side (40µm, red). Then begin the finishing procedure using a back-and-forward move-ment.
- Next, switch again to the less abrasive side (15µm, yellow) to complete the procedure with the polishing.
- Intensiv ProxoStrip Plus must be removed with a vertical movement after repositioning it to the uncoated segment.
- In the event the contact area is very tight, it is useful to start by inserting a wooden wedge to open up a slight separation between the teeth before placing Intensiv ProxoStrip Plus in the area to be treated.

Maintenance and sterilisation

- Intensiv ProxoStrip Plus is supplied non-sterile. Prior to first use on the patient and immediately after each use, the instruments must be disinfected, cleaned, and steriliz-ed.
- Keep Intensiv ProxoStrip Plus separate from stainless steel instruments.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrati- ons recommended by the manufacturer.
- Sterilization must be performed with validated procedu- res. If possible, use a single-pulsed or fractionated vacu- um autoclave and subvacuum drying. Chemiclave steriliz- ers may also be used.
- Concerning the sterilization process we refer to the ISO standard 17664; so we suggest to follow these advices:
 - Cycles at 134°C:
 - Tmin = 134°C – Tmax = 138°C
 - Pressure = 3.15 bar abs
 - Time = 4 min (raisable)
 - Cycles at 121°C:
 - Tmin = 121°C – Tmax = 125°C
 - Pressure = 2.10 bar abs
 - Time = 16 min (raisable)

Intensiv ProxoStrip Plus (grit size 40µm and 15µm)
Ref. PXP4015/6 Package of 6 Strips
Ref. PXP4015/12 Package of 12 Strips

Additional models

Intensiv ProxoStrip Anterior (grit size 40µm and 15µm)
Ref. PXA4015/6 Package of 6 Strips
Ref. PXA4015/12 Package of 12 Strips

Intensiv ProxoStrip (grit size 40µm and 15µm)
Ref. PX4015/6 Package of 6 Strips
Ref. PX4015/12 Package of 12 Strips

Intensiv ProxoContour (grit size 60µm and 40µm)
Ref. PXC6040/6 Package of 6 Strips

Intensiv ProxoContour Coarse (grit size 80µm and 40µm)
Ref. PXCC8040/6 Package of 6 Strips

Intensiv ProxoPolish (grit size 15µm and 8µm)
Ref. PXPO1508/6 Package of 6 Strips

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