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PRESSURE INDICATOR PASTE (PIP)

Intended Use/Purpose/Indications

PIP Paste is intended to be used as a dental impression or articulation material, to check the seating of crowns, bridges, and full immediate and partial dentures. It is used to show the exact location, size, shape, and thickness of pressure areas on dentures that cause pain, discomfort, or displacement. It is thinly applied by brush to the dry denture/prosthetic which is then placed in position in the patient’s mouth. The patient is asked to bite down normally, or the dentist applies moderate hand pressure, the prosthetic is removed, and the dentist can view the pressure areas that need to be adjusted.

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Medical Conditions to be Treated: Prevent soreness and future adjustments for dental appliances. Allows user to identify pressure areas from dental appliances that may cause pain, discomfort, or displacement.

Intended User: The paste is intended to be used by dental professionals.

General directions (read carefully):

- Dry the denture surface to be tested. PIP will only adhere to dry surfaces.
- Brush the PIP over the area to be tested, so thin that the brush marks show.
- Be sure the tissues are wet with saliva, PIP does not stick to wet surfaces. If mouth is inclined to be dry, spray tissues and coat denture with PIP Spray.
- Position denture and apply moderate hand pressure. Excessive chewing or hand pressure should be avoided. Care should be taken so that dentures are not pressed so deeply into tissues that areas of excessive pressure cannot be distinguished from normal areas. If pressure areas are so mild that they do not show up immediately, leave denture in place longer, or apply more pressure.
- Remove denture and grind areas that show through the paste. Caution: Leave PIP on until after grinding.
- Wipe off the remaining PIP with cleansing tissues.

IMPORTANT - Repeat entire procedure several times until the pressure or overextension is no longer apparent.

Denture sore spots

- Locate the approximate area of soreness by asking whether it is upper or lower, right or left, front or back.
- Brush PIP thinly over the indicated quadrant of the denture and spray the painted area of the denture with PIP Spray.

Place denture in mouth and ask patient to "do whatever makes it hurt, open wide, move cheeks, etc."

For smooth, accurate, bubble-free alginate or hydrocolloid impressions:

Spray teeth and soft tissue with PIP Spray before taking impression. PIP Spray eliminates saliva bubbles and puts a free-flowing inert barrier over mucin and salivary chemicals that might otherwise react with alginate to produce a fuzzy impression.

For prevent powder, chalky stone models: After removing impression from mouth, immediately spray with PIP Spray, and then pour model material. The stone or plaster flows smoothly into every crevice, resulting in a smooth, hard, accurate model. PIP Spray encourages free flow of the stone and is one of the most effective bubble-preventatives known. PIP Spray forms an inert barrier between the acid of the alginate and the alkaline stone. Also, use PIP Spray to prevent bubbles in wax patterns and inlay impressions.

To prevent mirror fog: Spray your mirrors with PIP Spray. Because it is such an effective wetting agent, visibility will be greatly improved.

PIP Remover: To remove PIP and other impression materials from skin, dentures, instruments and other surfaces, use PIP Remover on a cloth or swab.

Disposal Considerations: PIP Paste waste is very toxic to aquatic life with long lasting effects and should not enter the sewage system. Dispose of unused materials in accordance with federal, state, and local regulations.

 Keystone Industries
52 West King Street
Myerstown, PA 17067 USA
(800) 333-3131 keystoneindustries.com
To order outside of the US call
+49 (0) 77 31 91 21 01

keystoneindustries.com/symbolglossary

Intended Use/Purpose/Indications: PIP Paste is intended to be used as a dental impression or articulation material, to check the seating of crowns, bridges, and full immediate and partial dentures. It is used to show the exact location, size, shape, and thickness of pressure areas on dentures that cause pain, discomfort, or displacement. It is thinly applied by brush to the dry denture/prosthetic which is then placed in position in the patient’s mouth. The patient is asked to bite down normally, or the dentist applies moderate hand pressure, the prosthetic is removed, and the dentist can view the pressure areas that need to be adjusted.

Medical Conditions to be Treated: Prevent soreness and future adjustments for dental appliances. Allows user to identify pressure areas from dental appliances that may cause pain, discomfort, or displacement.

Intended User: The paste is intended to be used by dental professionals.

General directions (read carefully):

- Dry the denture surface to be tested. PIP will only adhere to dry surfaces.
- Brush the PIP over the area to be tested, so thin that the brush marks show.
- Be sure the tissues are wet with saliva, PIP does not stick to wet surfaces. If mouth is inclined to be dry, spray tissues and coat denture with PIP Spray.
- Position denture and apply moderate hand pressure. Excessive chewing or hand pressure should be avoided. Care should be taken so that dentures are not pressed so deeply into tissues that areas of excessive pressure cannot be distinguished from normal areas. If pressure areas are so mild that they do not show up immediately, leave denture in place longer, or apply more pressure.
- Remove denture and grind areas that show through the paste. Caution: Leave PIP on until after grinding.
- Wipe off the remaining PIP with cleansing tissues.

IMPORTANT - Repeat entire procedure several times until the pressure or overextension is no longer apparent.

Denture sore spots

- Locate the approximate area of soreness by asking whether it is upper or lower, right or left, front or back.
- Brush PIP thinly over the indicated quadrant of the denture and spray the painted area of the denture with PIP Spray.

Place denture in mouth and ask patient to "do whatever makes it hurt, open wide, move cheeks, etc."

For smooth, accurate, bubble-free alginate or hydrocolloid impressions:

Spray teeth and soft tissue with PIP Spray before taking impression. PIP Spray eliminates saliva bubbles and puts a free-flowing inert barrier over mucin and salivary chemicals that might otherwise react with alginate to produce a fuzzy impression.

To prevent mirror fog: Spray your mirrors with PIP Spray. Because it is such an effective wetting agent, visibility will be greatly improved.

PIP Remover: To remove PIP and other impression materials from skin, dentures, instruments and other surfaces, use PIP Remover on a cloth or swab.

Disposal Considerations: PIP Paste waste is very toxic to aquatic life with long lasting effects and should not enter the sewage system. Dispose of unused materials in accordance with federal, state, and local regulations.

 Keystone Industries
52 West King Street
Myerstown, PA 17067 USA
(800) 333-3131 keystoneindustries.com
To order outside of the US call
+49 (0) 77 31 91 21 01

keystoneindustries.com/symbolglossary

New Dentures

Use of PIP at time of delivery can prevent soreness and future adjustments. This can be done in an orderly fashion by first checking the denture base and then proceeding with the borders. All new dentures should be checked at delivery. For patient’s comfort, check dentures with PIP when inserting immediately after surgery.

Partial Dentures

The fit of the cast framework will almost always be improved if they are checked in the mouth with PIP brushed inside the clasps, connectors, etc. Areas that prevent the framework from fitting properly into place are often found inside the clasp of its upright connector. When these specific areas are ground, as indicated by the PIP, the case will fit properly into place and be more comfortable.

Inlays

Applying the PIP to the inlay and pressing it into the cavity may find areas that prevent an inlay from seating. Reducing the specific areas that show through the PIP allows the inlay to fit properly into place. Excessive contacts may also be checked in this manner.

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