



# endosuccess apical surgery



## Apical surgery



**AS3D**  
F00065



**AS6D**  
F00079



**AS9D**  
F00067



**ASRD**  
F00081

**ASLD**  
F00080



**Diamond-coated universal tip 30µm, length 3mm, taper 9%**

Apical surgery of anterior teeth. It should be used without pressure, at the lowest possible effective power.



**Diamond-coated tip 30µm, length 6mm, taper 9%**

Second instrument in the sequence, used to obtain a preparation length of 5mm at least.



**Diamond-coated tip 30µm, length 9mm, taper 8%**

Used for complex cases and for the preparation of the root canal up to the coronal third.

The diamond coating is only present on the extremity of the instrument not to over-prepare the canal.

*The AS9D tip should first be introduced into the canal and oriented in the root axis before being activated to prevent the creation of a «false route».*



**Right-oriented tip, diamond-coated 30µm, length 3mm, taper 10%**

Apical surgery of premolars and molars.



**Left-oriented tip, diamond-coated 30µm, length 3mm, taper 10%**

Apical surgery of premolars and molars.

*It should be used with very light pressure.*