Oscillating Instruments

DE

Oszillierende Instrumente

Intensiv Bevelshape, Intensiv Cavishape, Intensiv Proxoshape and Intensiv Rootshape

Zu verwenden in Kombination mit:

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To be used in combination with the contra-angles:

- Intensiv Swingle, WG 69 LT Cons, with light or
- Intensiv Swingle, WG 69 A Cons, without light

Alternatively:

- KaVo 61 LRG or 61 LG
- KaVo contra-angle Intracompact E61C

Files

Intensiv Bevelshape:

Margin beveling in cavity, crown and chamfer preparations
Intensiv Swingle WG 69 LT / KaVo 61 LRG / KaVo 61 LG / KaVo E61C

Intensiv Cavishape:

Contouring and finishing of cavity margins
Intensiv Swingle WG 69 LT / KaVo E61C

Intensiv Proxoshape:

Contouring, finishing and pre-polishing of proximal filling surfaces
Intensiv Swingle WG 69 LT / KaVo 61 LRG / KaVo 61 LG / KaVo E61C

Intensiv Proxoshape Coarse:

Modelling of interproximal aesthetic restorations
Intensiv Swingle WG 69 LT / KaVo E61C

Intensiv Proxoshape Flexible:

Finishing and polishing of aesthetic restorations with difficult to access proximal areas. Disposal of excess and over hangings filling
Intensiv Swingle WG 69 LT / KaVo 61 LRG / KaVo 61 LG / KaVo E61C

Intensiv Rootshape:

Rootplaning and odontoplasty
Intensiv Swingle WG 69 LT / KaVo 61 LRG / KaVo 61 LG / KaVo E61C

Recommendations for use

- Set the instrument to the recommended speed before applying it to the tooth.
- Do not stop the instrument in the interdental space.
- In case of very tight contact points we suggest to separate the teeth by placing a wooden wedge cervically.
- Insert the instrument as far as possible.
- Provide an adequate waterspray cooling (minimum 50ml/min.).
- Pay attention to recommended speeds (10-40000 rpm) and contact pressures (1.0-2.0 N).
- Avoid tilting, levering or bending the instrument.
- Cleaning as for rotary diamond instruments.

Not following the instructions for use may lead to an increased safety risk, a breaking of the instrument cannot be excluded.

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Instructions for use and safety recommendations

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