# AKZENTA

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Breakthrough Performance level	Achieved breakthrough time (min)
Class 1	>10min
Class 2	>30min
Class 3	>60min
Class 4	>120min
Class 5	>240min
Class 6	>480min

Performance level	AQL	Inspection level
Level 3	<0,65	G1
Level 2	<1,5	G1
Level 1	<4,0	S4

Art. No.	EN ISO 374-1:2016+A1:2018		EN ISO 374-4:2019		EN ISO 374-5:2016	
	Permeation levels are based on breakthrough times	Breakthrough Performance Level	Resistance to chemical degradation (%)			
GLOBN51-AKZ-WH	40% Sodium Hydroxide (K)	6	-33.5			
GLOBN51-AKZ-LA	30% Hydrogen Peroxide (P)	4	22.1	Protection against bacteria and fungi - Pass		
GLOBN51-AKZ-IB	37% Formaldehyde (T)	6	-0.3	Protection against viruses - Pass		
GLOBN51-AKZ-BK						

GB

EN ISO 374-1:2016+A1:2018	EN ISO 374-1:2016+A1:2018	EN ISO 374-1:2016+A1:2018
Type A	Type B	Type C
UVWXYZ	XYZ	X
Type A - At least Breakthrough Performance Level Class 2 (more than 30 minutes) against at least 6 chemicals on the list. * Fig. 1	Type B - At least Breakthrough Performance Level Class 2 (more than 30 minutes) against at least 3 chemicals on the list. * Fig. 1	Type C - At least Breakthrough Performance Level Class 1 (more than 10 minutes) against at least 1 chemicals on the list. * Fig. 1

The EU Declaration of Conformity is accessible at [www.akzenta.com](http://www.akzenta.com).

EN ISO 374-1:2016+A1:2018 Protection against chemicals and microorganisms  
EN ISO 374-5:2016 Protection against bacteria and fungi. Protection against viruses.

AQL= Acceptable Quality Level that indicates the percentage (or amount) of defective gloves per production run.  
This product complies with the requirements of the Regulation (EU) 2016/425 on Personal Protective Equipment Category III, MDR (EU) 2017/745 concerning Medical Devices Class 1, Regulation (EC) No. 1935/2004 on Materials and Articles intended to come into Contact with Food.  
The gloves are tested in accordance with EN ISO 21420:2020 General requirements for gloves, EN ISO 374-1:2016+A1:2018 and EN ISO 374-5:2016, EN 455-1, EN 455-2, EN 455-3, EN 455-4, EN 455-5. These gloves are fit for special purposes. They are suitable to protect your hands only from chemical splashes when handling chemicals.

Do not use these gloves when protection in the cuff area is needed. This information does not reflect the actual duration of protection in the workplace and the differentiation between pure chemicals. The chemical resistance has been assessed under laboratory conditions from samples taken from the hand area and relates only to the chemical tested. If glove length is equal to or over 400 mm, the cuff is also tested. Results can be different if the chemical is used in a mixture. It is recommended to make sure that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation. When in use, protective gloves may provide less resistance to dangerous chemicals due to changes in physical properties. Movements, rubbing, degradation caused by the chemical contact, etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider when selecting chemical resistant gloves. The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimens. Before usage, inspect the gloves for any defects or imperfections. Protect from heat, moist, light and ozone. Keep unused gloves in the original packaging. Store in a cool dark room.

Disposable gloves must be discarded after use. The product material is not known to cause allergic reactions. If irritation of the skin should occur, wash the affected area with mild soap and water. Seek medical attention if the irritation persists.

Product Instructions for use  
a. Before usage, inspect the gloves for any defect or imperfections  
b. Usage is for single use only. If re-used, the risk of contamination and infection increases due to improper cleaning processes and increased risk of holes and tear during re-use due to weakening of

- gloves by cleaning processes.  
c. Sizing: Select the right size of glove for your hands.  
d. Donning: Hold the glove by the bead with one hand. Align the glove thumb with your other hand thumb and slide your hand into the glove, one finger into each glove finger. Pull by the glove palm to get a good fit. Don the other glove by the same procedure.  
e. Inspection: punctures or tears may occur after donning. Inspect the glove after donning and stop to use them immediately if a damage is found.  
f. Doffing: Hold the glove bead and pull toward the finger until the glove comes off.  
g. Disposal: For the proper disposal of all used gloves, please follow your institution's policies for disposal.

Cleaning

Not applicable as the glove is single use in intended manner.

Shelf Life

The shelf life of the product is 5 years from the date of manufacture.

Reporting

In the event any serious incident occurs in relation to the use of this device, please report the incident to the manufacturer or its authorized representative and the competent authority of the Member State in which the user is established.

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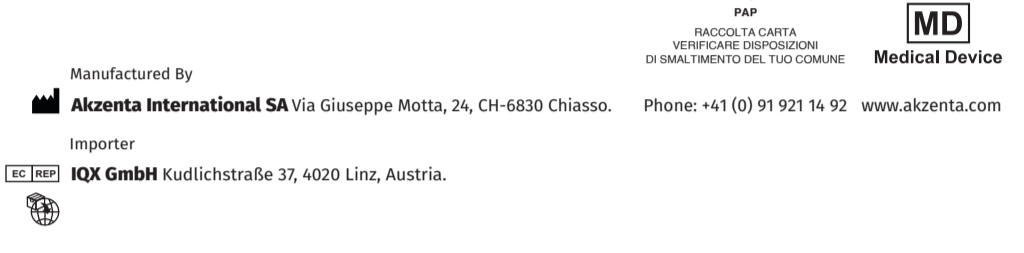
For single use only

EU DoC can be downloaded at : [www.akzenta.com](http://www.akzenta.com)

Art. No.	Model	Size
GLOBN51-AKZ-WH	Nitrile Smart   One	XS/S/M/L
GLOBN51-AKZ-LA	Nitrile Smart   One	XS/S/M/L
GLOBN51-AKZ-IB	Nitrile Smart   One	XS/S/M/L/XL
GLOBN51-AKZ-BK	Nitrile Art   One	XS/S/M/L/XL

AQL 1,5 EN 455 - 1, 2, 3, 4

These gloves are tested and approved by Notified Body: 2777 module B

SATRA Technology Europe Ltd Bracetown Business Park, Clonee, Dublin 15, Dublin, Ireland

Notified Body responsible for Module D Conformity Assessment:

Satra Technology Europe Ltd. Notified Body No. 2777 Bracetown Business Park, Clonee, Dublin 15 Dublin, Ireland

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