
MixStar eMotion







Automatic Mixing Device

Key assignment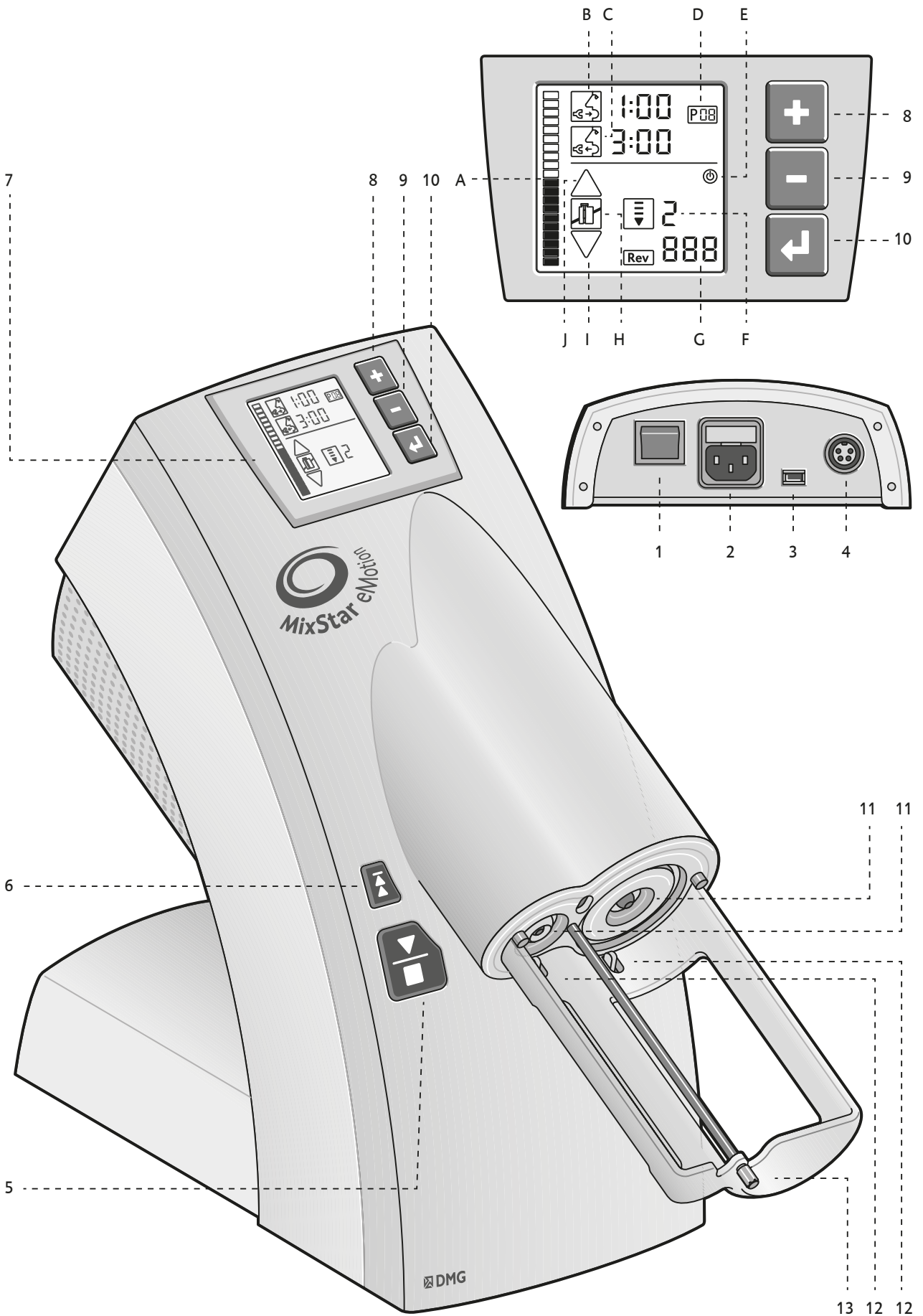

Handling

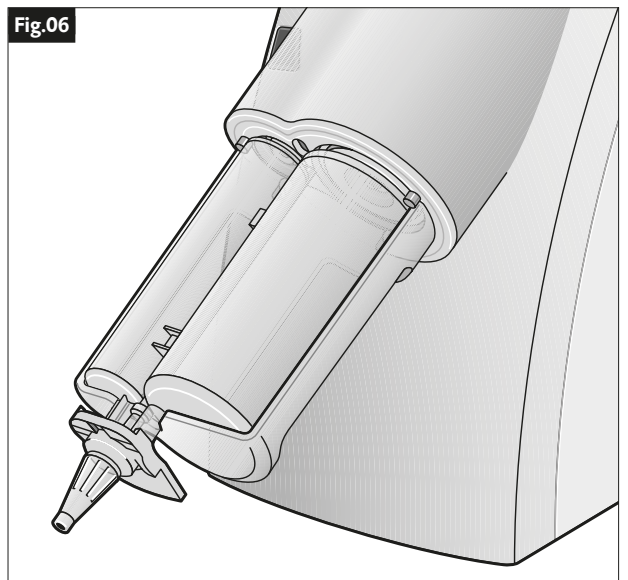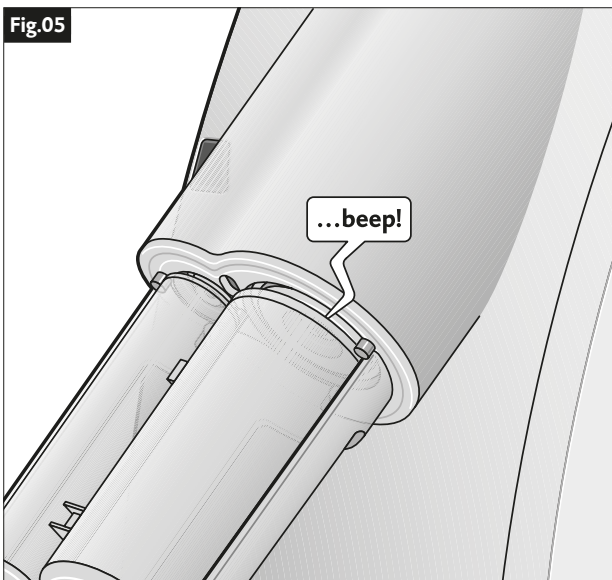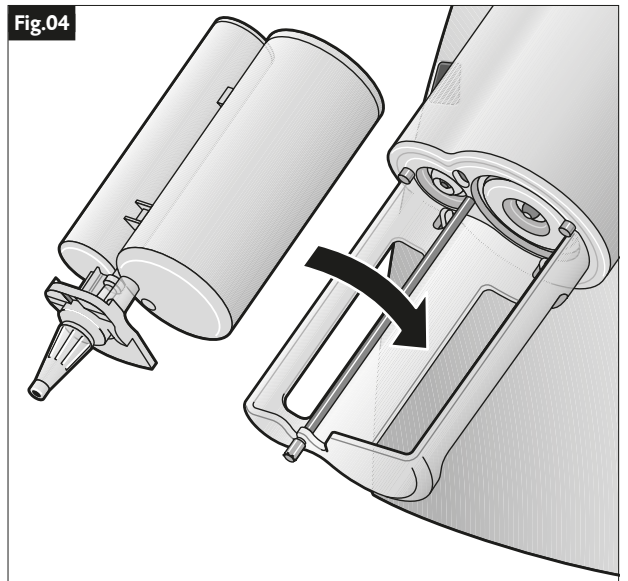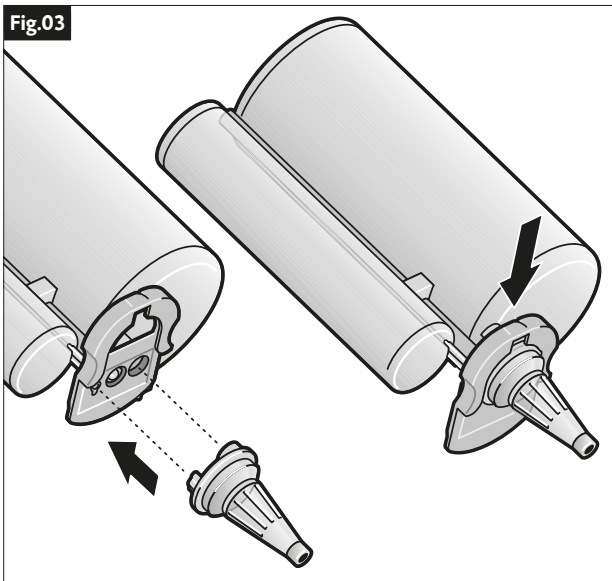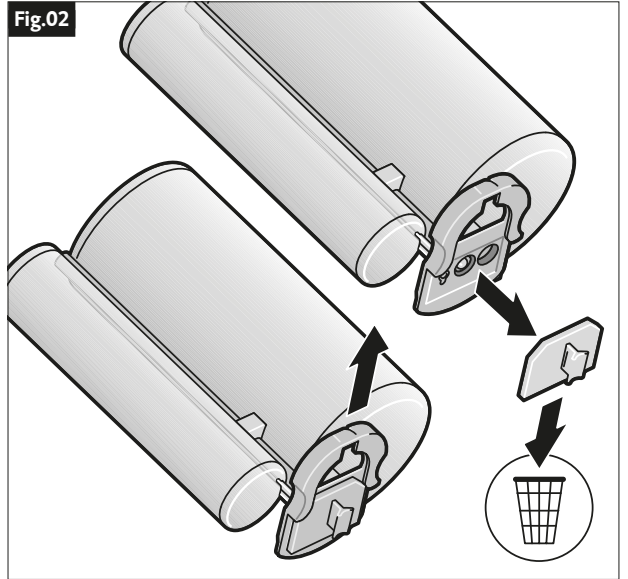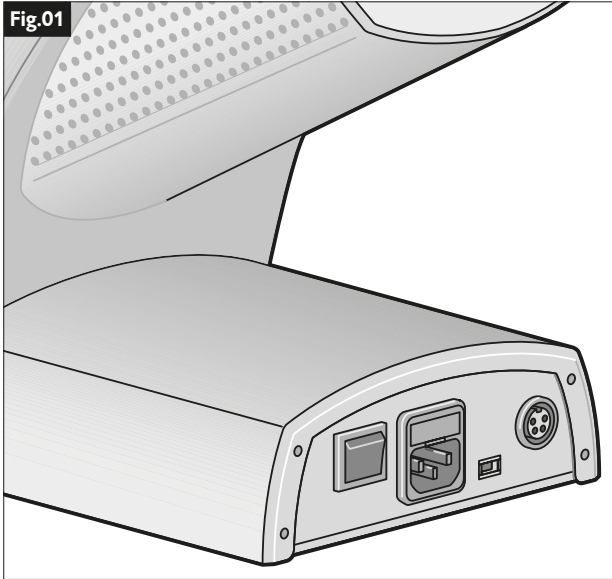Handling

Fig.07

Fig.08

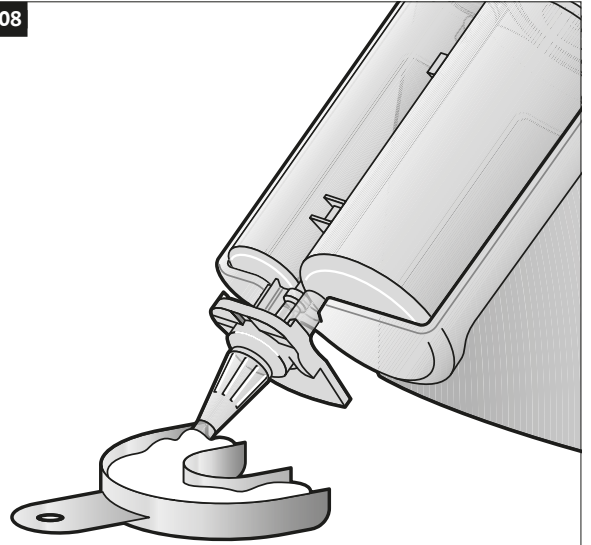
Fig.09

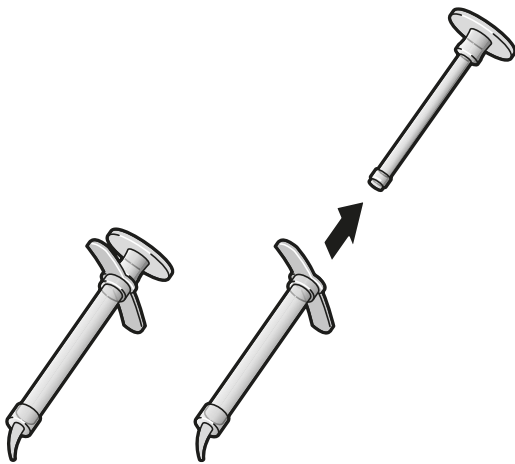
Fig.10

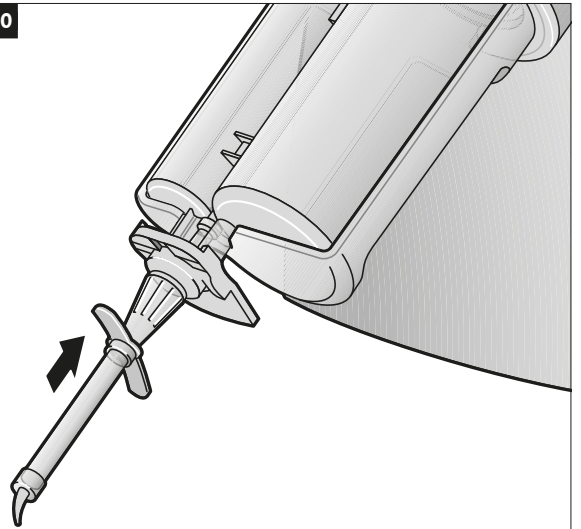
Fig.11


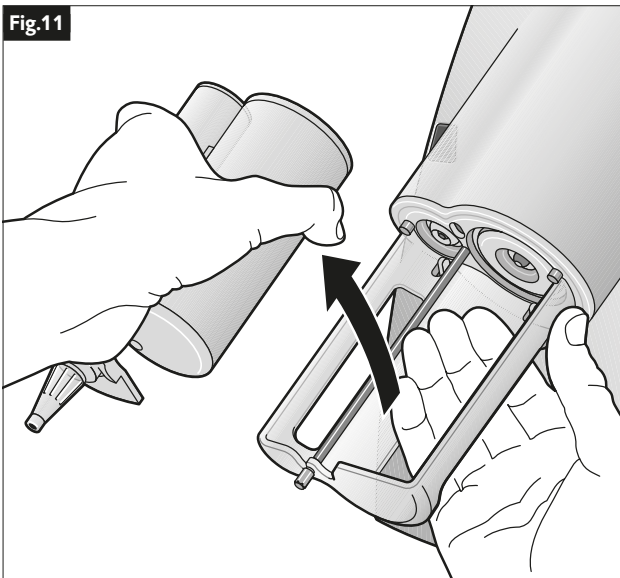
Fig.12

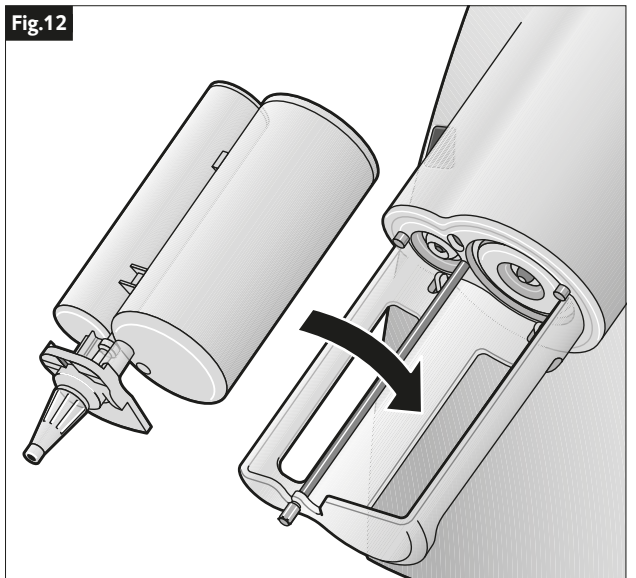Inhalt / Contents

DE	Gebrauchsinformation	1
EN	Instructions for use	5
FR	Mode d'emploi	9
IT	Istruzioni d'uso	14
ES	Instrucción de uso	19
PT	Instruções de uso	23
NL	Gebruiksaanwijzing	28
DA	Brugsanvisning	33
SV	Användarinstruktioner	37
PL	Instrukcja użycia	41
RU	Информация по применению	46
TR	Kullanım kılavuzu	50
RO	Instrucțiuni de utilizare	55
CS	Návod k použití	59
EL	Οδηγίες χρήσης	63
ZH	使用说明	68
JP	取扱説明書	72
KO	사용 정보	76

Gebrauchsinformation

Deutsch

Produktbeschreibung

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English

Product description

The new MixStar eMotion is a fully automatic mixing device for the mixing of impression materials. The device can be operated at 3 different speed levels, offering optimal individual comfort. Each speed level is continuously electronically monitored and regulated, ensuring that the speed is constant, even when working with viscous consistencies. To facilitate practice workflows, two optional timers for working time and intraoral setting time can be saved to programs. The pre-filled MixStar cartridges are marked with both times for easy coordination. Operation is simple, as the mixing process can be initiated with a short push of a button and the tray can be guided with both hands. An optional foot pedal switch can be connected to the device for hands-free operation. An integrated USB interface can be used for a simple software update.

Intended purpose

Automatic mixing of impression materials

Limitations of use

- Please only operate the MixStar eMotion using MixStar-compatible products. If other products are used, correct functioning cannot be guaranteed.
- Only use the MixStar eMotion for its intended purpose.

Indications

The indications for using MixStar eMotion are determined by the impression material selected for the dental treatment. This information can be found in the manufacturer's instructions for use for the selected material.

Contraindications

Contra-indications for using MixStar eMotion are determined by the impression material selected for the dental treatment. This information can be found in the manufacturer's instructions for use for the selected material.

Patient target group

Persons receiving treatment as part of a dental procedure.

Intended users

Dentist, dental assistant staff

Notes for use

Device and display functions

[01]
[02]
[03]

On/Off switch
Power connection with fuses
USB interface

[04]	Foot switch connection
[05]	Start/Stop button
[6]	Reverse button
[7]	Display
[8]	Plus button
[9]	Minus button
[10]	Menu button
[11]	Plunger discs
[12]	Cartridge recognition
[13]	Mixer shaft

Fill level indicator [A]

The amount of material remaining in the cartridge can be monitored using the fill level indicator. The fill level indicator flashes as long as the fill level cannot be determined or when the last two indicator segments have been reached and there is only enough material left in the cartridge to fill one more impression tray.

Timer 1 [B]

Timer 1 is used to set the working time for the selected material. The countdown begins when the mixing process starts. A long audio signal indicates the end of the working time period.

Timer 2 [C]

Timer 2 is used to set the intraoral setting time for the selected material. The countdown begins when the mixing process has ended. Three successive audio signals indicate the end of the intraoral setting time.

Program display [D]

The device has 99 program memory locations, allowing the timer settings to be assigned to a fixed program memory location and accessed directly.

Standby indicator [E]

After 10:00 minutes, the display switches to standby mode. The display automatically reactivates by pressing any button.

Mixing speed [F]

Unless a menu is selected, the mixing speed can at any time be increased by pressing the Plus button [8] or decreased by pressing the Minus button [9]. Three speed levels are available.

Level 1	Low mixing speed
Level 2	Medium mixing speed
Level 3	High mixing speed

Software version [G]

The version of the software currently installed on the device is visible in the lower right corner for approx. 2 seconds when the device is switched on.

Cartridge indicator [H]

The cartridge indicator shows whether a cartridge has been inserted into the device and is ready for operation. If the indicator symbol is crossed out even though a cartridge has been inserted, then the cartridge has not been inserted correctly.

Advance indicator [I]

If the advance indicator symbol is displayed, the pistons can move forward. The symbol flashes when the pistons move quickly towards the material. During the mixing process, the flashing symbol is also highlighted.

Reverse indicator [J]

If the reverse indicator symbol is displayed, the pistons can move backwards. The symbol flashes when the pistons retract quickly. When the cartridge exchange position is reached, the symbol is no longer visible.

Recommended use

(Fig. see »Key assignment« and »Handling«)

Getting started [Fig. 01]

1. Place the device on a stable surface.
2. Connect the power cord to the device [2] and insert the power plug into the power socket.
3. Then switch the device on using the toggle switch [1] located on the back of the device.

All indicator symbols on the display will flash for about 2 seconds following the device's short internal self-test.

The cartridge symbol will be crossed out if a cartridge has not been inserted or has not been inserted correctly.

If a cartridge is already inserted in the device, the fill level indicator will flash until the fill level can be determined (after advancing the pistons). For safety reasons, the device only works when a cartridge is inserted.

Using the foot switch (optional)

If the optional foot switch is to be used, plug the foot switch connector into the socket [4] on the back of the MixStar eMotion and screw into place. Pressing the foot switch mixes and dispenses the impression material until the foot is removed from the foot switch.

Preparing a cartridge [Fig. 02-06]

1. Slide the locking mechanism upwards in order to open the cartridge. Do not remove the locking mechanism from the cartridge.
2. Remove the seal from the cartridge and discard.
3. Firmly attach a new mixing tip.
4. Secure the mixing tip by sliding the locking mechanism into the locked position. After use, leave the used mixing tip on the cartridge as a seal.
5. Insert the cartridge into the device from above. A short audio signal confirms correct insertion of the cartridge. Discard approximately 3 cm of the initially ejected material of each new cartridge before preparing the first impression.

Filling the impression trays [Fig. 07-08]

Both hands remain free for filling the impression tray due to the simple two button operation or operation via the foot switch.

By pressing the Plus button [8] or the Minus button [9], the mixing speed can be increased or decreased across three levels. This is also possible during the dispensing process.

1. Start the mixing process by pressing the Start/Stop button [5] or by using the foot switch. If Timer 1 has been set, it will now start its countdown. The end of the working time period is indicated by a long audio signal. When using a new MixStar cartridge, discard the first 3 cm of the mixed material. Please observe the manufacturer's instructions for use for the material selected. After pressing the Start/Stop button [5], all the elements which have not been set are hidden from the display to provide a clearer overview.
2. Hold the impression tray at a slight angle beneath the mixing tip. To avoid air bubbles, make sure the mixing tip is immersed in the material when filling and continuously fill the tray with the impression material.

- When the impression tray is sufficiently filled, interrupt the mixing process by pressing the Start/Stop button [5] or removing your foot from the foot switch.

The mixing process immediately stops and pressure on the cartridge is automatically relieved to prevent further flow of the material. If Timer 2 has been set, it will now start its countdown. Three successive audio signals indicate when the material can be removed from the mouth.

The device automatically stops dispensing after 02:00 minutes but can be restarted by pressing the Start/Stop button [5].

Filling elastomer syringes [Fig. 09-10]

Mixing speed 1 is recommended for the filling of elastomer syringes. By pressing the Plus button [8] or the Minus button [9], the mixing speed can be increased or decreased across three levels. This is also possible during the dispensing process.

- Remove the piston from a standard elastomer syringe.
- Insert the rear opening of the elastomer syringe onto the mixing tip.
- Start the mixing process by pressing the Start/Stop button [5] or by using the foot switch.

If Timer 1 has been set, it will now start its countdown. The end of the working time period is indicated by a long audio signal.

- When the syringe is sufficiently filled, interrupt the mixing process by pressing the Start/Stop button [5] or removing your foot from the foot switch.

The mixing process immediately stops and pressure on the cartridge is automatically relieved to prevent further flow of the material.

If Timer 2 has been set, it will now start its countdown. Three successive audio signals indicate when the material can be removed from the mouth.

Changing the cartridge [Fig. 11-12]

- The cartridge can only be removed if the pistons are in the cartridge exchange position. If necessary, press the Reverse button [6] and wait until the pistons return to their original position.

The reverse indicator [J] flashes as long as the pistons are in the process of retracting. As soon as the cartridge exchange position is reached, the reverse indicator is no longer displayed.

- Using one hand, push the cartridge through the cartridge holder openings from below and remove the cartridge from above. After use, leave the used mixing tip on the cartridge as a seal.

- Insert the new cartridge into the device from above (see »Preparing the cartridge«).

In order to be able to use all the remaining material in a cartridge, have another prepared cartridge ready to go before starting the mixing process. This ensures that the remaining material does not have to be discarded due to the working time being exceeded during cartridge replacement.

Changing the mixing tip [Fig. 03]

Attach a new mixing tip before each mixing process.

- Slide the locking mechanism upwards.
- Remove the old mixing tip. If the cartridge has been stored for a longer period of time, it is important to remove any material blocking the cartridge opening.
- Then firmly align the mixing tip openings with the corresponding cartridge openings.
- Lock the mixing tip in place with the locking mechanism.
- Attach a new mixing tip before each mixing process. The device cannot operate if material has cured inside the mixing tip.

If the mixing tip is not securely attached to the cartridge, there is a risk that the material may leak out past the tip and cause contamination.

Setting the timers

The device is equipped with two timers and 99 program memory locations. The timers are used to set the individual working time [B] and intraoral setting time [C] of the material used. The maximum time that can be set is 09:00 minutes 59 seconds. An audio signal indicates when the set time expires (0:00) and the timer resets to the last set value.

The last time set is retained even if the device is switched off. Your settings must be assigned to a fixed program memory location [D] and be called up again by reselecting the corresponding program when needed. The device can be operated using both, one or none of the timers.

The device is delivered with preset programs, adapted to DMG material specifications.

- Press the Menu button [10] briefly. Select the program display, which then flashes.
- Select the program memory location where the timer settings are to be stored.

To do this, press the Plus button [8] or the Minus button [9] to move up and down the programs available. By keeping either button pressed, the programs scroll in steps of 10. By briefly pressing one of the two buttons, the programs scroll one at a time.

- The selected program is confirmed by pressing the Menu button [10] again briefly. Select Timer 1, which then flashes.

When the timer is selected, it counts to the next minute in one-second intervals when the Plus button [8] or Minus button [9] is pressed. By keeping either button pressed, counting continues in one minute intervals. After pressing one of the buttons again, the seconds can be set.

Briefly press the Menu button [10] to select Timer 2, and set as previously described.

Please refer to the manufacturer's instructions for use for the material selected to find the relevant working time and intraoral setting time. When setting Timer 2, please ensure that the transfer time you require (from end of mixing to insertion in mouth) is added to the specified intraoral setting time.

- Pressing the Menu button [10] closes the menu and the settings are saved under the set program number. A short audio signal confirms the saving process.

If the Start/Stop button [5] is pressed before saving the settings, the menu will close. Settings that have not been saved will be lost.

Preset programs

The device currently comes with twelve pre-set programs.

Program	Material	Working time	Recommended time in mouth
P1	Honigum Heavy	02:15 min	03:15 min
P2	Honigum Heavy Fast	01:15 min	02:00 min
P3	Honigum Putty	01:45 min	03:15 min
P4	Honigum Mono	02:00 min	03:15 min
P5	StatusBlue	01:15 min	01:45 min
P6	Silagum Putty Soft	02:00 min	03:30 min

Program	Material	Working time	Recommended time in mouth
P7	Honigum Pro Heavy	02:45 min	03:15 min
P8	Honigum Pro Heavy Fast	01:30 min	02:00 min
P9	Honigum Pro Putty	01:45 min	03:15 min
P10	Honigum Pro Mono	02:45 min	03:15 min
P11	Honigum Pro Mono Fast	01:30 min	02:00 min
P12	Honigum Pro Heavy Scan	02:45 min	03:15 min

These presets can be changed. When setting Timer 2, please ensure that the transfer time you require (from end of mixing to insertion in mouth) is added to the specified intraoral setting time.

If the device is reset to factory settings the preset programs are also reset to their original settings.

Selecting programs

1. Press the Menu button [10] briefly. Select the program display, which then flashes.
2. Press the Plus button [8] or the Minus button [9] to move up and down the programs available until you reach the desired memory location. By keeping either button pressed, the programs scroll in steps of 10. By briefly pressing one of the two buttons, the programs scroll one at a time.
3. Pressing and holding the Menu button [10] closes the menu and selects the corresponding program.

Operating without timers

If the timers are not being used, these can be hidden in the display for a better overview.

To do so, press the Minus button [9] to set the program display to P-- (see »Selecting programs«) and save the setting by pressing the Menu button [10].

After pressing the Start/Stop button, both timers are hidden from view. This setting will remain active even after the device has been restarted. Pressing the Menu button takes you back to the setting mode and the timer symbols are displayed again.

Restart

For a device restart, turn off the device at the main switch for at least 5 seconds.

Reset to factory settings

Press and hold down the Plus button [8] and Minus button [9] while switching on the device until the self-test is completed.

Previously stored programs will be deleted. The original preset programs (see »Preset programs«) are loaded. The mixing speed is set to level 2.

Maintenance and care

The device does not require maintenance with normal use. However, if a malfunction does occur please contact Customer Services.

With the exception of the plunger discs and fuses, the device does not contain parts that can be replaced by the user. Replacement fuses and plunger discs are supplied with the appliance.

Disinfection and cleaning

For your own safety, please unplug the device before disinfection and cleaning.

Please clean the device using only a soft cloth, and, if needed, a mild detergent. Never use solvent-based or abrasive cleaning agents as these can damage the plastic components. Do not allow any cleaning liquids to get into the device.

Use a simple wiping disinfection method to keep the device and power cable clean. Disinfectants containing alcohol, such as Pursept-A Xpress (Fa. Merz) or FD 322 (Fa. Dürr) can be used for disinfection. Pursept-A and FD 322 are not available in all countries.

Cleaning/changing the pistons

If the plunger discs are dirty or damaged, they can be cleaned or replaced as follows:

1. Ensure the pistons are in their original position and remove the cartridge.
2. Press and hold the Menu button [10]. An audio signal sounds. All symbols except for the cartridge and advance indicators disappear from the display.
An audio signal sounds and the Start/Stop button [5] can be operated. The Menu button [10] must be pressed down the entire time.
3. Advance the pistons until the plunger discs can be easily reached.
4. Cleaning: Use a soft cloth to clean the plunger discs.
Replace: Unscrew the retaining screws of the plunger discs with the hex wrench provided. Remove the plunger discs from the shafts and replace them.
5. Press and hold the Menu button [10] again and press the Reverse button [6]. The pistons return to their initial position.
Plunger discs and hex wrench are included in the »Service Kit« accessory item.

Replacing the fuses

1. Unplug the power cable.
2. Using a screwdriver, remove the fuse drawer [2] above the power connection on the back of the device and replace the fuses.

Electrical connection	115 V	230 V
Fuse [2 x type]	T 4 A, 250 V	T 2 A, 250 V

Fuses are included in the »Service Kit« accessory item.

Troubleshooting

In the event of a possible malfunction, the device shows various error codes on the display. E01 indicates possible misuse of the device. In this case, follow the outlined advice. If the malfunction cannot be remedied this way or if another error code is displayed, please contact our free service hotline 0 800 - 3 64 42 62 (Germany only) or phone +49 40 84 00 60 or send an email to info@dmg-dental.com.

Device does not start

- Check whether the power cable is securely connected to the device and plugged into the socket.
- Check fuses and replace them if necessary. Pay attention to the corresponding specifications on the rating plate.

Device starts but no impression material is delivered

- Check whether a new mixing tip is in place.
- Foil bags: Cartridge has not been filled, foil bags are not sufficiently open.

Flow of the impression material is too slow

- Ensure that the impression materials are at room temperature (min. 18 °C/64 °F).
- Foil bags: Foil bags have been inserted incorrectly.

Impression material is dispensed, mixing component doesn't rotate

- Check whether the mixing shaft engages properly with the mixing tip.
- Attach a new mixing tip if the hexagonal opening is damaged.

Damage to the foil bags

- Check plunger discs. Replace damaged plunger discs, tighten plunger discs if loose.

Device switches off by itself

- Remove cured material from the cartridge.
- Check whether a new mixing tip is in place.
- Then switch the device off at the main switch for at least 5 seconds.

Interactions

Interactions when using the MixStar-eMotion are not specified. These result from the material used for the dental treatment. Information on interactions can be found in the manufacturer's instructions for use for the material selected.

Residual risks / side effects

Residual risks and side effects when using the MixStar-eMotion are not specified. These result from the material used for the dental treatment. Information on residual risks and side effects can be found in the manufacturer's instructions for use for the material selected.

Warnings / precautions

For dental use only!

Keep out of the reach of children!

- Please read these instructions for use carefully.
- Make sure that the mains voltage indicated on the rating plate of the device corresponds to the mains voltage used.
- Only plug the device into an earthed power socket. If an extension cord is used, ensure that this is also earthed.
- Always use the device on a stable surface.
- If the device has been stored in a cold environment below +10 °C / +50 °F, please wait until the MixStar-eMotion has warmed up to room temperature before switching it on. Otherwise, the condensation that forms on cold surfaces may lead to a short circuit.
- Only use the MixStar eMotion for its intended purpose.
- Only use power cables that are expressly approved for use in your country.

Performance characteristics of the device

Device data

Housing made from high impact resistant plastic, hazard class 1, type B	
Dimensions H x W x D	370 x 180 x 430 mm
Weight	8.9 kg
Ambient temperature	+ 5 to + 40 °C / + 41 to + 104 °F

Device data

Recommended operating temperature	+ 18 to + 40 °C / + 64 to + 104 °F
Piston retraction time from foremost position	11 seconds in cartridge exchange position
Electrical connection	115 V / 230 V
Frequency	60 Hz / 50 Hz
Fuse [2 x type]	T 4A, 250 V / T 2A, 250 V
Output	200 W = 200 VA
Duty cycle	Max. 02:00 minutes
Pause time	At least 05:00 minutes
USB connection	Mini USB type B 5 pin
Foot switch connection	Socket 4 pin type MAS 4100, threaded
Foot switch rated voltage	24 V
Foot switch rated current	0.1 A

Storage / disposal

- Store at - 20 °C to + 60 °C / - 4 °F to + 140 °F and a maximum relative humidity of 80 %.
- Transport exclusively in original packaging at - 20 °C to + 60 °C / - 4 °F to + 140 °F and a maximum relative humidity of 80 %.
- Disposal must comply with national regulations.

Disposal note (only valid for Germany): DMG commissions the collection of the device and its proper disposal.

Contact: Free-of-charge service hotline **0 800 - 3 64 42 62** or via email to info@dmg-dental.com stating the disposal request.

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This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference in which case the user will be required to correct the interference at his own expense. Caution: Changes or modifications not expressly approved by the party responsible for compliance could void the user’s authority to operate the equipment.

Service Address USA:

DMG America
65 Challenger Road, Suite 340 Ridgefield Park, NJ 07660 | USA

Federal Law (USA) restricts this device to sale by or on order of a dentist or other practitioner licensed by law of the state in which he or she practices to use or order the use of this device.

DMG Chemisch-Pharmazeutische Fabrik GmbH
Elbgaustraße 248 · 22547 Hamburg · Germany · www.dmg-dental.com
Fon: + 49. (0) 40. 84 006-0
Fax: + 49. (0) 40. 84 006-222
www.dmg-dental.com

092227/2021-02