# R-Motion®

R-Motion® Glider

## EN

- Create straight-line coronal and radicular access
- Use an ISO 010 hand file to explore the canal
- Determine the working length (WL)

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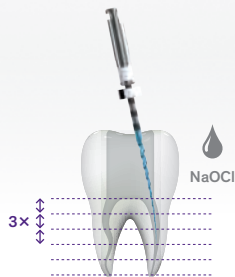
R-Motion® (shaping file)


- Progress with **R-Motion Glider** down to WL with **gentle 2-3 mm strokes** (max. 3)
- Irrigate and recapitulate

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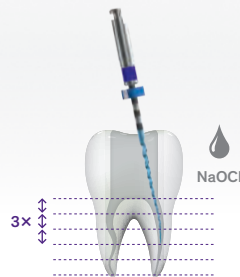- Use the **appropriate R-Motion shaping file** with **gentle 2-3 mm strokes** (max. 3), applying very light apical pressure and allowing file to passively progress along the canal until reaching the WL
- Irrigate and recapitulate

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FKG  
swiss endo

- Once the canal has been shaped, proceed with the cleaning of the canal
- Obturate by selecting the appropriate GP point and sealer

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30/.04

or/ou/oder

Select the final shaping file depending on your own therapeutic approach.

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50/.04

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Clean & Obtrurate  
Nettoyer & Obturer  
Reinigen & Füllen

\* Select the R-Motion single-file shaping instrument which is best suited to the anatomy of the canal being treated. If the radiographs do not allow you to clearly identify the anatomy of the canal, it can be gauged using hand files.

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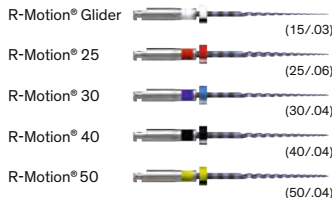
## (EN) General information

## Parameters for use:

R-Motion® instruments must only be used in **reciprocating** (counterclockwise) mode.

## Indications for use:

R-Motion® endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

The R-Motion® Glider instrument is designed for glide path of root canals in reciprocating mode.

R-Motion® 25/30/40/50 instruments are designed for shaping and cleaning root canals in reciprocating mode.

## Contraindications:

Nickel-Titanium instruments must not be used on individuals with a known allergic sensitivity to these metals.

**Adverse reactions:** None known.

## Notice and precautions:

- Take several radiographs from different angles to determine the anatomy of the root canals (length, width and curvature).
- Irrigate the canal thoroughly and frequently after the R-Motion® Glider instrument or R-Motion® single-file shaping instrument is extracted.
- If the R-Motion® Glider or an R-Motion® single-file shaping instrument is not progressing easily, remove it from the canal, clean it, inspect its cutting edges, then irrigate the canal and recapitulate with an ISO 010 hand file.
- Single use instruments must not be retreated for subsequent use. This could compromise their integrity.
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).
- When an instrument reaches the end of its life, please dispose of it in accordance with the applicable laws and regulations.

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1. Create straight-line coronal and radicular access.
2. Use an ISO 010 hand file to explore the canal.
3. Determine the working length using a radiograph and/or an apex locator.
4. Use the mechanised **R-Motion® Glider** instrument, applying gentle strokes of 2-3 mm to establish a glide path until the working length is reached. After 3 strokes, remove and clean the instrument, then irrigate; recapitulate with an ISO 010 hand file.
5. Select the R-Motion® single-file shaping instrument which is best suited to the anatomy of the canal being treated. In most cases, and depending on your approach to apical preparation, the **R-Motion® 25** or **R-Motion® 30** will be appropriate. *If the radiographs do not allow you to clearly identify the anatomy of the canal, it can be gauged using hand files.*
6. Start the shaping procedure, applying your usual irrigation/ disinfection protocol.
7. Use gentle strokes of 2-3 mm, applying very light apical pressure and allowing the R-Motion® single-file shaping instrument to progress passively along the canal. After 3 strokes, remove and clean the instrument, then irrigate; recapitulate with an ISO 010 hand file.
8. Repeat the previous step until the R-Motion® instrument has reached its working length. *If necessary, a brushing motion may be applied to remove coronal interferences and/or ensure uniform shaping of irregular canals.* Once the working length has been reached, remove the instrument to prevent over-enlargement of the foramen.

Once all the canals have been shaped, proceed with the cleaning of the canals. *For an optimal result, the use of XP-endo® Finisher is recommended.*

Once the cleaning of the canals is completed, proceed to the next phase of treatment:

9. Obturate with gutta-percha and sealer. *TotalFill® BC Points™ and TotalFill® BC Sealer™ are recommended.*

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