Activator universal Plus

Optosil®/Xantopren®

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GB Instructions for use

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SE Bruksanvisning

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NO Bruksanvisning

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Stand: 2013-07

Technical Information

Silicon based precision condensation curing impression material.

Instructions for use **GB**

Optosil®/ Xantopren®	EN ISO 4823	Colours	BASE	Dosage**			Mixing-time	Working-time**	Time in mouth
				Activator universal Plus					
				Liquid	Paste	s			
Optosil® Comfort®	Typ 0	yellow	1 spoon (10,7 g/9,5 ml)	6-8 drops (0,14 g/0,13 ml)	6-8 graduations (0,32 g/0,29 ml)	30	1:00	4:00	
Optosil® P Plus	Typ 0	yellow	1 spoon (14,4 g/9,5 ml)	4-6 drops (0,09 g/0,09 ml)	4-6 graduations (0,21 g/0,19 ml)	30	1:00	4:00	
Xantopren® H green	Typ 1	green	6 graduations (11,0 g/8,8 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:00	4:00	
Xantopren® M mucosa	Typ 2	orange	6 graduations (9,0 g/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:30	4:00	
Xantopren® L ⁺ blue	Typ 3	blue	12 graduations* (8,6 g/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:30	4:00	
Xantopren® VL ⁺ Plus	Typ 3	blue	12 graduations* (8,5 g/7,6 ml)	12 drops (0,27 g/0,26 ml)	12 graduations (0,65 g/0,59 ml)	30	1:30	4:00	

* At room temperature of 23°C (73°F), 50% rel. humidity.
Higher temperatures shorten, lower temperatures prolong these times.
* 1 graduation = 1,3 cm (applies to mixing pad available)
* 2 Bottom ring on mixing dish is equivalent to 7,6 ml
* 4 When using the dispenser: 1 stroke (4,5 g / 4,0 ml) + 6 drops (0,14 g / 0,13 ml)
The stated dosage should be observed

Typ 0 = putty consistency
Typ 1 = heavy bodied consistency
Typ 2 = medium bodied consistency
Typ 3 = light bodied consistency

User Advice

The **Optosil®/Xantopren® assortment** is a complete system. Components can be individually combined for all impression techniques. For best results usage with other silicon materials and Activators is not recommended. Visually inspect for damage before first use. Damaged products must not be used. Close carefully after use.

Pouring the model: Pouring can take place 30 minutes after the impression is removed from the mouth. Impressions remain dimensional stable up to 7 days.

Recommended model material: All commercially available modeling stones can be used. Our suggestion: Class III Gips, MOLDANO®/MOLDADUR® Class IV Gips, MOLDASTONE®/MOLDASYN™ Impressions may be copper or silver plated in the usual way.

Electroplating: Impressions should generally be treated by immersion for 10 minutes* in 5,25 % Sodium Hypochlorite solution. Other silicon specific disinfectant solutions should be used as directed by the products manufacturer.

Disinfection: * Gelson Luis Adabo, DDS, PhD, Elaine Zanarotti, Renata Garcia Fonseca, and Carlos Alberto dos Santos Cruz, DDS, PhD Effect of disinfectant agents on dimensional stability of elastomeric impression materials. J Prosthet Dent, 81, 621-4, 1999

Adhesion: We always recommend the use of our Universal Adhesive. Do not store above 25°C (77°F).

Warning and precaution for Activator universal Plus: Flammable. Inhalation can cause health problems. Eyes, respiratory organs and skin can be irritated. In an emergency, rinse off immediately with copious amounts of water and consult an eye specialist. Avoid skin contact. Wear protective clothing. **Wear suitable gloves (EN 374, category III, e.g. Dermatri from KCL, Germany)**. Do not swallow and ingest. If health problems arise, immediately seek medical attention immediately. Intestinal blockage may arise in rare cases. Hypersensitive persons may experience an allergic reaction to the product. Not recommended for breast feeding patients and children under 6 years. For use by dental personnel only.

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