
B1000 Pulppen



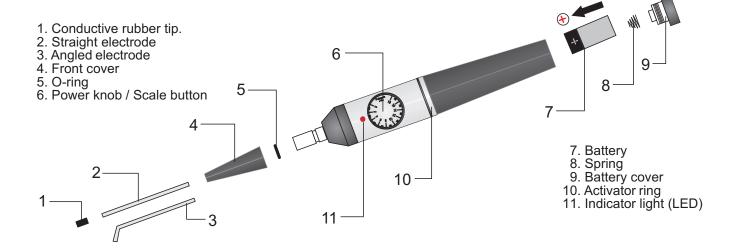



Introduction

The PULPPEN B1000 is a highly accurate instrument destined for daily clinical use.

Denlux has successfully incorporated the latest available technology in the PULPPEN, to provide you with a compact, lightweight, hand held instrument, capable of providing accurate, scientific readings.

After carefully reading the 'User's Manual', you may want to refer to the 'Technical Manual' for additional information on the theory of operation defining the function of the PULPPEN and other types of pulp vitality testers.



In the following please observe carefully the sections marked with the sign shown here. These sections contain important information.

WARNING!!



NOT to be used by or on persons using a pacemaker (concerns user and patient)

Instructions for use

Before first using the PULPPEN

Unscrew the battery cover (9) at the rear end of the PULPPEN. Check that the battery (type TR175 7V, Denlux part no. 15101) is provided and insert the positive end (+) first. Hereafter remount the battery cover.

Next, insert a straight or angled electrode (2 or 3). Push the electrode into place until it reaches the bottom. Make sure a conductive rubber tip (1) is placed on the tip of the electrode.

The PULPPEN is now ready for use!

Using the PULPPEN

To switch ON the PULPPEN B1000 pick up the instrument so that your hand creates a connection between the activator ring and the metal body surrounding the power knob.

The red indicator light (LED) (11) will start flashing indicating that the instrument is ready for use. The PULPPEN will stay on for as long as you maintain this grip. When you remove your hand from the instrument it will switch itself OFF automatically. Make sure to reset (to zero) the power knob before taking another reading.

Before taking a reading, dry the tooth that is to be diagnosed. The tooth should always be kept dry to avoid leakage current which could give false readings!


A circuit must be established from the PULPEN's tip, through the patient, your hand, and back to the metal body of the PULPPEN. There must be a contact between your hand and the metal body of the PULPPEN at all times. Likewise, you must make sure that there is a contact between your other hand and the patient in order to complete the circuit. To do so, place a moistened finger on the patient's lower lip

If wearing latex examination gloves, an electrical circuit must be established between the patient and the PULPPEN.

This can be done by using the Contact-Kit (Extra accessory - shown below; Denlux part Number 15110): one end, a metal electrode (handle), is given to the patient to hold; the other, a metal ring, is fixed to the metal body of the PULPPEN.

Contact-Kit for PULPPEN

(Extra accessory)

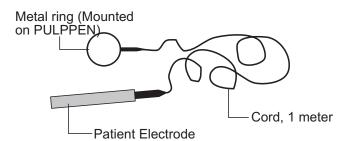

Place the PULPPEN tip on to the dried target tooth's surface. While maintaining contact with the tooth, using your index finger, turn the power knob / scale button. The PULPPEN will emit electronic pulses through the tooth. The power (amplitude) of the emission will increase as you turn the knob clockwise.

As soon as the patient feels a tingling sensation from the target tooth, and reacts according to an agreed hand signal, remove the PULPPEN's tip from the tooth and read the response level on the power knob / scale button.

Always reset the power knob / scale button to zero before taking a new reading.

Response levels B1000

Normal tooth sensitivity levels are (on scale button):

Incisors **Bicuspids** Molars



Response Levels

Response levels outside these ranges do not systematically indicate an abnormal condition. Individual cases can vary greatly

If no reaction from the patient is observed additional examination will be necessary. A perfectly healthy tooth might due to moist on its surface, alloy fillings or calcification give a false negative response.

On the other hand leak currents may cause the patient to feel a stimulus even though the nerve tissue is dead - in this case the stimulus is felt in a neighbouring tooth.

In case of no response

If the patient do not react to the stimuli provided by the PULPPEN the following should be thoroughly checked:

- That the red indicator light (LED) (11) is flashing, indicating correct function of the PULPPEN. If not, there is no contact established between the activator ring and metal body **OR** the battery power is insufficient to allow measurement.
- That there is a correctly established connection between tooth and PULPPEN as well as between dentist and patient.

If the above matters are checked and possible reasons for malfunction are removed, a new reading can be taken.



Battery power level

The indicator light (LED) functions as a power indicator. In case of no flashing during operation, the battery must be changed. Normal battery life span is 1 - 2 years by regular use.

Always use the same type of battery (Denlux part number 15101) as supplied with the PULPPEN. Use of other types of batteries can cause malfunction and/or incorrect readings.



Servicing / Repair

If a malfunction occurs, the PULPPEN should be returned to the distributor for repair by a qualified service engineer.

DO NOT attempt to repair the PULPPEN yourself!



A Hygiene

The PULPPEN hand piece may be disinfected using disinfectant wipes or sprays. Electrodes should be steam autoclaved after each use.

Conductive rubber electrode tips should be disposed of after use and a new one used for each patient.



Storing

DO NOT place the PULPPEN on metal surfaces or surfaces containing carbon. The PULPPEN could switch itself on resulting in loss of battery power.

Always remove the battery if the PULPPEN is not to be used for extended periods of time; this improves battery longevity substantially

