

Smart Solutions from GC

for all your luting challenges

Let's Join Forces!



How to choose the optimal luting cement?

Today more than ever you are spoiled for choice when planning and delivering indirect restoration. From the moment your patient comes in, you have much to consider before deciding on the treatment plan.

As a dentist you know that no two patients are alike and it is therefore important to follow a holistic approach in treatment planning considering the choice of indirect restoration and type of preparation which is best suited to the patient's clinical situation.

An important part of this process is choosing the right luting cement



The key questions you should ask before making the final choice are:
Which is the optimal luting cement to help me

Protect
the tooth
structure?

Protect
the prosthetic
restoration?

The selection of the optimal luting cement is the guarantee for a beautiful smile and long-term patient satisfaction



Discover our High 5 solutions for all your luting challenges

With more than 95 years of experience and one of the most comprehensive product portfolios, we at GC believe we are the ideal partner to guide you towards the most appropriate luting solution.

Because after all, efficient luting is a team effort.

Each of the GC luting solution is suited to different clinical situations and offers you their unique features like physical properties, working time, protocol of use and cost...

- G-CEM ONE
- G-CEM Capsule

Adhesive resin cement

- G-CEM LinkForce (dual-cure)
- G-CEM Veneer (light-cure)

Universal self-adhesive resin cement

**HIGH
5
GC LUTING
SOLUTIONS**

Resin-modified glass ionomer (RMGI) cement

- GC Fuji PLUS
- GC FujiCEM 2
- GC FujiCEM Evolve

Zinc oxide cement

Conventional glass ionomer (GI) cement

- GC Fuji I
- GC Fuji TEMP LT

Let's Join Forces!

Download the **GC Luting Guide**



• FREEGENOL
TEMPORARY PACK



Depending on the clinical situation, the first question you should ask yourself is: Should I CEMENT or BOND?

When to cement?

Cementation is typically used when your preparations allow mechanical retention.

The following clinical aspects will define your choice for the optimal CEMENTATION procedure:

Retentive Preparation



Select **GI** & **RMGI** cement that offers the best balance of ease of use, speed and tooth protection.

Isolation not possible



Select **GI** or **RMGI** cement as they offer moisture tolerance with excellent chemical adhesion to the tooth.

Moderate aesthetics needed



Select **GI** or **RMGI** cement when your clinical situation and/or selected substrate (zirconia, metal & porcelain fused to metal) does not require a very high aesthetic cement.

Patients with special needs



Select **GI** or **RMGI** cement for high caries risk and geriatric patients.

Your options

Conventional glass ionomer (GI) cement | GC's solution: **GC Fuji I**
Resin-modified glass ionomer (RMGI) cement | GC's solutions: **GC Fuji PLUS**,
GC FujiCEM 2 and **GC FujiCEM Evolve**.

Source: Dr Lucile Dahan, France (1) and Prof Jean-François Lasserre, France (2, 3 and 4)

"Most of the time the margins of the preparation are subgingival and total control of humidity proves to be difficult!"

DR LUCILE DAHAN, FRANCE

When to bond?

Bonding is typically used when extra adhesion is required or when aesthetics are of the utmost importance.

Preparation is not retentive



Select **adhesive resin cement** or **universal self-adhesive cement with tooth primer** when the preparation is not retentive.

Self-adhesive cement alone (ease of use) can be used for moderate retentive preparations.

Isolation possible



Select **self-adhesive** or **adhesive resin cement** when a dry working field can be assured.

High aesthetic needed



Select **self-adhesive** or **adhesive resin cement** for highly aesthetic outcomes.

Extra adhesion needed



Select **adhesive resin cement** or **universal self-adhesive cement with tooth primer** or **adhesive** for bonding veneers and hybrid ceramics (e.g. CERASMART270).

Your options

Self-adhesive resin cement | GC's solutions: **G-CEM ONE** and **G-CEM Capsule**

Universal self-adhesive resin cement with tooth primer | GC's solution: **G-CEM ONE** with **G-CEM ONE Adhesive Enhancing Primer** or **G-CEM ONE** with **G-Premio BOND**

Adhesive resin cement | GC's solutions: **G-CEM LinkForce** (dual-cure) and **G-CEM Veneer** (light-cure)

Source: Prof Jean-François Lasserre, France (1, 2 and 3)

Discover all of our luting solutions!



With one of the most comprehensive luting portfolios, we at GC can help you maintain your patient's smile.



Our solutions for permanent cementation

Glass ionomer based cements	Composite resin based cements
<p>GC Fuji™ I</p>  <p>Conventional glass ionomer (GI) cement</p> <ul style="list-style-type: none"> • Cost-effective solution • Resin-free cement <p>Page 7</p>	<p>G-CEM ONE™ & G-CEM™ CAPSULE</p>  <p>Universal self-adhesive resin cement</p> <ul style="list-style-type: none"> • Efficient self-cure mode • Easy-to-use <p>Page 10</p>
<p>GC Fuji PLUS®</p>  <p>Resin-modified glass ionomer (RMGI) cement</p> <ul style="list-style-type: none"> • Long-term clinical success • Worry-free cement <p>Page 8</p>	<p>G-CEM LinkForce™ (dual-cure)</p>  <p>Adhesive resin cement</p> <ul style="list-style-type: none"> • High adhesion • High aesthetics <p>Page 11</p>
<p>GC FujiCEM™ 2 GC FujiCEM™ Evolve</p>  <p>Resin-modified glass ionomer (RMGI) cement</p> <ul style="list-style-type: none"> • Easy-to-use • Versatile • High bond to zirconia <p>Page 9</p>	<p>G-CEM® Veneer (light-cure)</p>  <p>Adhesive resin cement for restoration lower than 2 mm thick</p> <ul style="list-style-type: none"> • High aesthetics • Easy placement <p>Page 12</p>

Our solutions for temporary cementation

Zinc oxide cement	Conventional glass ionomer cement
<p>FREEGENOL TEMPORARY PACK</p>  <p>Page 13</p>	<p>GC Fuji TEMP LT®</p>  <p>Page 13</p>

GC Fuji I

Conventional Glass Ionomer cement

The biocompatible, resin-free luting solution with 25 years of great clinical success.

GC Fuji I, GC's first luting GI cement, is widely used across the world. Discover why GC Fuji I is still a material of choice.



Cost-efficient solution



Resin-free cement for patients allergic to methacrylate (resin)



Good solution for cementation on implant abutments. Easy removal of excesses considerably reduces the risk of peri-implantitis.

Source: Dr Emmanuel d'Incau, France

Key clinical advantages

- **Chemical adhesion to tooth** structure, ensuring an excellent **marginal seal**
- **Moisture-tolerant solution** when a dry field cannot be assured
- **Quick and easy** without the need for etching and bonding steps of the tooth
- **Resin-free cement** for patients allergic to methacrylate (resin)
- Virtually **no post-operative sensitivity**: a pulp friendly solution

Practical advantages

- **Cost efficient** and **easy-to-use** solution for metal-based-restorations
- Economic consideration: €€€€€€

Shade range

- Powder/Liquid: Light yellow
- Capsules: Light yellow

Clinical applications

	Metal / PFM	Zirconia / Alumina
Inlays / onlays	●	N.A.
Crowns / bridges	●	●
Posts/ inlay cores	●	-

"GC Fuji I is perfectly indicated for the permanent cementation of fixed elements on implant abutments. The removal of excess is simple and easy to control, thereby reducing the risk of peri-implantitis."

DR EMMANUEL D'INCAU, FRANCE

GC Fuji PLUS

Resin-Modified Glass Ionomer cement



Long-term clinical success



Perfect balance between protection of the tooth and adhesion



Easy removal of excess



Predictable and worry-free solution

Efficient and proven over 20 years - a predictable, worry-free solution.

GC Fuji PLUS is one of the most well-known products and it is considered a benchmark material in its class, offering the tooth protection of a glass ionomer cement balanced with the strength and aesthetics of a resin cement.

Key clinical advantages

- **Chemical adhesion to tooth** structure, ensuring an excellent marginal seal
- **Moisture-tolerant solution** when a dry field cannot be assured
- **Quick and easy:** no need for etching and bonding of the tooth
- Rubbery consistency for **easy removal of excess**
- Virtually **no post-operative sensitivity:** a pulp friendly solution

Practical advantages

- **Proven and predictable performance** for most routine cases
- Available in different shades
- Also available in extended working time (EWT) version for long-span bridges
- Economic consideration: €€€€€€

Shade range

- Powder/Liquid:
A3, Yellow, Translucent
- Capsules: A3, Yellow

Clinical applications

	Metal / PFM	Feldspathic / leucite ceramics* (e.g. GC Initial LRF)	Lithium Disilicate (e.g. GC Initial LiSi Press)	Zirconia / Alumina	Composite / Fibre-reinforced resin
Inlays / onlays	●	● (inlays)	●	N.A.	●
Crowns / bridges	●	-	●	●	●
Posts/ inlay cores	●	N.A.	N.A.	●	●

“Our office has been using GC Fuji Plus for almost 2 decades. I don’t remember any failure yet and we have cemented more than 5000 crowns & bridges!”

DR HAN HEESEN, NETHERLANDS

* Feldspathic & leucite-reinforced ceramics

GC FujiCEM 2 / GC FujiCEM Evolve

Resin-Modified Glass Ionomer cement

Simplicity is innovation!

The resin-modified glass ionomer cement in paste-paste delivery form. This easy-to-use and versatile luting cement is your perfect partner for most of your routine cementation.



Handmix

Automix



Easy dispensing

Easy removal of excess of GC FujiCEM Evolve

Source: Dr J. Tapia Guadix

Key clinical advantages

- **Chemical adhesion to tooth** structure, ensuring an excellent **marginal seal**
- **Moisture-tolerant:** a “must-have” when moisture control cannot be guaranteed
- **Quick and easy:** no need for etching and bonding of the tooth
- Rubbery consistency for **easy removal of excess, with tack-cure option for GC FujiCEM Evolve**
- Virtually **no post-operative sensitivity:** a pulp friendly solution
- Strong bonding to restorative, mainly Zirconia for GC FujiCEM Evolve

Practical advantages

- Choose from handmix or automix: the choice is yours!
- Economic consideration: €€€€€€

Shade range

- GC FujiCEM 2: Light yellow
- GC FujiCEM Evolve: Universal

Clinical applications

	Metal / PFM	Feldspathic / leucite ceramics* (e.g. GC Initial LRF)	Lithium Disilicate (e.g. GC Initial LiSi Press)	Zirconia / Alumina	Composite / Fibre-reinforced resin
Inlays / onlays	●	● (inlays)	●	N.A.	●
Crowns / bridges	●	-	●	●	●
Posts/ inlay cores	●	N.A.	N.A.	●	●

“GC FujiCEM Evolve is a very user-friendly luting agent, combining ease of use, moisture tolerance and versatility. It is ideal to cement both zirconia and metal-ceramic restorations, particularly in the presence of iuxta- and sub-gingival margins. Furthermore, cement excess removal is extremely easy and safe and field isolation is not mandatory. A smart material to make cementation procedures easier and quicker.”

PROF ROBERTO SORRENTINO, ITALY

* Feldspathic & leucite-reinforced ceramics

G-CEM ONE

Universal self-adhesive resin cement



Flexible: Optional fully-adhesive protocol provides optimal bond strength in retentive and non-retentive indications.



Easy excess removal with the option of tack curing

Long-lasting aesthetic results

Source: Dr Kazunori Otani, Japan

Looking for a solution to simplify your daily luting procedures, and even more?

Choose G-CEM ONE, a dual cure universal self-adhesive resin cement with high adhesive bond strength for daily procedures, which can also be used in challenging and non-retentive situations when applying the optional tooth primer or adhesive.

Key clinical advantages

- **Reliable:** efficient chemical polymerization even under opaque or thick restorations
- **Flexible:** increased tooth adhesion thanks to the Touch Cure technology of the Adhesive Enhancing Primer (AEP) or the use of the universal bonding agent G-Premio BOND
- **Aesthetic:** excellent wear resistance for long lasting invisible margins

Practical advantages

- **Universal:** reliable performance for any type of restoration, including non retentive ones (when using the Adhesive Enhancing Primer or G-Premio BOND)
- **Technique insensitive:** moisture tolerance, ideal viscosity, easy excess removal
- **Simple:** standardized protocol with very easy handling
- **Economic consideration:** €€€€€€

Shade range

- A2, Translucent, AO3, White Opaque

Clinical applications

	Metal / PFM	Feldspathic / leucite ceramics* (e.g. GC Initial LRF)	Lithium Disilicate (e.g. GC Initial LiSi Press)	Zirconia / Alumina	Composite / Fibre-reinforced resin	Hybrid ceramics (e.g. CERASMART270)
Inlays / onlays	●	●	●	●	●	●
Crowns / bridges	●	●	●	●	●	●
Veneers	N.A.	●	●	●	●	●
Posts/ inlay-cores	●	N.A.	N.A.	●	●	N.A.


“Excesses of G-CEM ONE are super easy to remove and the system is easy to use. We also like the flexibility to decide whether you need the primer or not and the assurance that everything sets well even in the self-cure mode. It’s a great product!”

DR MELIKE AND DR DAVID BERGFORT, GERMANY

* Feldspathic & leucite-reinforced ceramics

G-CEM LinkForce

Dual-cure adhesive resin cement

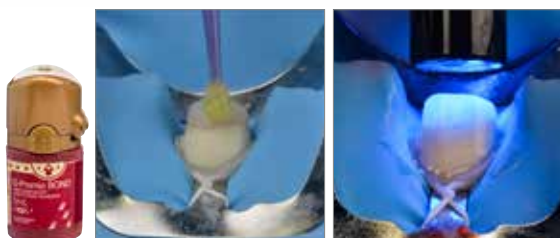

Easy application of G-Premio BOND with or without prior etching (3µm) for optimal adhesion



One universal primer for stable Easy seating and perfect adaptation adhesion to all substrates thanks to a very low film thickness

Source: Dr Antonio Saiz-Pardo, Spain

Strength and aesthetics in one system for all indications, all substrates.

G-CEM LinkForce is the universal adhesive resin cement that is ideal to use whenever additional retention is needed and a must-have for all CAD/CAM ceramic and hybrid ceramic blocks such as CERASMART270.

Key clinical advantages

- Secure adhesion in all situations with **only one system, three base elements**:
 - **G-Premio BOND** bonds to ALL Preparations with no compromises
 - **G-Multi Primer** ensures a stable adhesion to ALL Restorations
 - **G-CEM LinkForce** provides a strong link in ALL Indications
- Optimal light-curing feature for veneer indications
- **Efficient self-cure mode**, useful for luting more opaque or thick restorations

Practical advantages

- Optional use of G-Premio BOND Dual-Cure Activator for post luting
- The universal, aesthetic and powerful solution to all your adhesive cementation challenges
- Choice of **four aesthetic shades** and their corresponding **try-in pastes**
- Economic consideration: €€€€€€

Shade range

- A2, Translucent, Bleach, Opaque

Clinical applications

	Metal / PFM	Feldspathic / leucite ceramics* (e.g. GC Initial LRF)	Lithium Disilicate (e.g. GC Initial LiSi Press)	Zirconia / Alumina	Composite / Fibre-reinforced resin	Hybrid ceramics (e.g. CERASMART270)
Inlays / onlays	●	●	●	●	●	●
Crowns / bridges	●	●	●	●	●	●
Veneers	N.A.	●	●	●	●	●
Posts/ inlay-cores	●	N.A.	N.A.	●	●	N.A.


"I use G-CEM LinkForce as a first choice in all clinical indications. Day after day it still surprises me."

DR CASTI, ITALY

* Feldspathic & leucite-reinforced ceramics

G-CEM Veneer

Light-cure adhesive resin cement

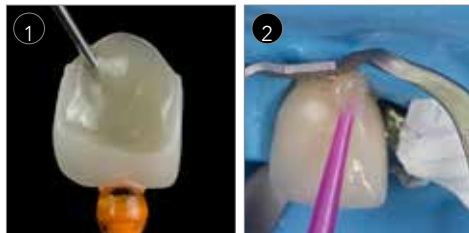

Easy dispensing

Thixotropic consistency: easy removal of excess



Light-curing composite: high aesthetics over time

A versatile resin cement for easy luting of restorations with 2mm thick.

G-CEM Veneer: a light-cure resin cement for high aesthetic demand with easy placement featuring Full-coverage Silane Coating (FSC) technology.

Key clinical advantages

- **Thixotropic consistency** for easy placement of veneers and easy excess removal.
- Superior performance thanks to **high filler rate** of 69% (w/w): high wear resistance, high bond and flexural strength for long-term aesthetics.
- The alternative to heated composites with less risk of failures for thin indirect restorations.

Practical advantages

- Easy to use and universal system with G-Premio BOND and G-Multi PRIMER.
- 4 aesthetic shades matching all cases, with G-CEM Try-in pastes.
- Ergonomic syringe.
- Economic consideration: €€€€€

Shade range

- A2, Translucent, Bleach, Opaque

Clinical applications

	Glass-ceramics* (e.g. GC Initial LRF, GC Initial LiSi Press)	Hybrid ceramics (e.g. CERASMART270)	Composite (e.g. GRADIA PLUS)
Veneers	●	●	●
Inlays / onlays	● (< 2.0 mm thickness)	● (< 2.0 mm thickness)	● (< 2.0 mm thickness)

“To me, G-CEM Veneer kit seems to be particularly suited for the modern luting challenges linked to minimally invasive ceramic restorations. It demonstrates a good flowability, while keeping a high viscosity in order to maintain good marginal sealing ability before polymerisation.”

PROF JEAN-FRANCOIS LASSERRE, FRANCE

* Feldspathic, leucite-reinforced ceramics and lithium disilicate

GC's temporary cements...

FREEGENOL TEMPORARY PACK

Zinc oxide cement



Key features

- All-purpose temporary solution
- Adjustable consistency and setting times
- Easy removal of excess
- Eugenol free, avoid interaction with resin cements

GC Fuji TEMP LT

Conventional glass ionomer cement in Paste-pak cartridge



Key features

- Assured retention and easy removal
- Easy seating thanks to low film thickness (6µm)
- Rubbery consistency for easy removal of excess
- Radiopaque
- Ideal for cementation on implant abutments

Also discover GC's Primers...

Whether you prefer a specialised or a universal primer, GC has a solution.

Universal solution G-Multi PRIMER

One universal primer for **ALL** your indirect restorations.



Specialised solutions CERAMIC PRIMER II

A specialised primer for all **aesthetic indirect restorations**. Compatible with glass-ceramic, hybrid ceramic, zirconia, alumina and composite restorations.



METAL PRIMER Z











The hero for all your **metal and porcelain fused to metal restorations**.



GC Luting navigation chart.
Find the right cement(s) to master your luting challenges!

		Glass Ionomers			Resins			
		Self-cure			Dual Cure		Light-Cure	
		Conventional	Resin Modified		Self-adhesive	Universal	Adhesive	Adhesive
		GC Fuji I	GC Fuji PLUS	GC FujiCEM Evolve	G-CEM ONE	G-CEM ONE + AEP or G-Premio BOND	G-CEM Link Force	G-CEM Veneer
Crowns & bridges	Metal	●						
	Zirconia		●	●				
	Lithium disilicate (e.g. Initial LiSi Press)				●	●	●	—
	Composite (e.g. GC GRADIA PLUS)							
	Feldspathic & Leucite reinforced ceramics (e.g. Initial LRF)	—						
	Hybrid ceramics (e.g. CERASMART270)		—	—	—			
Inlays & onlays	Metal	●						—
	Zirconia		●	●				—
	Lithium disilicate (e.g. Initial LiSi Press)				●	●	●	●*1
	Composite (e.g. GC GRADIA PLUS)	—						
	Feldspathic & Leucite reinforced ceramics (e.g. Initial LRF)		inlays	inlays				●*1
	Hybrid ceramics (e.g. CERASMART270)		—	—	—			
Veneers	Lithium Disilicate (e.g. Initial LiSi Press)							
	Composite	—	—	—	—	●	●	●*1
	Feldspathic & Leucite reinforced ceramics (e.g. Initial LRF)	—	—	—	—	●	●	●*1
	Hybrid Ceramics (e.g. CERASMART270)							
Posts & inlay cores	Metal	●						
	Zirconia	—	●	●	●	●	●	—
	Fibre reinforced	—						

Discover more features about GC luting cements

	Conventional glass ionomer		Resin-modified glass ionomer				Self-adhesive resin		Adhesive resin	
	GC Fuji I 	GC Fuji I 	GC Fuji PLUS 	GC Fuji PLUS EWT*2 	GC Fuji PLUS 	GC FujiCEM 2 GC FujiCEM Evolve 	G-CEM 	G-CEM ONE 	G-CEM LinkForce 	G-CEM Veneer 
Shades	Light yellow	Light yellow	Yellow, Translucent & A3	A3	Yellow & A3	Light yellow	A2, Translucent, AO3 & BO1	A2, Translucent, AO3 & White Opaque	A2, Translucent, Opaque & Bleach	A2, Translucent, Opaque & Bleach
Mixing time	20 sec	10 sec	20 sec	20 sec	10 sec	20 sec **	10 sec	N.A.	N.A.	N.A.
Working time	2 min	2 min 15	2 min 30	3 min 30	2 min	2 min 15	2 min 30	2 min 40 2 min 10 (with AEP)	3 min	N.A.
Final finishing time (commencing time)	4 min 30	4 min 30	4 min 30	5 min 45	4 min 15	4 min 30	4 min	4 min	4 min	N.A.

Notes: *1 For indirect restorations with thin thickness (≤ 2mm) that enable the use of a purely light-curing technique. *2: EWT: Extended Working Time: perfect for the cementation of bridges.


** if handmix

Let's Join Forces! Discover our key luting solutions for all your luting challenges

GC FujiCEM 2, GC FujiCEM Evolve & GC Fuji PLUS

Resin-modified glass ionomer cement

Proven to be a benchmark in their class. They also are ideal worry-free solutions for your routine procedures, especially for metal, porcelain fused to metal & zirconia. They are your 'must-haves', especially when moisture control becomes a challenge!

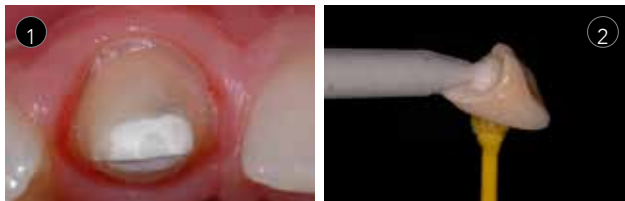

- 1 | Perfect solution for subgingival margins!
- 2 | Easy dispensing
- 3 | Easy removal of excess
- 4 | Full protection for the tooth




Source: Prof Jean-François Lasserre (1), Dr Frédéric Raux (2 and 4) & Dr Emmanuel d'Incau (3), France

G-CEM ONE

Universal self-adhesive resin cement

Simple solution for all luting indications with predictable results. Even non-retentive indications can be covered thanks to the optional tooth primer with Touch Cure technology or universal adhesive G-Premio BOND. The excellent self-curing ability provides perfect results even with thick or opaque restorations.

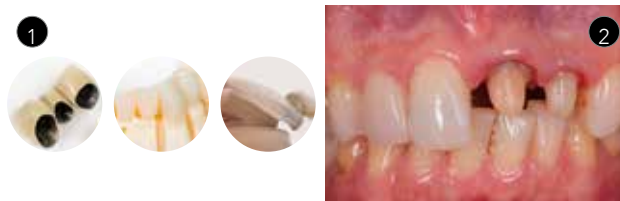

- 1 | can be used for retentive preparation alone and for non retentive preparations in combination with G-CEM ONE AEP or G-Premio BOND
- 2 | Technique insensitive & moisture tolerant
- 3 | Easy handling and excess removal
- 4 | Aesthetic results in time



Source: Dr Kazunori Otani, Japan

G-CEM LinkForce & G-CEM Veneer Adhesive resin cement





When uncompromised adhesion is needed, this product is THE solution. This universal adhesive resin cement allows you to work in a standardised way and offers a strong bond in all indications.









- 1 | Easy application of G-Premio BOND
- 2 | Light-curing of the thin bonding layer for optimal adhesion
- 3 | Universal Primer for stable adhesion to all substrates
- 4 | Easy seating and perfect adaption



Source: Dr Antonio Saiz-Pardo, Spain

GC Fuji I	
	000136 Intro Package: 1x Powder Light Yellow (35 g) & 1x Liquid (20 mL)
	000134 Refill Powder Light Yellow (35 g)
	000135 Refill Liquid (20 mL)
	000240 Refill Capsules Light Yellow (x 50)
GC Fuji PLUS	
	001409 Intro Package: Full Set A3 (15 g Powder A3, 7 mL Liquid & 6.5 mL Conditioner) Refill-EWT (extended working time version) Powder A3 (15 g)
	000231 Refill Powder Yellow (15 g)
	000219 Refill Powder Translucent (15 g)
	001410 Refill Powder A3 (15 g)
	001411 Refill Powder A3 (15 g)
	000220 Refill Liquid (7 mL)
	000222 Refill Capsules Yellow (x50)
	001418 Refill Capsules A3 (x50)
GC FujiCEM 2	
	900898 Automix Package: 2x Paste Pak cartridges (13.3 g) & 44x mixing tips
	900897 Refill 2x Paste Pak cartridge (13.3 g / 7.2 mL)
	900902 Refill Mixing Tips SL (x20)
	900903 Refill Mixing Tips SL for Endo (x10)
	001573 GC Paste Pak Dispenser
GC FujiCEM Evolve	
	012948 Single Pack: 1x syringe (9.2 g / 5.0 mL), 15x GC Push and Click Tip Regular
	012949 Triple Pack Handmix: 3x syringe (9.2 g / 5.0 mL), 1x Mixing Pad #2
	012950 Triple Pack Automix: 3x syringe (9.2 g / 5.0 mL), 45x GC Push and Click Tip Regular
	012954 GC Push and Click Tip Regular (x15)
	012955 GC Push and Click Tip Endo (x15)
G-CEM	
	002588 50 Capsules Assorted 20x A2, 20x Translucent, 5x A03, 5x BO1
	002584 Refill Capsules A2 (x50)
	002585 Refill Capsules AO3 (x50)
	002586 Refill Capsules Translucent (x50)
	002587 Refill Capsules BO1 (x50)
	004414 GC Capsule elongation tips (x50)
G-CEM ONE	
	013663 System Kit: 1x syringe A2 (4.6 g / 2.7 mL), 1x syringe TR (4.6 g / 2.7 mL) 1x Adhesive Enhancing Primer (4 mL), 1x G-Multi PRIMER (5 mL), accessories
	013661 Starter Kit A2: 1x syringe (4.6 g), 1x Adhesive Enhancing Primer (2 mL), accessories
	013662 Starter Kit Translucent: 1x syringe (4.6 g), 1x Adhesive Enhancing Primer (2 mL), accessories
	014543 Universal Starter Kit A2: 1x G-CEM ONE Syr.(4.6g), 8x GC Automix Tips Regular, 2x GC Automix Endo tips, 1x G-Premio BOND 5mL
	014544 Universal Starter Kit Translucent: 1x G-CEM ONE Syr.(4.6g), 8x GC Automix Tips Regular, 2x GC Automix Endo tips, 1x G-Premio BOND 5mL
	014545 Universal System Kit: 1x G-CEM ONE Syr. A2 (4.6g), 1x G-CEM ONE Syr. TR (4.6g), 15x GC Automix Tips Regular, 5x GC Automix endo tips, 1x G-Premio BOND 5mL, 1x G-Multi PRIMER 5mL
	013664 Twin Refill A2, 2x syringes (4.6 g / 2.7 mL)
	013665 Twin Refill Translucent, 2x syringes (4.6 g / 2.7 mL)
	013666 Single Refill AO3, 1x syringe (4.6 g / 2.7 mL)
	013667 Single Refill White Opaque, 1x syringe (4.6 g / 2.7 mL)
	013668 Value Refill A2, 8x syringe (4.6 g / 2.7 mL)
	013669 Value Refill Translucent, 8x syringe (4.6 g / 2.7 mL)
	013640 G-CEM ONE Adhesive Enhancing Primer (4 mL) bottle refill

G-CEM LinkForce	
	009540 System Kit: 1x syringe A2 (8.7 g / 5 mL), 1x syringe Translucent (8.7 g / 5 mL), G-Premio BOND (5 mL), G-Premio BOND DCA (3 mL), G-Multi PRIMER (5 mL), G-CEM Try-in Pastes (A2 & Translucent) (1.2 mL / 1.5 g), GC Etchant & accessories
	009542 Starter Kit A2: 1x syringe A2 (8.7 g / 5 mL), G-Premio BOND (5 mL), G-Multi PRIMER (5 mL) & accessories
	009543 Starter Kit Translucent (TR): 1x syringe Translucent (8.7 g / 5 mL), G-Premio BOND (5 mL), G-Multi PRIMER (5 mL) & accessories
	009544 Refill A2 - 1x (8.7 g / 5 mL)
	009545 Refill Translucent TR - 1x (8.7 g / 5 mL)
	009546 Refill Opaque - 1x (8.7 g / 5 mL)
	009547 Refill Bleach - 1x (8.7 g / 5 mL)
	012803 GC G-CEM Try-In Paste A2 (1.5 g / 1.2 mL)
	012804 GC G-CEM Try-In Paste Translucent (1.5 g / 1.2 mL)
	012805 GC G-CEM Try-In Paste Opaque (1.5 g / 1.2 mL)
	012806 GC G-CEM Try-In Paste Bleach (1.5 g / 1.2 mL)
	012694 G-Premio BOND (5 mL) bottle refill
	009552 G-Premio BOND DCA (3 mL)
	900668 GC Automix Tip Regular (x10)
	900495 GC Automix Tip for Endo (x10)
G-CEM Veneer	
	012383 Kit: 1x syringe A2 (1.7 g / 1 mL), 1x syringe TR (1.7 g / 1 mL), 1x G-CEM Try-in Paste Refill A2 (1.5 g / 2 mL), 1x G-CEM Try-in Paste Refill A2 (1.5 g / 2 mL), G-Premio BOND (5 mL), G-Multi PRIMER (5 mL), accessories
	012384 Refill A2 (1.7 g / 1 mL)
	012385 Refill Translucent (1.7 g / 1 mL)
	012386 Refill Opaque (1.7 g / 1 mL)
	012387 Refill Bleach (1.7 g / 1 mL)
	012803 GC G-CEM Try-In Paste A2 (1.5 g / 1.2 mL)
	012804 GC G-CEM Try-In Paste Translucent (1.5 g / 1.2 mL)
	012805 GC G-CEM Try-In Paste Opaque (1.5 g / 1.2 mL)
	012806 GC G-CEM Try-In Paste Bleach (1.5 g / 1.2 mL)
	012694 G-Premio BOND (5 mL) bottle refill
Temporary Cements	
	000087 FREEGENOL TEMPORARY PACK 1-1 Pack tubes (55 g Base, 20 g Accelerator, 2.5 mL Cleaner)
	004253 GC Fuji TEMP LT 2x (13.3 g / 7.2 mL) Paste Pak cartridges 2x (13.3 g / 7.2 mL)
Primers	
	009553 G-Multi PRIMER (5 mL) Liquid
	008551 GC CERAMIC PRIMER II (3 mL) Liquid
	008620 GC CERAMIC PRIMER II Unit dose pack (10 pcs)
	009295 GC METAL PRIMER Z (5 mL) Liquid

GC EUROPE N.V. | Head Office
 Researchpark
 Haasrode-Leuven 1240
 Interleuvenlaan 33
 B-3001 Leuven
 Tel. +32.16.74.10.00
 Fax. +32.16.40.48.32
 info.gce@gc.dental
<https://www.gc.dental/europe/en>

GC UNITED KINGDOM Ltd.
 Coopers Court Newport Pagnell
 Buckinghamshire MK16 8JS
 United Kingdom
 Tel. +44.1908.218.999
 Fax. +44.1908.218.900
 info.uk@gc.dental
<https://www.gc.dental/europe/en-GB>