xantasil®

Heraeus

DE	Gebrauchsanweisung .....	5
GB	Instructions for use .....	11
FR	Mode d'emploi .....	17
ES	Instrucciones de uso .....	23
IT	Istruzioni per l'uso .....	29
SE	Bruksanvisning .....	35
DK	Brugervejledning .....	41
NO	Bruksanvisning .....	47
FI	Käyttöohjeet .....	53

Heraeus

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Stand: 2012-01

Xantasil is a medium flow, addition-curing, elastomer dental impression material that can be used for typical alginate applications instead of traditional alginates.

**Material**

Xantasil Dynamix

Xantasil

**Packaging**

Cartridge, 5:1, 380 ml

Cartridge, 1:1, 50 ml

**Areas of application**

- Impressions for the fabrication of
  - study models
  - temporary restorations
  - splints
  - orthodontic appliances
- Impressions of opposing jaws

**Side effects**

This product or one of its components may cause hypersensitive reactions in some cases. If this is suspected, information on the ingredients can be obtained from the manufacturer.

**Notes on use**

Do not combine with other impression materials. In all addition-curing silicone impression materials, the setting process may be affected by contact with products containing active sulphur or nitrogen, or by fresh composite fillings and superstructures. Inhibiting substances may be found, for example, in latex gloves or clot retraction solutions.

Xantasil should be at room temperature when used (23°C/73°F). Addition-curing silicone impression materials are chemically resistant. Material which has not set can soil clothing. Visually inspect for damage before use. Damaged products must not be used.

### Product Data / Accessories

Xantasil	EN ISO 4823	Product colour		Mixing ratio	Mixing time	Processing time*	Min. time in mouth***	Accessories
		Base	Catalyst					
<b>Xantasil Dynamix fast set</b>	Type 2 medium flow	light orange	white	5:1	[s] auto	[min:s] 1:00**	[min:s] 1:30***	Mixing Tips dynamic yellow 66035999
<b>Xantasil Cartridge fast set</b>	Type 2 medium flow	orange	white	1:1	auto	1:00	1:30***	green 66001989

\* based on room temperature conditions 23°C (73°F), 50 % rel. humidity. Higher temperatures reduce these times; lower extend them.

\*\* applies when using Dynamix® mixer from Heraeus Kulzer

\*\*\* Total working time: ≥ 02:30 [min:s]

### Model fabrication:

The impression can be cast as fast as 30 minutes after removal from the mouth – there are no other time constraints. The impression is stable for long-term storage.

**Recommended model material:**

All standard model materials can be used.

We recommend:

Class III plaster, MOLDANO®

Class IV plaster, MOLDASTONE®/MOLDASYNT®

**Disinfection:**

Impressions can be treated with an aqueous sodium hypochlorite solution (5.25%, for up to 10 minutes\*) or with other disinfectant solutions suitable for silicones. When doing so, please observe the disinfection/processing instructions issued by the relevant manufacturers.

\* (DGZMK 3/93 V1.0)

**Adhesion:**

We recommend using our Universal Silicone Adhesive.

**Storage conditions:**

Do not use Xantasil impression materials beyond the expiration date.

Do not store above 25°C (77°F).

**Note on cartridge system, 50 ml**

**Dispense a small amount of material from a 50 ml cartridge before attaching a mixing tip. After use, leave the mixing tip on the cartridge to keep it closed. Immediately before the next use, remove the mixing tip and attach a new one. Mixing tips are designed for single use.**

## Dynamix system, 380 ml – step by step

Fig. 1  
Remove cap.  
Insert cartridge  
into the mixer

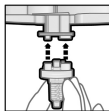
Fig. 2  
Attach Dynamix  
Mixing Tip

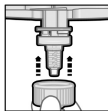
Fig. 3  
Position Dynamix  
fixation ring

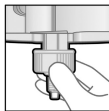
Fig. 4.1  
Attach Dynamix  
fixation ring.  
Slide over  
recesses

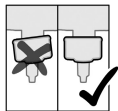
Fig. 4.2

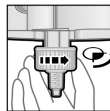
Fig. 5  
Lock by giving  
fixation ring  
a ¼ turn  
clockwise

### Inserting the Dynamix cartridge:

For information on how to handle the mixer being used please refer to the detailed instructions issued by the relevant manufacturer.

1. Hold the tab on the cap of the outlets and bend it upwards through 90° without applying any excessive force.
2. Take the tab between your thumb and index finger whilst you hold it securely with your other hand and remove the cap from the cartridge completely. (Fig. 1)
3. Place one of the enclosed Dynamix Mixing Tips over the outlets of the cartridge (Fig. 2).
4. If the mixing tip cannot be easily attached, check whether the internal hexagon at the center of the mixing tip is aligned correctly with the hexagon of the top shaft.

- 
5. Once the mixing tip is in the correct position (Fig. 3), push the Dynamix fixation ring over the mixing tip until it stops and turn it to the right (1/4 turn) until it is securely locked in place. (Figs. 4.1, 4.2, 5)
  6. Dispense the mixed impression material into a suitable impression tray.
  7. After placing the material, the plungers of the mixer must be pulled back out of the cartridge in order to prevent prolonged pressure and thus the additional flowing of excess material from the mixing tip.

#### **Notes about safe use of the Dynamix System**

Dynamix cartridges are heavy and can be damaged if they are dropped. Damaged cartridges must no longer be used.

Only use Dynamix cartridges in mixers with a closed protective flap.

Before using for the first time, adjust the levels of the two material components. Dispense approx. 2–3 cm material and discard this amount before starting to fill the tray. When the cartridge is used in different mixers, 2–3 cm of material has to be discarded after each device change. Each time before you use the material check the outlet channels of the cartridge for blockages.

At the end of a working day or if the mixer is not going to be used for a lengthy period, remove the cartridge from the device and store it in a horizontal position with the outlets closed off by the mixing tip or cap.

**After use, leave the mixing tip on the cartridge to keep it closed. Immediately before the next use, remove the mixing tip and attach a new one.  
To do so, detach the Dynamix fixation ring and use it again for the new mixer attached.  
Mixing tips are designed for single use.**

Only use Dynamix System components for dispensing and mixing the impression material in Dynamix cartridges.

Check the system components for physical damage on a regular basis and only use components which are intact.

Empty cartridges must be disposed of safely in accordance with the EU directive 94/62/EC.

#### **Warnings and safety instructions**

If material comes into contact with the eyes, rinse it out with water for several minutes. If symptoms continue, consult a doctor.

Do not swallow and ingest. If health problems arise after swallowing impression material, immediately contact a doctor. In rare cases, intestinal blockage may arise.

**To be used only by dental personnel for its intended purpose.**

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Heraeus Kulzer GmbH
Heraeus Kulzer, LLC