



Varios
iPiezo engine®



EC REP

NAKANISHI INC.
www.nsk-inc.com

700 Shimohinata
Kanuma-shi
Tochigi 322-8666,
Japan
TEL : +81 (0)289-64-3380
FAX : +81 (0)289-62-5636

NSK Europe GmbH
www.nsk-europe.de

Elly-Beinhorn-Strasse 8
65760 Eschborn,
Germany
TEL : +49 (0)6196 77606-0
FAX : +49 (0)6196 77606-29

NSK United Kingdom Ltd
www.nsk-uk.com

Office 5, Gateway1000,
Arlington Business Park,
Whittle Way,
Stevenage, SG1 2FP, UK
TEL : +44 (0)1438 310670
FAX : +44 (0)1438 310671

NSK Oceania Pty Ltd
www.nsk-inc.com

Unit 22, 198-222 Young St.
Waterloo, Sydney,
NSW 2017, Australia
TEL : +61 (0)2 8306 3000
FAX : +61 (0)2 8306 3030

NSK America Corp
www.nskamerica.com

1800 Global Parkway
Hoffman Estates, IL 60192,
USA
TEL : +1 888 675 1675
FAX : +1 800 838 9328

NSK Middle East
www.nsk-inc.com

Room 6EA-701, 7th Floor,
East Wing No.6
Dubai Airport Free Zone,
PO Box 54316 Dubai, UAE
TEL : +971 (0)4-204-5507
FAX : +971 (0)4-204-5508

NSK Asia
www.nsk-inc.com

1 Maritime Square,
#09-33 HarbourFront Centre,
Singapore 099253
TEL : +65-6376-5018
FAX : +65-6376-0233



Versatile Power & Performance



Varios[®]

iPiezo engine[®]

Follow Your Intuition

The new, stylish and versatile Varios series features the intuitive NSK iPiezo engine offering you more effective and efficient treatment in all clinical applications. Together with a wide range of tips, the applications are endless; from Perio, Scaling and Endodontic functionality to Implant maintenance. Twin LED lights at the head of the slim-line handpieces ensures much clearer vision and allows for easier identification of calculus and plaque during oral hygiene and scaling procedures.

Varios2 Handpieces

Outstanding Performance

Weighs only 33g, Varios handpiece's slim and easy-to-handle body offers superior performance while reducing your fatigue even in long use. The twin LED lights assure excellent, virtually shadow free visibility, dramatically improving working conditions in the operating area. A lightweight, compact and comfortable handpiece delivers outstanding performance to meet your needs in a variety of clinical practices.

Advantages

- Slimmest and lightest* handpiece with LED illumination
- Twin LED lights through twin optical glass rods
- Optic and non-optic handpieces available
- Extensive tip range for a wide range of applications




Well Balanced, Slimmest and Lightest Handpiece with LED Illumination

By using the slim handpiece excellent accessibility and high visibility is ensured, offering more comfortable use. The handpiece can be repeatedly autoclaved up to 135°C and disinfected as well.



Varios2 Handpiece

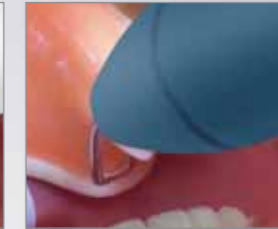

Competitor



Optic



Non-Optic



Twin LED lights



NSK LED delivers natural daylight quality light bringing significant advantages to the way you work

Clearer : LEDs generate natural daylight quality light which assures clearer vision when compared to halogen light.

Durable : LED light sources are longer lasting and more durable than halogen bulbs.

*as of Nov. 2009



Washable in the thermodisinfectior



Autoclavable up to 135°C

Varios 970 **Optic / Non-Optic** iPiezo engine®

Multifunction Portable Unit for Extreme Versatility

All-in-one – Stylish compact all-in-one unit for Periodontic, Endodontic, Scaling, MI applications and Implant maintenance. Convenient large capacity bottles of 400ml of the same solution each or different solutions with independent pumps. The all new Varios features the latest developments in piezo technology; The iPiezo engine from NSK assures stable output by automatically adjusting to the optimum frequency depending on the loading of the tip.



Large LED Display

All operating parameters are clearly displayed on the large LED display controlled by easily recognizable buttons.



iPiezo engine®

Feedback Function

The NSK iPiezo engine supplies the appropriate power to the tip depending on the condition of tooth surface. iPiezo engine provides a stable frequency output without dropping the power, even when hard calculus.

Auto Tuning Function

Auto Tuning Function provides stable and smooth frequency to the tip at the selected power range. It allows the operator to feel the power output and approach the treatment precisely which ensures the ideal result.



Easily Interchangeable Solution Bottles

2 large 400ml bottles are quickly and easily exchanged. Appropriate solution can be used for several clinical purposes.



Memory Function

User settings including power and volume are memorized internally saving valuable time. Additionally, even during power failures, these user settings remain intact.

Simple Unit Design for Clean Maintenance

Simple and seamless unit design realizes optimum hygiene for daily use.

Irrigation Setting with Right and Left Bottle and Tap Water

Varios 970 can use both right and left bottle, and tap water. Select the Irrigation Mode with the select key on the front panel. LED on front panel indicates which bottle is in use.



Varios 970 **Optic / Non-Optic** iPiezo engine®

Improved Durability Against Antiseptic Solutions

The introduction of the resin based water pipe of the bottle, pump and handpiece, assures dramatically improved durability as it substantially reduces the corrosion and rust caused by the residuum of antiseptic solutions.



Auto Cleaning Mode

The Auto Cleaning Mode flushes the tubing and handpiece at the push of a button after the use of special solutions. Activating Varios 970 auto cleaning mode flushes both irrigation lines independently.



Dial Function


The newly designed interface, dial type and palmtop type, makes it easy to adjust and control the water volume from the attached bottle and water supply.



360° Foot Control

New Varios Foot Control can be operated easily from any angles.

MODEL: **FC-70**
ORDER CODE: **Z1008001**
• 2.5m cord

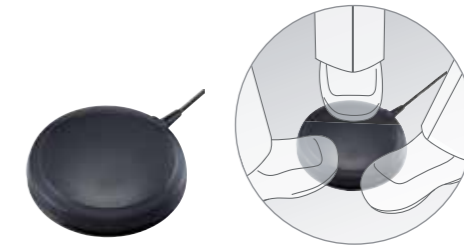

Sterilization Cassette, Tip Wrench

The Sterilization Cassette is designed for the safe processing and storage of Varios components.

MODEL: **Tip Wrench**
ORDER CODE: **Z221076**

MODEL: **Sterilization Case**
ORDER CODE: **Z1035001**



Varios 970 Complete Sets



Optic Varios 970 Optic Complete Sets

MODEL: **VA970 LUX (120V)** ORDER CODE: **Y1001166**

MODEL: **VA970 LUX (230V)** ORDER CODE: **Y1001168**

Non-Optic Varios 970 Non-Optic Complete Sets

MODEL: **VA970 (120V)** ORDER CODE: **Y1001174**

MODEL: **VA970 (230V)** ORDER CODE: **Y1001175**

Specifications

- Power supply : AC 120V or 230V 50/60 Hz
- Frequency : 28-32 KHz
- Max. output : 11 W (G mode)
- Dimensions : W160 x D270 x H190 (mm)
(Including Bottle)
- Unit weight : 2.1 kg (Except attachment)
- Bottle volume : 400 mL (x 2 bottles)

Contents

- Control unit • Handpiece • Handpiece cord • 3 Scaling tips (G4, G6, G8) • Foot control (FC-70)
- 2 x Irrigation bottles • 3 x Tip wrench with torque limiter • AC power cord • Sterilization case

Varios 370 **Optic / Non-Optic** iPiezo engine®

Compact Multipurpose Control Unit at Your Fingertips

Varios 370 has been especially developed to offer a compact, portable control unit allowing easy installation into any dental unit. It is simple to operate and suitable for a variety of clinical procedures.



Advantages

- Powered by iPiezo engine
- Potable ultra compact body
- Simple and intuitive operation control panel
- Large water adjustment and power knob for easy operation
- Holders are available for various installation
- Handpiece is washable in the thermodisinfectant and autoclavable up to 135°C

Control Unit Holder

Control Unit Holder is available for various install positions.



Varios 370 Units Installation Examples



On the dental console arm with Stand (optional).



On/Under the dental bracket table with Control Unit Holder



Behind the rear of the dental mobile cart with Mounting Bracket (optional).

Varios 370 Complete Sets



Optic Varios 370 Optic Complete Sets

MODEL: **VA370 LUX (120V)** ORDER CODE: **Y1001958**

MODEL: **VA370 LUX (230V)** ORDER CODE: **Y1001343**

Non-Optic Varios 370 Non-Optic Complete Sets

MODEL: **VA370 (120V)** ORDER CODE: **Y1001956**

MODEL: **VA370 (230V)** ORDER CODE: **Y1001340**

Specifications


- Power supply : 100-240V AC 50/60 Hz
- Frequency : 28-32 KHz
- Max. output : 1.1 W (G mode)
- Dimensions : W80 x D115 x H32 (mm)
(with out knob and cord)
- Unit weight : 0.43 kg (without attachment)

Contents

- Control unit with handpiece cord
- Control unit holder
- AC adaptor
- Handpiece
- Handpiece holder
- Water filter set
- Water supply connector (120V only)
- Foot control (FC-71)
- 3 Scaling tips (G4, G6, G8)
- Tip wrench with torque limiter

Varios tips

NSK Offers A Wide Choice of Varios Ultrasonic Scaler Tips For Clinical Applications.

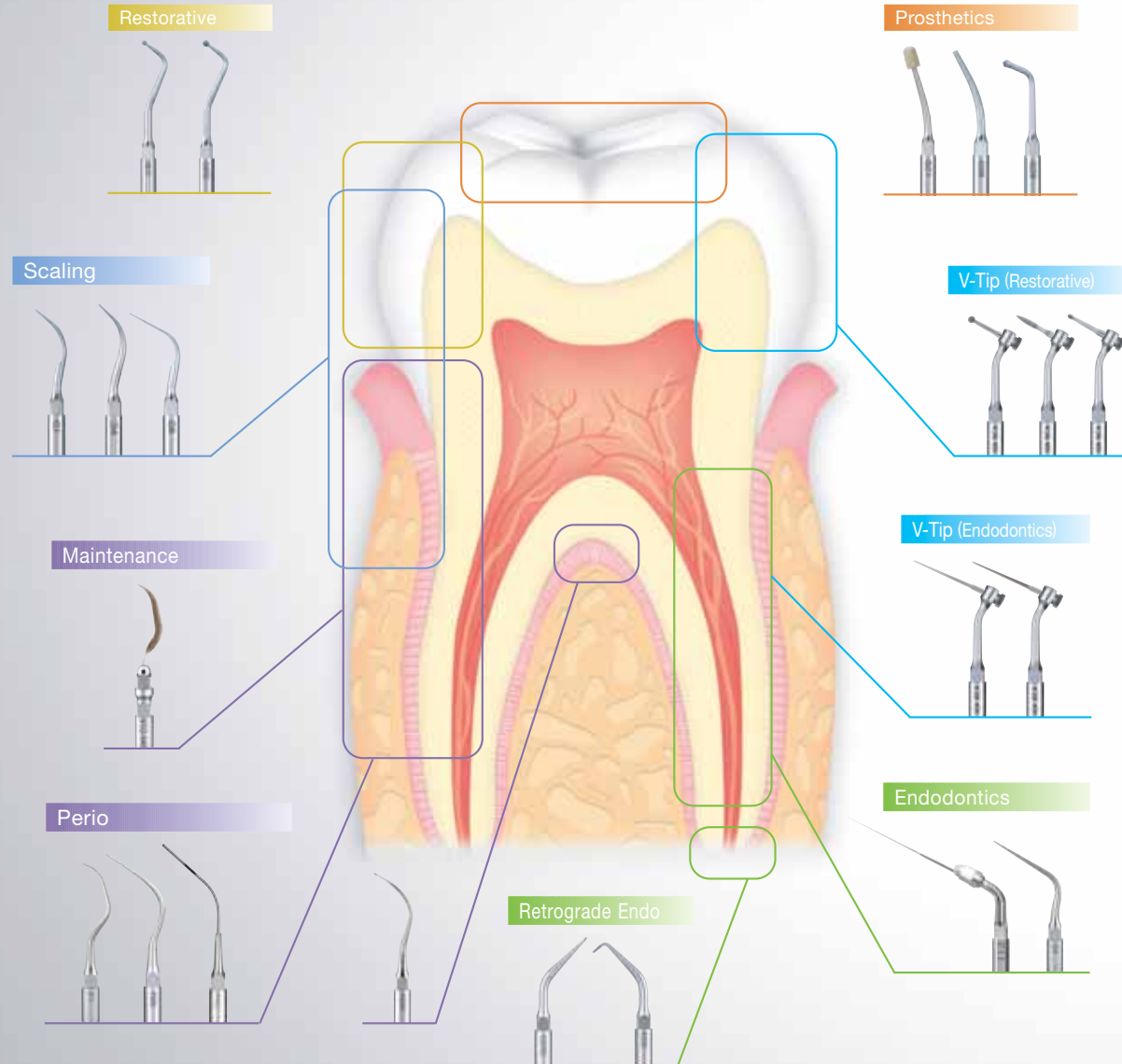

* The direction of the tip's curve is defined by the tip's anterior view.

Scaling

Cross Section Diagram	Tip Image 1	Tip Image 2	Tip Image 3	MODEL: G1	ORDER CODE: Z217101
				MODEL: G1-E	ORDER CODE: Z291101
				MODEL: G2	ORDER CODE: Z217102
				MODEL: G2-E	ORDER CODE: Z291102
				MODEL: G3	ORDER CODE: Z217103
				MODEL: G3-E	ORDER CODE: Z291103
				MODEL: G4	ORDER CODE: Z217104
				MODEL: G4-E	ORDER CODE: Z291104
				MODEL: G5	ORDER CODE: Z217105
				MODEL: G6	ORDER CODE: Z217106
				MODEL: G6-E	ORDER CODE: Z291106
				MODEL: G8	ORDER CODE: Z217108
				MODEL: G11	ORDER CODE: Z217411
				MODEL: G12	ORDER CODE: Z217412
				MODEL: G13	ORDER CODE: Z217413



G1
Single rooted / Multi rooted



G4
Single rooted / Multi rooted



G6
Single rooted / Multi rooted



G8
Single rooted / Multi rooted



Washable in the thermoisinfectant




Autoclavable up to 135°C

Perio (Root Planing)

			MODEL: P1	ORDER CODE: Z217404	
			MODEL: P1D	ORDER CODE: Z217405	
	• Diamond coated				
			MODEL: P2D	ORDER CODE: Z217402	
	• Diamond coated • Right curved tip				
			MODEL: P3D	ORDER CODE: Z217403	
	• Diamond coated • Left curved tip				
			MODEL: P10	ORDER CODE: Z217410	
	• Sharp edge				
			MODEL: P11R	ORDER CODE: Z217408	
	• Right curved tip				
			MODEL: P11L	ORDER CODE: Z217409	
	• Left curved tip				
			MODEL: P20	ORDER CODE: Z217420	
			MODEL: P20-E	ORDER CODE: Z291420	
			MODEL: P21R	ORDER CODE: Z217422	
	• Right curved tip		MODEL: P21R-E	ORDER CODE: Z291422	
			MODEL: P21L	ORDER CODE: Z217421	
	• Left curved tip		MODEL: P21L-E	ORDER CODE: Z291421	



Single rooted

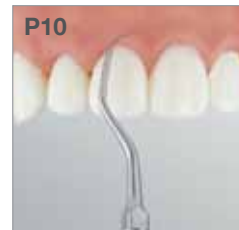

Single rooted / Multi rooted



Single rooted / Multi rooted




Single rooted

Perio (Root Planing)

			MODEL: P25R	ORDER CODE: Z217424	
	• Right curved tip		MODEL: P25R-E	ORDER CODE: Z291424	
			MODEL: P25L	ORDER CODE: Z217425	
	• Left curved tip		MODEL: P25L-E	ORDER CODE: Z291425	
			MODEL: P26R	ORDER CODE: Z217426	
	• Right curved tip		MODEL: P26L	ORDER CODE: Z217427	
	• Left curved tip				



Single rooted / Multi rooted

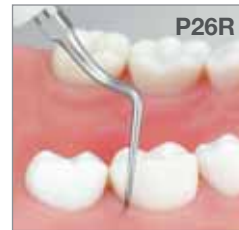

Single rooted / Multi rooted

Perio (Irrigating and Cleaning)

			MODEL: P40	ORDER CODE: Z217550	
	• For irrigation (ø 0.6 mm)				
			MODEL: P41	ORDER CODE: Z217555	
	• For irrigation (ø 0.8 mm)				



Dual irrigation points

Maintenance (V-Tip)

	V-Tip Holder		MODEL: V10	ORDER CODE: Z217041	
		* Plastic Tip is not included			
					• Includes E-Tip replacement wrench
			MODEL: V-P10	ORDER CODE: Y900184	
					• Pack of 3



Single rooted / Multi rooted

* The direction of the tip's angle is defined by the tip's anterior view.






Washable in the thermoisinfectior



Autoclavable up to 135°C

Endodontics

Cross Section Diagram ○			MODEL: E4	ORDER CODE: Z217428	NSK Satelec®	• For root canal cleaning
○			MODEL: E4D	ORDER CODE: Z217528	NSK Satelec®	• Diamond coated • For root canal enlargement
○			MODEL: E5	ORDER CODE: Z217305	NSK Satelec®	• Water not required • For lateral condensation
○			MODEL: E6	ORDER CODE: Z217306	NSK Satelec®	• Water not required • For lateral condensation
○			MODEL: E7	ORDER CODE: Z217307	NSK Satelec®	• Water not required • For removal of filling and foreign material
○			MODEL: E7D	ORDER CODE: Z217317	NSK Satelec®	• Diamond coated • For enlargement of root canal wall
○			MODEL: E8	ORDER CODE: Z217308	NSK Satelec®	• Water not required • For removal of filling and foreign material
○			MODEL: E8D	ORDER CODE: Z217318	NSK Satelec®	• Diamond coated • For enlargement of root canal wall
1.2 1.1 ○			MODEL: E15D	ORDER CODE: Z217319	NSK Satelec®	• Diamond coated • For locating root canals

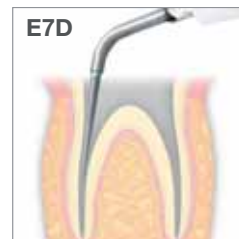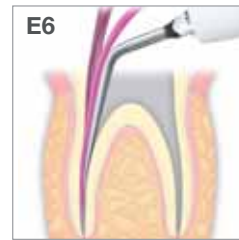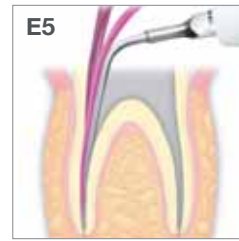


Retrograde Endo

120° Angle Holder for U files



95° Angle Holder for U files

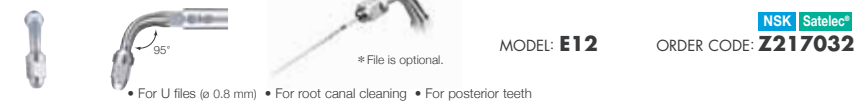

Retrograde Endo

Cross Section Diagram ○			MODEL: E30RD	ORDER CODE: Z217212	NSK Satelec®	• Diamond coated • For posterior teeth (right angled)
○			MODEL: E30LD	ORDER CODE: Z217213	NSK Satelec®	• Diamond coated • For posterior teeth (left angled)
○			MODEL: E31D	ORDER CODE: Z217210	NSK Satelec®	• Diamond coated • For anterior and posterior teeth (70°)
○			MODEL: E32D	ORDER CODE: Z217211	NSK Satelec®	• Diamond coated • For anterior teeth (90°)



Restorative (For Minimal Intervention)

0.9 1.0 ○			MODEL: G61D	ORDER CODE: Z217601	NSK Satelec®	• Diamond coated
1.23 1.4 ○			MODEL: G62D	ORDER CODE: Z217602	NSK Satelec®	• Diamond coated



Washable in the thermoisinfector



Autoclavable up to 135°C

V-TIP System Line Up



V-Tip Holder



MODEL: **V30** ORDER CODE: **Z217030**
 • Includes E Tip replacement wrench



Restorative

For Minimal Intervention

MODEL: **V-G70** ORDER CODE: **Y900118**
 • Diamond coated • Pack of 3

MODEL: **V-G71** ORDER CODE: **Y900119**
 • Diamond coated • Pack of 3

MODEL: **V-G72** ORDER CODE: **Y900120**
 • Diamond coated • Pack of 3

MODEL: **V-G75** ORDER CODE: **Y900121**
 • Diamond coated • Pack of 3

MODEL: **V-G76** ORDER CODE: **Y900122**
 • Diamond coated • Pack of 3

For Finishing / Trimming / Polishing

MODEL: **V-G77** ORDER CODE: **Y900165**
 • Diamond coated • Normal Grade • Pack of 3

MODEL: **V-G78** ORDER CODE: **Y900166**
 • Diamond coated • Fine Grade • Pack of 3

MODEL: **V-G79** ORDER CODE: **Y900167**
 • Diamond coated • Super Fine Grade • Pack of 3

For Caries of Dentin

MODEL: **V-51** ORDER CODE: **Y900161**
 • Stainless Tip • Pack of 3

MODEL: **V-53** ORDER CODE: **Y900162**
 • Stainless Tip • Pack of 3

MODEL: **V-533** ORDER CODE: **Y900163**
 • Stainless Tip • Pack of 3

Endodontics

For Root Canal Cleaning

MODEL: **V-550** ORDER CODE: **Y900169**
 • For root canal cleaning 17mm • Pack of 3

MODEL: **V-551** ORDER CODE: **Y900170**
 • For root canal cleaning 23mm • Pack of 3

MODEL: **V-552** ORDER CODE: **Y900171**
 • For root canal cleaning 26mm • Pack of 3

MODEL: **V-553** ORDER CODE: **Y900172**
 • For root canal cleaning 32mm • Pack of 3

For Root Canal Enlargement

MODEL: **V-550D** ORDER CODE: **Y900173**
 • Diamond coated • For root canal enlargement 17mm • Pack of 3

MODEL: **V-551D** ORDER CODE: **Y900174**
 • Diamond coated • For root canal enlargement 23mm • Pack of 3

MODEL: **V-552D** ORDER CODE: **Y900175**
 • Diamond coated • For root canal enlargement 26mm • Pack of 3

MODEL: **V-553D** ORDER CODE: **Y900176**
 • Diamond coated • For root canal enlargement 32mm • Pack of 3


• Cervical decay removal




• Contact surface of tooth decay removal




• Root canal enlargement

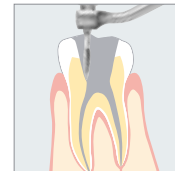


• Root decay removal

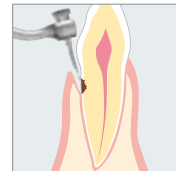

• Root canal cleaning

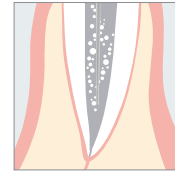

• Removal of broken files

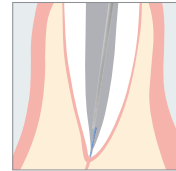

Spare Parts

MODEL	ORDER CODE	MODEL	ORDER CODE	MODEL	ORDER CODE
VA2-LUX-HP Optic Handpiece	E351050	Water Supply Tube (4m) 4m water supply tube for Varios 370	0914406 040	Handpiece Holder for Varios 370	10001505
VA2-HP Non-Optic Handpiece	E350050	Water Filter Case	Y900069	VA Bottle 400 (400 mL) for Varios 970	Z1047002
VA970LUX-SC Handpiece Cord for Varios 970 LLX	E351055	Water Filter	U387042	Irrigation Pump for Varios 970	10000643
VA970-SC Handpiece Cord for Varios 970	E350055	Tube Ring (Pack of 2)	Y900077	O-ring (for Handpiece Cord)	0310020 080
FC-70 Foot control (2.5m)	Z1008001	Tip Wrench (CR-10)	Z221076	O-ring (Thin section) for VA Bottle	0312090 100
FC-71 Foot control (5m) for Varios 370	Z1008002	E Tip Replacement Wrench	Z217399	O-ring (Thin section) for VA Bottle	0310075 150
Water Supply Connector	U387030	Tip Holder	Z221080	Varios 350/370 Stand	Z254
Water Filter Set (2m) (2m water supply tube, water filter, water filter case)	U387040	Tip Cover L	Z217850	Varios 350/370 Mounting Bracket	Z259
Water Filter Set (4m) (4m water supply tube, water filter, water filter case)	U1007046	Tip Cover S	Z217851	Control Unit Holder	Y1001812
Water Supply Tube (2m) 2m water supply tube for Varios 570/970	U387580	Sterilization case	Z1035001	Spanner Wrench (5x8)	Y1001301





A Wide Range of Tips to
Expand Clinical Practice