


Moisture control made easy

OptraGate®

Latex-free
lip and cheek retractor

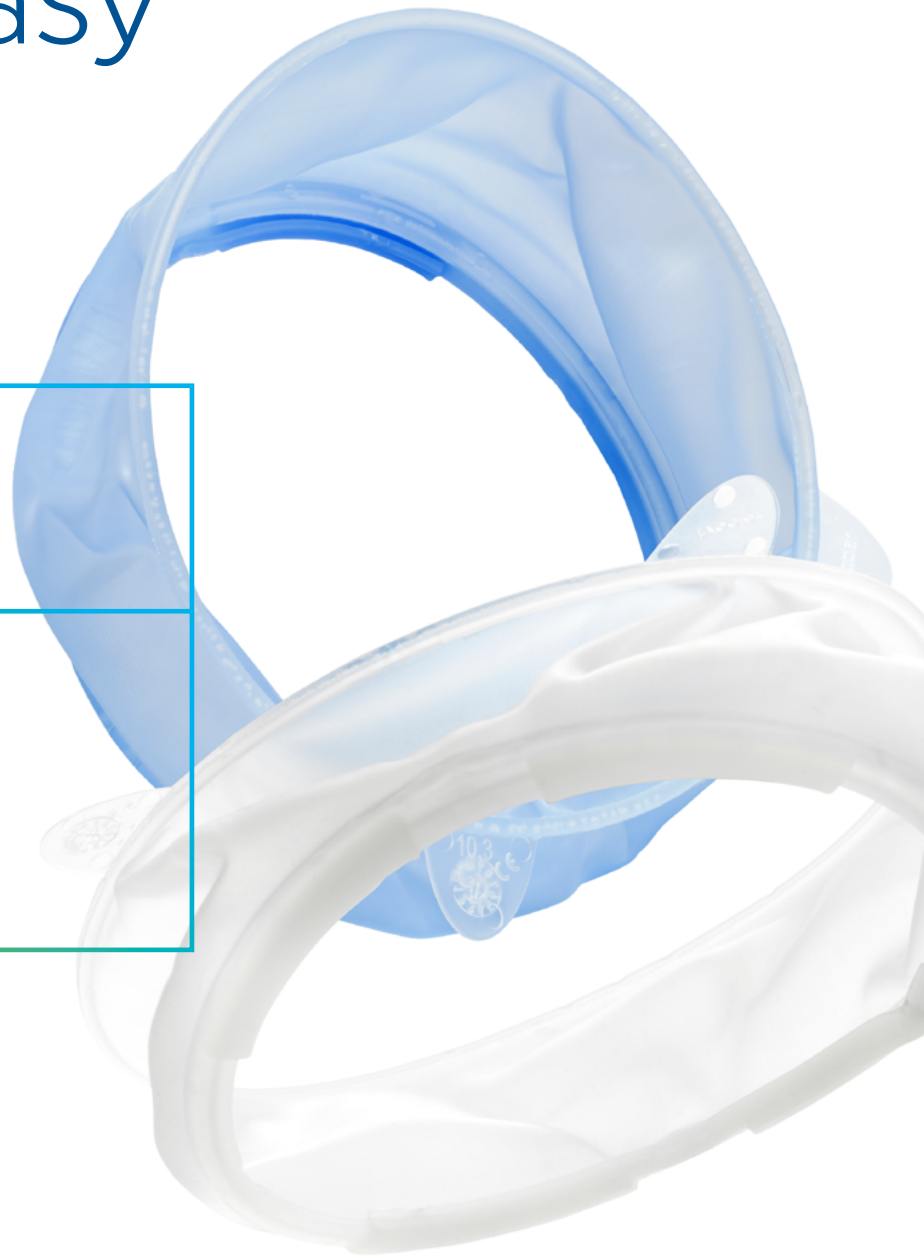

For more comfort in everyday practice

OptraGate®, our latex-free lip and cheek retractor, supports you optimally in your day-to-day procedures. Made of soft, flexible material, the retractor is comfortable to wear and establishes generous access to the treatment field. Enabling improved visibility of the treatment field, OptraGate helps you achieve and maintain easy and effective moisture control.

Esthetics

Relative isolation for improved adhesion and durability

With OptraGate, it is easier to keep the treatment field tidy and dry. This helps decrease the risk of contamination, which could otherwise impair the bonding properties of the materials being used ^[1].

Quality

Relaxed treatment experience for your patients

The material feels comfortable to wear because of its flexibility and texture. OptraGate makes it easier for patients to keep their mouth open during treatment, allowing you to focus more on your procedure.

Efficiency

Quickly placed, efficiently treated

The flexible material ensures easy placement and a good fit in the mouth. There are no obstructions from lips or cheeks, which would hinder an efficient treatment.

^[1] M. R. R. Cajazeira, T. M. De Sabóia, L. C. Maia, Influence of the operator field isolation technique on tooth-colored direct dental restorations, Am J Dent. 2014, 27, 155-159

Every second,

OptraGate® is used in a treatment worldwide*.

Your benefits at a glance:

- ✓ Effective and fast relative isolation
- ✓ Easy-to-access, clear treatment field for a relaxed and efficient treatment process
- ✓ Straightforward to place due to 3-dimensional flexibility and soft material
- ✓ Versatile uses for a variety of clinical situations in your everyday practice

* Based on Ivoclar sales figures

The original, developed by a dentist for dentists



"OptraGate provides me as the operator with a panoramic view of the teeth. One special characteristic is the comfortable feel for my patients, as their lips and cheeks are gently retracted. In addition, OptraGate features a distinctive ability to adapt and be flexible, supporting me as the operator throughout the entire dental procedure and enabling me to perform any functional movement."

Dr Domonkos Horvath, Germany

FAQs about OptraGate

What sizes are available?

There is a choice of three sizes: regular, small and junior.

How do I know what size to use?

When choosing the correct size, you should consider the distance between the corners of the mouth when the mouth is in a relaxed position (with the lips closed or slightly apart). You can use the reference scale printed on the packaging as a guide to select the right size.

Is OptraGate comfortable to wear during lengthy procedures?

If a suitable size has been chosen, patients usually find OptraGate comfortable to wear even over long periods of time.

Flexible material – flexible use

Offering a comfortable wearing experience, high compatibility and a wide range of applications – OptraGate is designed for versatile uses in a variety of clinical situations.


OptraGate excels in many clinical situations, beyond just moisture control:

- ✓ Prophylactic care
- ✓ Digital dentistry
- ✓ Direct restorative treatments
- ✓ Orthodontics
- ✓ Prosthetics
- ✓ Diagnostics
- ✓ Bleaching
- ✓ Paediatric dentistry



How do I insert OptraGate correctly?
See here!

ivoclar.com/en_li/products/accessories/optragate

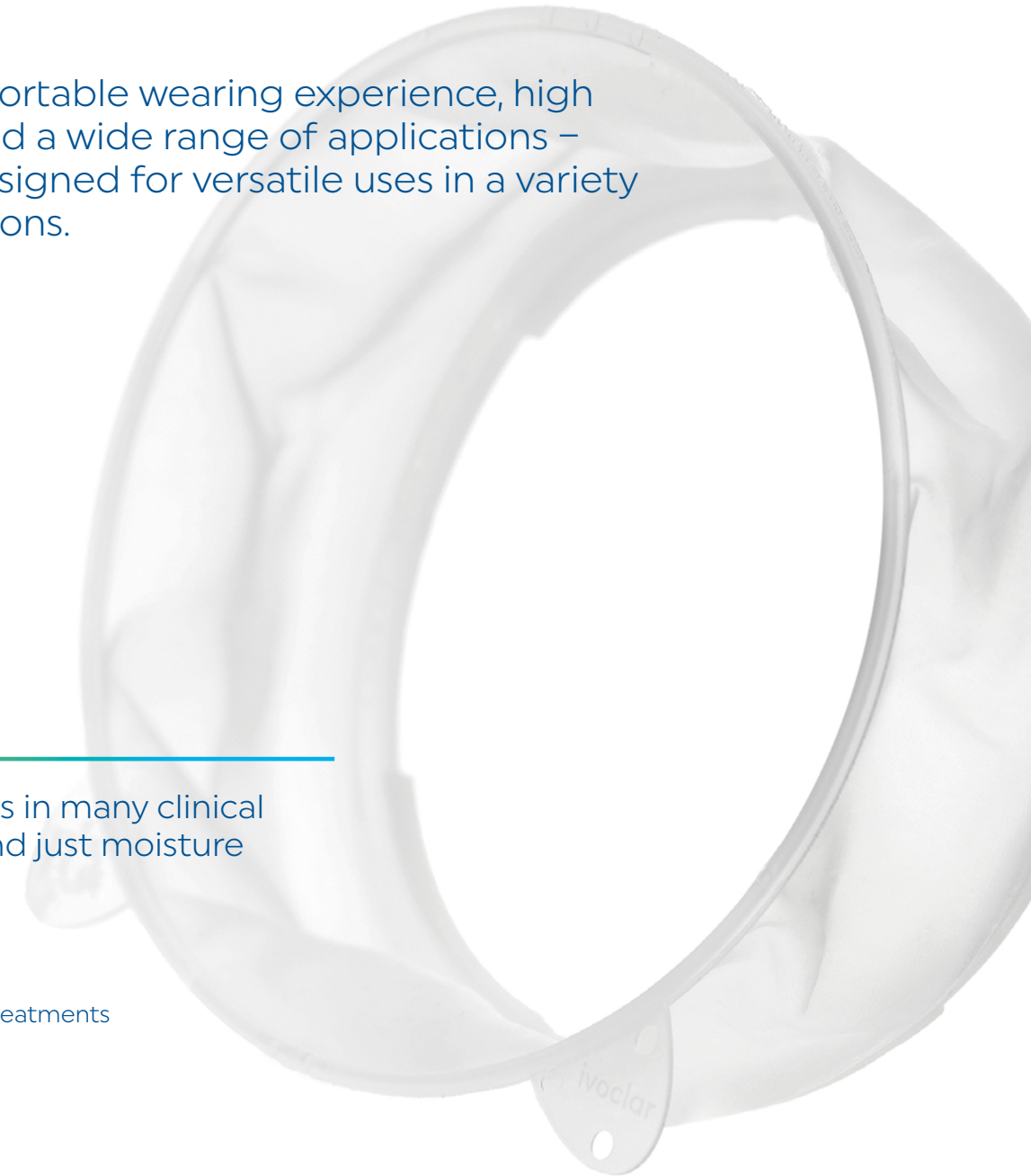

OptraDam Plus – For absolute isolation

OptraDam Plus is a three-dimensional rubber dam with flexible plastic rings and an anatomical shape, designed to establish a completely dry treatment field using an innovative three-dimensional isolation technique.

Your benefits at a glance:

- ✓ Improves the ease of placement due to the presence of flexible plastic rings and the lack of a metal frame
- ✓ Enhances patient comfort by not involving a metal frame
- ✓ Reduces the spread of aerosols in the dental office ^[2]

[2] P. Samaranayake, J. Reid, N. Evans, The efficacy of rubber dam isolation in reducing atmospheric bacterial contamination, ASDC Dent. Child., 1989, 56, 442-444.

Have you checked out our range of clinical accessories?



OptraGloss®

High-gloss polishing of composites
in only one step
www.ivoclar.com/optragloss

OptraSculpt®

Efficient contouring due to anti-stick effect
www.ivoclar.com/optrasculpt



Find out more
about the range of clinical
products from Ivoclar:
ivoclar.com/direct-restoratives

Direct restorative procedures for efficient esthetics

Unlock the potential of our coordinated products for direct restorative procedures, enabling the creation of efficient high-quality esthetic restorations.



Isolate

Comfortable moisture control
with OptraGate



Bond

Efficient bonding with the
Adhese Universal VivaPen



Restore

Tetric Line – the solution
for all cavity classes



Contour

Effortless contouring
with OptraSculpt



Cure

Powerful performance
with Bluephase PowerCure



Polish

Shiny surfaces
with OptraGloss



Protect

Effective prevention
with Fluor Protector S