## Instructions for use

- EN** see [www.fkg.ch](http://www.fkg.ch) for additional information
- FI** Lisätietoja [www.fkg.ch](http://www.fkg.ch)
- DE** weitere Informationen auf [www.fkg.ch](http://www.fkg.ch)

## ScoutRace & ISO 10

Endodontic instruments  
Endodontiainstrumentit  
Endodontische Instrumente

FKG REF. 99.7AA.10.15A.XX - n°104 - 2018/01

## ScoutRace & ISO 10

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### SafetyMemoDisc (SMD)

Pull off petals to master metal fatigue:

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## Speed: 600-800 rpm

Torque: 1.5 Ncm

- Long back and forth gentle strokes
- Light touch, let the instrument work
- Work 3-4 seconds in a row, get out, clean the blade and irrigate the canal

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For canals with a severe apical curvature or S-shape

Manual scouting with ISO 06-08 SSt files up to estimated working length (WL); confirm WL with an apex locator

1. Introduce ScoutRace (SR) 10/.02 in rotation and reach WL;
2. Continue with SR 15/.02 up to WL;
3. Finish glide path with SR 20/.02 up to WL;
4. Final shaping with the main NiTi sequence (e.g. iRace or BioRace).

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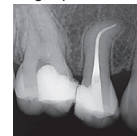
For narrow and/or calcified canals

Manual scouting with ISO 06-08 SSt files up to estimated working length (WL); confirm WL with an apex locator.

1. Introduce Race 10/.02 in rotation and reach WL;
2. Continue with Race 10/.04 up to WL;
3. Finish glide path with Race 10/.06 up to WL;
4. Final shaping with the main NiTi sequence (e.g. iRace or BioRace).

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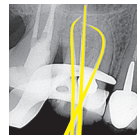
P - 50/.04

## EN Sterilization protocol for reusable medical devices

### Non sterile products

#### First use

Carefully follow the rules herebelow

#### Cleaning and Sterilization

### Sterile products

#### First use

1. Open the alveolus
2. Extract the instrument (use of gloves imperative)
3. Use the instrument according to good dental practice

#### Second use:

Carefully follow the rules herebelow  
**Second and following uses.**

### Cleaning

1. Manual cleaning with/without Ultrasonic assistance

Through cleaning allows efficient disinfection & sterilization.

#### 2. Rinse

Under distilled/demineralized water at least 1 min.  
Then dry the instruments.

### Sterilization

5. According EN/ISO 17664 protocol  
Autoclave: 134°C/273°F,  
2.2 bar during 18 min.

#### 3. Inspection - Check

Sort out damaged or worn out instruments.

#### 4. Packing

Place the instruments in a suitable support/container and pack the devices in sterilisation pouches ISO 11607-1.

#### 6. Storage

Keep devices in sterilization wrap/pouch in a dry and clean environment.

 Always refer to manufacturer's instructions for cycle & duration

### Second and following uses

#### 7. Pre-disinfection

Plunge instruments after use in detergent solution and eventually brush them manually

#### 8. Rinse

Under running water at least 1 min.  
Then dry the instruments.

#### 9. Follow steps 1 - 6 above

 Complete operating protocol and warnings on [www.fkg.ch](http://www.fkg.ch)

### General information

#### Pre-/Disinfection DO NOT USE

solutions containing:  
- phenol (corrosion)  
- aldehyde (blood fixation)  
- di-/trietanolamines (corrosion)

**Sterilization with chemiclav or dry air devices** have not been validated by FKG Dentaire.

Refer to manufacturer's instructions for cycle & duration. Nevertheless our instruments withstand such methods.

#### Caution:

This product contains nickel and should not be used for individuals with known allergic sensitivity to this metal.

 200°C (392°F)

A single patient use is recommended to avoid cross-contamination.

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