Sendoline S5 System STERILE

Sendoline Big Apical Files

Sendoline S1 System

Sendoline S3 System

EN

Instructions for use

Sendoline sterile rotary and reciprocating endodontic files

Intended use

Sendoline instruments covered by these instructions are intended for mechanical treatment of the root canal by qualified users in a clinical environment. The purpose of the instruments is to clean and shape the root canal system.

Design

The device operative part is made from nickel-titanium and is defined as a type 2 instrument according to EN ISO 3630-1. The shank is type 1 according to EN ISO 1797.

Contra-indications

General contra-indications for root canal treatment.

Precautions

- Prior to use, store the instruments in their original packaging in a dry, dust free and clean environment. Protect from sunlight.
- The instruments are provided in sterile condition; sterilized using gamma. If the package is damaged or unintentionally opened, the instruments must not be used.
- The instruments are intended for single use with the purpose to avoid cross-infection and to minimize the risk of file fracture following repeated use.
- The instruments are sharp and must be handled with care to avoid injuries. Follow national healthcare regulations for safe disposal of sharps.
- In case of a serious incident from using the instruments, immediately contact Directa AB and the applicable authority in your country.

Indications for use

The instruments are intended for mechanical instrumentation and should be used together with the specified endodontic handpiece/motor using the settings, if any, specified in the tables in this document.

- Create straight-line access to the canal orifice(s).
- Establish working length (WL) and create a reproducible glide path with small size manual hand instruments and/or mechanical glide path instruments.
- Work in the canal using the file sequence of the device model of choice in accordance with canal shaping techniques applicable to mechanical instruments.

Symbols

 USA: For professional use only	 Reciprocating movement
 Setting	 Clockwise rotation
 File type S – the S figurative mark is the property of Directa	 File type K

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