VOCO**FinalTouch**

Instructions for use

Product description:

FinalTouch is a light-curing composite for individual shade characterisation of direct and indirect composite restorations and masking of dental hard tissue discolourations. **FinalTouch** is applied underneath/between the layers of restorative composite. **FinalTouch** contains 38 - 40 % w/w inorganic fillers in a methacrylate matrix.

Colours:

white, blue, yellow, orange, brown

Table 1: Available colours and their possible applications.

Colour	Possible applications
White	- Reproduction of enamel opacities, calcium deposits and white spots - Reproduction of halo effects - Accentuation of cusps and ridges - Masking of discoloured dental hard tissue
Blue	- Reproduction of translucent enamel areas and opalescent regions
Yellow, orange	- Reproduction of discolourations around the neck of the tooth - Highlighting of dentine core - Accentuation of fissures - Reproduction of individual enamel characteristics
Orange, brown	- Reproduction of enamel cracks and fissure discolourations

Indications:

- Personalised characterisation of direct and indirect composite restorations
- Masking of tooth discolourations
- Characterisation of chairside CAD/CAM veneers

Preparation:

Allow the material to reach room temperature before use. Ensure that the working field is sufficiently dry. Clean the teeth and/or restorations to be treated. **FinalTouch** is used together with light-curing restorative composites and adhesives. Pre-treat the dental hard tissue and existing substrate surface in accordance with the instructions for use of the restorative composite/adhesive used.

Application of FinalTouch:

FinalTouch can be applied directly to the restorative using the metal cannulas supplied, or it can be removed from the syringe and applied with a fine brush/probe or thin root canal instrument. If lower colour intensity is required, the selected **FinalTouch** colour can be mixed with a translucent composite flow material (e.g., **GrandioSO Flow Incisal**) before use, ensuring that no air bubbles are formed in the process.

A) Shade characterisation of direct composite restorations:**Using FinalTouch when placing fillings:**

1. Application of restorative composite as per manufacturer's specifications.
2. To spread **FinalTouch** over a surface area, it can be applied and covered directly when layering the composite. To reproduce enamel cracks and fissure discolourations, mould a fine notch at the required point in the final composite layer, insert the corresponding **FinalTouch** colour into the notch directly from the cannula or with a suitable instrument, and cure it. Pay attention to the specifications concerning the curing time and maximum layer thickness (table 2). Ensure that the top layer is always the restorative composite.

Subsequent insertion of FinalTouch:

When characterising older fillings, it is also possible to cut in characterisation patterns at a later stage. Then clean with water and dry with moisture-free and oil-free air. Once dry, apply and light-cure a suitable adhesive in accordance with the manufacturer's specifications. The required colour can then be inserted into the prepared pattern and cured in accordance with the specifications for curing time, light intensity and maximum layer thickness (table 2). **FinalTouch** needs to be covered with a suitable restorative composite.

B) Shade characterisation of indirect composite restorations:

Corresponding to the procedure for direct composite restorations, **FinalTouch** can be inserted when layering inlays or onlays. It is also possible to insert and cure characterisation patterns at a later stage in restorations which are already polymerised (e.g. CAD/CAM composite restorations). **FinalTouch** needs to be covered with a suitable restorative composite.

C) Shade characterisation of chairside CAD/CAM veneers:

Following adaptation of the veneer and subsequent cleaning and conditioning as per manufacturer's specifications, a thin layer of a light-curing adhesive is applied to the inner surface. This is then dried with an air stream before being polymerised in accordance with the manufacturer's specifications. The selected colour is then applied in a very thin layer, as it could otherwise have a negative effect on the fit of the veneer or its external appearance. Cure **FinalTouch** in accordance with the specifications in table 2 and fit the veneer as per the manufacturer's specifications.

D) Shade characterisation of acrylic teeth:

Once you have cut a characterisation pattern into the acrylic tooth, treat its surface with a suitable light-curing bonding agent for acrylic teeth. Insert the desired colour into the characterisation pattern and cure it. Then cover with restorative composite and polymerise.

E) Masking of tooth discolourations:

Before using **FinalTouch** to mask tooth discolourations, the prepared cavity needs to be pre-treated with a suitable adhesive in accordance with the manufacturer's specifications. Then apply white or a mixture of white and orange, in the layer thickness necessary for sufficient masking, with a brush or another suitable instrument and cure it. See table 2 for maximum layer thicknesses.

Then complete the layering with a restorative composite in accordance with the manufacturer's specifications.

Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. The curing time and maximum layer thickness is as follows when using an LED-/halogen light with a minimum light-output of 500 mW/cm².

Table 2: Maximum layer thicknesses and curing times of individual colours.

Colour	Maximum layer thickness	Curing time
White	0.5 mm	20 s
Blue	0.5 mm	20 s
Yellow	0.5 mm	20 s
Orange	0.5 mm	20 s
Brown	0.3 mm	20 s

Hold the light emission tip of the device as close as possible to the surface of the filling. Otherwise, the curing depth may be reduced. Incomplete curing may lead to discolouration and discomfort.

Information, precautionary measures:

- **FinalTouch** contains methacrylates and BHT. **FinalTouch** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
- The enclosed application tips are intended for single use and must be disposed of after only one use. Return the cap to the syringe after use.
- Phenolic substances, especially preparations containing eugenol or thymol, interfere with the curing of filling composites. Avoid the use of zinc oxide eugenol cements or other eugenol-containing materials in combination with filling composites.

Storage:

Store at temperatures of 4 °C to 23 °C. If refrigerated, the material must be allowed to reach room temperature before use. To avoid exposure to light and potentially resulting polymerisation, syringes should be closed immediately after dispensing. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA2321 E1 0816 99 © by VOCO
VOCO GmbH