Identium[®]

Adhesive

37404/5009

Gebrauchsinformation	Deutsch	2
Directions for use	English.....	3
Mode d'emploi	Français	4
Istruzioni per l'uso	Italiano	5
Instrucciones de uso	Español.....	6
Instruções de uso	Português	7
Bruksanvisning	Norsk	8
Brugsinformation	Dansk	9
Bruksanvisning	Svenska.....	10
Kullanım Kılavuzu	Türkçe.....	11

Identium® Adhesive

Vertrieb in den USA: **Kettenbach LP**
 7777 Center Avenue, Suite 280
 Huntington Beach, CA 92647
 USA

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Identium® Adhesive

Distributed in the U.S. by: **Kettenbach LP**
 7777 Center Avenue, Suite 280
 Huntington Beach, CA 92647
 USA

For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.

Adhesive for Identium® impression materials (Vinylsiloxanether®)

Made in Germany
 37404/5009

Product Description/Indication

Adhesive for Identium® impression materials (Vinylsiloxanether®). Adheres to metals and acrylics.

Instructions for use

Apply a thin coat of Identium® Adhesive onto the dry impression tray and allow it to dry for approx. 5 minutes. Warmth and compressed air curtail the drying time. Cover with a sufficient amount of impression material. When using plastic impression trays, please check that they are compatible with the adhesive. Replace the cap on the bottle immediately after use!

After mixing the impression material, load it into the tray within 30 minutes.

Remains of impression material can be cleaned off the tray with cleaning solvent, petroleum ether or orange oil.

Warnings

Do not use Identium® Adhesive with condensation-curing silicones, addition-curing silicones, polyether or polysulfide materials.

Precautions

Do not use after the expiry date.

Do not swallow the material! If accidentally swallowed: consult a doctor if problems arise or persist.

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

Avoid contact with skin and mucosa.

Do not allow the material to enter the sewerage or water system to avoid damage to the environment (hazardous to water). Do not dispose of with household waste.

Avoid contact with clothing, as the material cannot be removed by dry cleaning.

Keep away from sources of ignition.

Side effects

Although there are no known systemic side effects, specific contact allergies cannot be ruled out in rare cases.

Do not use the material in case of known allergy to one of the ingredients.

Composition

Silicone resin, solvent (ethyl acetate, xylene).

Storage Conditions:

Store at room temperature (18 – 25 °C/ 64 – 77 °F).

- | | |
|---------|---|
| R 11 | Highly flammable. |
| R 20/21 | Harmful by inhalation and in contact with skin. |
| R 36 | Irritating to eyes. |
| R 66 | Repeated exposure may cause skin dryness or cracking. |
| R 67 | Vapors may cause drowsiness and dizziness. |

Trademarks

Kettenbach and Identium® Adhesive are registered trademarks of Kettenbach GmbH & Co. KG.

Note

Not all Kettenbach products are available in all countries.

Legend

Conformity with MDD 93/42/EEC	CE	Use by	
Temperature limitation		Keep dry	
Keep away from sunlight		Catalog number	REF
Batch code		Caution, consult accompanying documents	
Dual system (Germany only)		Rx only For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.	
Highly flammable		Irritant	

The information provided for Kettenbach products is based on comprehensive research and experience in application technology. Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development. However, users must comply with and consider all recommendations and information in connection with any use.

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Kettenbach® ve Identium® Adeziv, Kettenbach GmbH & Co. KG şirketinin tescilli ticari markalarıdır.