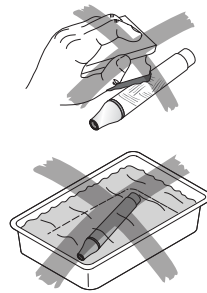
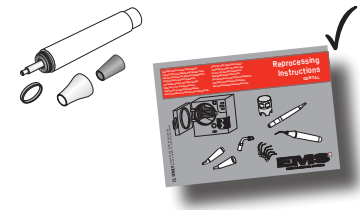
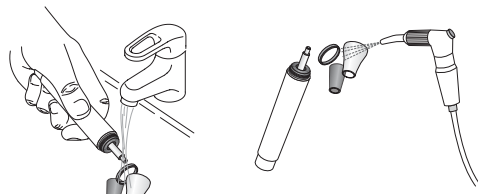
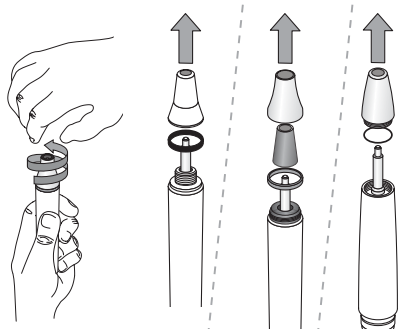
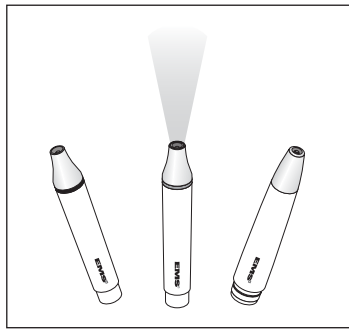
# PIEZON® HANDPIECE

## Piezon® handpiece

 Please carefully read and follow the recommendations of the instruction manual originally delivered with your product regarding the procedures of cleaning, disinfecting, sterilizing and packing of the components. Use an alcohol-based, commercially available, colorless disinfectant (ethanol, isopropanol).

 This product must only be used by a qualified and trained personnel (dentist or dental hygienist).

 Please conform to the recommendations of the Reprocessing Instructions manual delivered with your product regarding the procedures of cleaning, disinfecting, sterilizing and packing of the components.

 Unscrew the cap and the light guide (when equipped) to clean each component under running water before disinfection. Use an alcohol-based, commercially available, colorless disinfectant (ethanol, isopropanol).

 Make sure to always sterilize handpieces before use. Non-sterile handpieces may cause bacterial or viral infections. Please follow present-day rules about sterilization applicable in your country.

 Control regularly that the o-rings of the handpiece and handpiece cord are not damaged.

 Due to the sterilization procedures, the light guide (when equipped) may become opaque; this may reduce the light from the handpiece. In this case, replace the light guide for optimal light transmission. Optionally, use the handpiece without the light guide. The light source in the handpiece cannot be replaced.

This product is designed to be connected to an EMS Piezon equipment.

 Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

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EMS Germany GmbH
EMS Italia S.r.l.
EMS Corporation