VOCO

Provicol®/Provicol® C/ Provicol® QM

EN Instructions for use

MD EU Medical device

In accordance with DIN EN ISO 3107

Product description:

Provicol/Provicol C/Provicol QM, a eugenol-free cement with calcium hydroxide, is easy to process and does not influence the setting of resin-based filling and luting materials.

Provicol/Provicol C/Provicol QM contains calcium hydroxide to support tooth vitality.

Indications:

Provicol/Provicol C/Provicol QM is used as a temporary cement for the attachment and for sealing of small one-surface cavities.

Contraindications:

Provicol/Provicol C/Provicol QM contains modified colophony. **Provicol/Provicol C/Provicol QM** should therefore not be used in patients with a known hypersensitivity (allergy) to this ingredient.

Patient target group:

Provicol/Provicol C/Provicol QM is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Provicol/Provicol C/Provicol QM should only be used by a professionally trained dental practitioner.

Use:

Provicol QM (in the QuickMix syringe)

Due to technical reasons slight differences in the filling level between the syringe chambers can occur prior to the first use. Therefore remove syringe cap and squeeze the material out until the material is delivered evenly from both chambers. Place and lock the mixing tip type 10 with a 90° turn clockwise. The material is automatically mixed in the mixing tip.

All QM syringes come with a detachable piston. The piston can simply be pulled from the QM syringe and re-placed without causing any material leakage. The plugs in the syringe chambers remain in place preventing cross-contamination of the materials.

Provicol C (material in cartridges)

Insert **Provicol C** cartridge into the **VOCO** dispenser (type 2) or any other extruder gun with a suitable piston. Due to technical reasons, slight differences in the filling level between both cartridge chambers can occur. In this case, remove cartridge cap and squeeze the material out until the material is delivered evenly from both chambers. Then insert a mixing tip type 6 and lock with a 90° turn clockwise. By pressing the lever slowly and evenly, the material is extruded and automatically mixed in the correct ratio in the tip.

The attached mixing tip is intended for single use only. After use, store the cartridge (QM syringe) tightly sealed. Check that material flows freely through the openings before reusing.

The warranted product characteristics can only be achieved when using the supplied original mixing tips, which are also available as accessories.

Provicol (material in tubes)

Provicol should be used at room temperature (approx. 18 °C - 23 °C). Lower temperatures (use of a cooled glass plate) result in extended working times. The **Provicol** catalyst and base paste are mixed in a 1 : 1 w/w ratio. For that purpose, put two equal lengths of base and catalyst paste on a mixing pad of strong, non-absorbing material or a glass plate.

Then mix the two pastes for approximately 20 s until the material reaches a homogeneous consistency. Use a spatula made of plastic or stainless steel for mixing.

Application:

Apply a thin layer of **Provicol/Provicol C/Provicol QM** inside the temporary crown or bridge and affix it to the prepared tooth.

Provicol remains workable for about 1,5 - 2 min. **Provicol C/Provicol QM** about 1 min. All materials set in the mouth within 4 - 6 min.

Place the temporary into the mouth. Remove any excess material after 3 - 5 min.

Warnings, precautionary measures:

- In case of opened pulp or inferior dentine thickness, an adequate pulp protection is recommended.

- The sensitivity of cores can be reduced by use of desensitising preparations prior to temporary luting. Remove these preparations completely before definite luting (do not use preparations containing eugenol). Extensive superficial and multisurfaced cavities as well as temporary longterm sealing work should be treated with a harder cement.

- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Supplementary indications:

When using **Provicol** in tubes, it is recommended to moisten the cores with some water. Moisture (humid cores) additionally facilitates the removeability of the temporaries. Residues of **Provicol/Provicol C/Provicol QM** can be removed from crowns easily after they have been sprayed for a short time with a cooling spray.

Storage:

Store at 4°C-23°C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Provicol/Provicol C/Provicol QM** must be reported to VOCO GmbH and the responsible authority.

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VC 60 AA1074 E1 1222 99 © by VOCO