

Gelatamp

Resorbable hemostatic gelatin sponges for reliable and efficient bleeding control




	What is it?	When to Use?	Your Challenge, Our Solution	Practice Benefit	Patient Benefit
Gelatamp forte	High-absorbency gelatin sponge designed for rapid and effective bleeding control.	For surgical bleeding when sutures or ligatures are ineffective. To be used dry.	Provides rapid bleeding control, minimizes surgical time, and ensures hemostasis even in patients on anticoagulants.	Saves time during procedures, boosting operational efficiency.	Ensures safer treatments, especially for patients on anticoagulants or with clotting disorders.
Gelatamp with colloidal silver	Antimicrobial gelatin sponge with 5 % colloidal silver for the prophylaxis of wound infections and secondary haemorrhaging.	For tooth extractions, and prophylaxis of wound infections. To be used dry.	Prevention of inflammatory complications, stabilizes the coagulum and avoids impaired healing.	Reduces follow-ups and prevents post-extraction infections.	Promotes haemostasis with fewer complications.
Gelatamp white	Universal gelatin sponge offering reliable bleeding control for diverse surgical needs.	For general surgical procedures across various specialties. Can be used dry or soaked with sterile physiological solution.	Reliable bleeding control for routine surgeries, avoiding prolonged bleeding and complications.	Streamlines workflows for standard surgical needs.	Enhances care quality by reducing bleeding risks and postoperative pain.

All Gelatamp sponges reduce follow ups as they get resorbed and can be left in situ.

Gelatamp at a glance

	Feature	Sponge Size
Gelatamp forte	Due to its special sponge structure it absorbs quickly	10 × 10 × 4 mm
Gelatamp with colloidal silver	Effectively preventing wound infections	14 × 7 × 7 mm
Gelatamp white	Universal sponge for general surgical procedures.	10 × 10 × 10 mm


Speed of fluid absorption¹

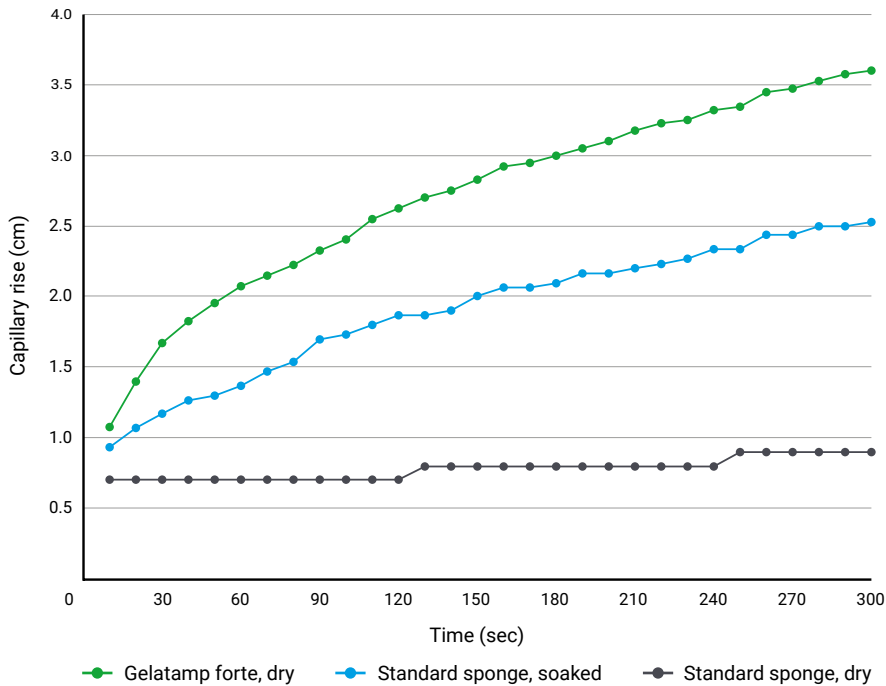


Special sponge structure for high absorbency

Gelatamp forte close-meshed and strong interconnections for high absorbency.

Gelatamp forte²

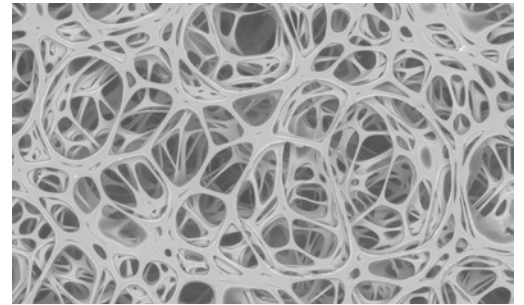

Gelatamp white²

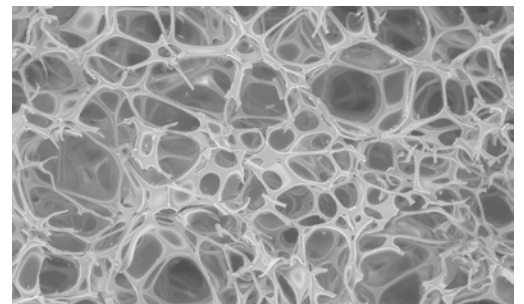

You have questions?
We are happy to support:
coltene.com/service/contact-us

COLTENE Group is a global leader in dental consumables and small equipment, specializing in natural teeth preservation and improvement. Originating from Altstätten, Switzerland, the group merges established brands, including Coltene, MicroMega, SciCan, and more, to deliver innovative solutions in restorative dentistry, endodontics, and infection control. Listed on the SIX Swiss Exchange, COLTENE Group is committed to enhancing dental workflows and patient care worldwide. For more information, visit coltene.com. For any further information please contact your preferred distributor or visit coltene.com. Please keep in mind that our products may not be available in all markets. Technical modifications on our products are not subject to notifications.

COLTENE Group
Feldwiesenstrasse 20
CH-9450 Altstätten
+41 71 757 5300 • info.ch@coltene.com

Coltène/Whaledent GmbH + Co. KG
Raiffeisenstraße 30
DE-89129 Langenau

¹ Source: Internal tests done with non sterile sponge material. Experimental setup capillary method: The sponge material was positioned on a glass plate with a 61° inclination and placed in a beaker filled with 10 ml dyed solution. The position of the rising liquid was determined at various points in time.

² Source: Internal SEM pictures taken from non sterile sponge material at 60 times magnification in secondary electron mode.

ORDER INFORMATION

Gelatamp forte
60036568 Blister pack, 24 pcs

Gelatamp with colloidal silver
274002 Box, 20 pcs
274007 Box, 50 pcs
274008 Blister pack, 20 pcs

Gelatamp white
60052842 Blister pack, 24 pcs